CLEARFIL™ POSTERIOR3

SELF-CURED ADHESIVE DENTAL RESTORATIVE FOR POSTERIOR TEETH

CE
0197

ENGLISH INSTRUCTIONS FOR USE

GENERAL INFORMATION

CLEARFIL POSTERIOR3 is a chemically hardening, radiopaque, hybrid composite material especially developed for the posterior region.

I. Indications

- Class I, II, V restorations of posterior teeth

II. Contraindication

- Patients with a history of hypersensitivity to methacrylate monomers

III. Precautions

[Safety precautions]

Allergic hypersensitivity

- Wear protective gloves or take other appropriate measures to prevent the occurrence of hypersensitivity due to contact with this product.
- If any hypersensitivity, such as a rash or dermatitis occurs, discontinue the use of the product and consult a doctor.

Contact with the human body

- Use caution to prevent the product from coming in contact with soft tissue or the skin, or getting into eye.
- If the product comes in contact with the human body, take the following measures:
 - If the product gets in the eye, wash the eye with copious amounts of running water and consult a doctor.
 - If the product comes in contact with soft tissue or the skin, wipe it away with a cotton pledge or gauze moistened with alcohol and wash with copious amounts of water.

[Storage precautions]

- Do not use after expiration date. Note expiration date on outside of package.
- The product must be refrigerated when not in use: bring it to room temperature before using.
- Keep away from extreme heat or direct sunlight.
- Keep the product out of reach of children.

IV. Clinical procedure

1. Check of occlusion and articulation pattern

The outline of the restoration is also dependent on the occlusion and articulation pattern, so this has to be studied carefully, before starting with the preparation.

The aim will be no occlusion points on the outline of the preparation.

2. Isolation

Rubber dam is recommended for isolation.

3. Cavity preparation

Determine the presence of caries and remove any infected dentin if present. CARIES DETECTOR is recommended to serve as a guideline for safe caries removal.

The form of the cavity preparation is dependent on the situation, e.g. primary caries or replacing of an amalgam restoration.

4. Placement of matrix band and wedges

In the case of a class II cavity, a transparent anatomic matrix band and light-reflecting wedges should be used.

5. Pulpal protection

Generally there is no need for cement lining or basing. However, any actual or near-pulpal exposure area should be covered with a hard setting calcium hydroxide material.

6. Etching

It is recommended to etch the cavity for 30 to 60 seconds with the aid of K-ETCHANT GEL.

Then thoroughly rinse the cavity with a spray for at least 20 seconds after which it must be carefully blow-dried. The enamel should now have a mat, white surface. From this moment onwards it is important for the cavity to remain absolutely dry and clean, in order to make an optimum adhesion process possible.


[NOTE]
Should contamination of the etched surface occur unexpectedly (for instance caused by blood or saliva), it is necessary for the entire etching procedure to be repeated.

7. Applying CLEARFIL NEW BOND

Mix two components, Universal Liquid and Catalyst Liquid in the proportion of 1:1.

Apply a thin coating of the mixture with a sponge on the entire wall of the cavity, the enamel and the dentine.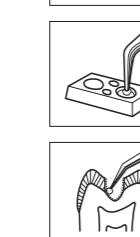

Evaporate the solvent gently and carefully with a flow of air. Place the mixed composite in the cavity within one minute.

8. Mixing the pastes

Take the relevant quantity of Universal Paste from the jar with a plastic spatula and put in the mixing plate. Take a similar quantity of Catalyst Paste with the other end of the spatula. Mix both pastes vigorously and thoroughly for 30 seconds in the mixing plate.

You can apply the composite into the cavity by means of hand instruments as well as the syringe technique. It is recommended to use synthetic matrix strips for this.

9. Applying the composite into the cavity

(at 25°C)
Mixing time Working time Setting time
30 seconds 120 seconds 60 seconds

10. Finishing and polishing

Contour the restoration and adjust occlusion and articulation using a fine diamond point.

Polish with silicon rubber points or polishing discs.

WARRANTY

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective.

Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

CLEARFIL is a trademark of Kuraray Co., Ltd.

It is classified as a Type 1 and Class 1 material by ISO 4049.

Ingredients

Ingredients ≥ 1 % by mass according to ISO 4049, and other ingredients:

Silanated silica filler/Silanated glass filler/
Silanated colloidal silica/
Bisphenol A diglycidylmethacrylate (Bis-GMA)/
Triethyleneglycol dimethacrylate (TEGDMA)/
Pre-polymerized organic filler/
Urethane tetramethacrylate/

The total amount of inorganic filler is approx. 67 vol%.
The particle size of inorganic fillers ranges from 0.04 µm to 40 µm.

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