LIGHT-CURING POLYMER-BASED CROWN AND BRIDGE MATERIAL

CHROMA ZONE™ COLOR STAIN

ENGLISH INSTRUCTIONS FOR USE

I. INTRODUCTION

CHROMA ZONE COLOR STAIN is a light-curing staining material for resin-based prosthetic material. The product is used during fabrication of crowns, onlays and inlays.

II. INDICATIONS

Staining of resin-based crowns, onlays and inlays

III. CONTRAINDICATIONS

Patients with a history of hypersensitivity to methacrylate monomers

IV. PRECAUTIONS

1. Safety precautions

- The product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylate monomers or any other components.
- If the patient demonstrates a hypersensitivity reaction such as rash, eczema, features of inflammation, ulcer, swelling, itching, or numbness, discontinue the use of the product, remove the product and seek medical attention.
- If the product comes in contact with human body tissues, take the following actions:
<If the product gets in the eye>
Immediately wash the eye with copious amounts of water and consult a physician.
<If the product comes in contact with the skin or the oral mucosa>
Immediately wipe the area with a cotton pledget or a gauze moistened with alcohol; rinse with copious amounts of water.
- Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomers.

2. Handling and manipulation precautions

- The product should not be used for purposes other than specified in [II.INDICATIONS]. Do not use the product to stain a denture base.
- The use of the product is restricted to a licensed dentist.
- Do not use the product in conjunction with any other stain materials.
- Avoid unnecessary exposure to direct sunlight or operatory lights; the material may harden during use.
- The product must be applied between the composite resins for crown and bridge. Do not apply the product to the finished resin surface directly or it may come off in a short period of time in the oral cavity, except the staining of pits and fissures in the posterior area.
- The product must not be mixed with composite resins to avoid the impaired physical properties.
- After application of the product, wash the brushes with ethyl alcohol or self-curing resin monomer solution.

3. Storage precautions

- The product must be used by the expiration date indicated on the package.
- The product must be stored at 2 - 25°C/ 36 - 77°F when not in use.
- Keep away from extreme heat or direct sunlight.
- The syringe should be tightly capped immediately after use to avoid hardening.
- The product must be stored in proper places where only dental practitioners can access it.

V. SHADES SYSTEM AND COMPONENTS

1. Shades of STAIN

The STAIN paste is available in 11 shades:

Clear (C), White (W), Pale Pink (PP), Yellow (Y), Russet Brown (RBR), Orange (O), Red (R), Brown (BR), Dark Brown (DBR), Blue (BLE) and Black (BL)

2. Components

Please see the outside of the package for contents and quantity.

- STAIN
- CLEAR PASTE
- Accessories
 - Color table
 - Mixing pad (S)
 - Disposable mixing dishes (8 wells)
 - Small brush (straight fine)

3. Ingredients

- STAIN: Clear (C), White (W) and Blue (BLE) shades

Principal ingredients:

- Triethyleneglycol dimethacrylate
- Aliphatic hexaacrylate
- Colloidal silica
- Silanated colloidal silica
- Initiators
- Pigments

- STAIN: Other shades

Principal ingredients:

- Triethyleneglycol dimethacrylate
- Aliphatic hexaacrylate
- Colloidal silica
- Silanated colloidal silica
- di-Camphorquinone
- Catalysts
- Accelerators
- Pigments

- CLEAR PASTE

Principal ingredients:

- Triethyleneglycol dimethacrylate
- Bisphenol A diglycidylmethacrylate
- Hydrophobic aliphatic dimethacrylate
- Urethane tetramethacrylate
- Pre-polymerized organic filler
- Silanated glass filler
- di-Camphorquinone

- Catalysts
- Accelerators

VI. CLINICAL PROCEDURES

VI-1. Curing unit and polymerization time

The recommended effective wavelength of dental curing unit is 400- 515 nm.

Table: Light curing time

Dental curing unit	Light curing time
Dentacolor XS	30 seconds

Table: Curing characteristics

Color	*Depth of cure (mm)
	Dentacolor-XS
Clear	4.52
White	1.25
Dark Brown	0.28
Clear Paste	1.73

* The depth of cure was measured by Kuraray Noritake Dental Inc. according to ISO 4049.

VI-2. Basic laboratory procedures

- Staining facing crown for crown and bridge materials
 - Fabrication of a metal frame
Fabricate a metal frame in the usual manner.
 - Application and light curing Opaque resin
Apply Body Opaque paste (e.g. EPRICORD, ESTENIA C&B) to the facing surface of metal frame. If necessary, apply Opaque Modifier to the cervical area to adjust the color and light cure it according to the manufacturer's instruction.
 - Forming and preliminarily light curing Cervical and Dentin
Form Cervical and Dentin paste, and light cure the paste preliminarily according to the manufacturer's instruction.
 - Shade selection of CHROMA ZONE COLOR STAIN
Select an appropriate shade of STAIN to adjust the color.
 - Staining Cervical and Dentin
Apply STAIN and light cure it with a dental curing unit (See table "Light curing time") for the specified length of time shown on the table "Light curing time".
 - Forming the Incisal edge
Form the incisal edge using a mixture of STAIN and CLEAR PASTE to reproduce the transparency required. And light cure it with a dental curing unit (See table "Light curing time") for the specified length of time shown on the table "Light curing time". Apply Amber paste thinly on the incisal edge and Transparent paste in the proximal area, then perform the final polymerization.
 - Completion
Shape and polish in the usual manner to complete the crown.
- Staining inlays and onlays
 - Fabrication of a working model
Fabricate a working model in the usual manner.
 - Application and drying of a resin separating agent
Apply a resin separating agent inside the cavity on the working model, and dry it.
 - Forming and light curing Dentin
Form Dentin paste (e.g. EPRICORD, ESTENIA C&B) on the cavity, and light cure the paste according to the manufacturer's instruction.
 - Shade selection of CHROMA ZONE COLOR STAIN
Select an appropriate shade of STAIN to adjust the color.
 - Staining Dentin
Apply STAIN to the occlusal area or the cervical area.
 - Light curing the STAIN
After the application, light cure STAIN with a dental curing unit (See table "Light curing time") for the specified length of time shown on the table "Light curing time".
 - Forming and curing Enamel
Apply Enamel paste to the cured STAIN, and light cure it according to the manufacturer's instruction.
 - Shaping and polishing
Shape and polish in the usual manner to complete the inlay or the onlay.
 - Staining pits and fissures
Stain the pits and the fissures, and light cure STAIN with a dental curing unit (See table "Light curing time") for the specified length of time shown on the table "Light curing time".

2. Staining inlays and onlays

- Fabrication of a working model
Fabricate a working model in the usual manner.
- Application and drying of a resin separating agent
Apply a resin separating agent inside the cavity on the working model, and dry it.
- Forming and light curing Dentin
Form Dentin paste (e.g. EPRICORD, ESTENIA C&B) on the cavity, and light cure the paste according to the manufacturer's instruction.
- Shade selection of CHROMA ZONE COLOR STAIN
Select an appropriate shade of STAIN to adjust the color.
- Staining Dentin
Apply STAIN to the occlusal area or the cervical area.
- Light curing the STAIN
After the application, light cure STAIN with a dental curing unit (See table "Light curing time") for the specified length of time shown on the table "Light curing time".
- Forming and curing Enamel
Apply Enamel paste to the cured STAIN, and light cure it according to the manufacturer's instruction.
- Shaping and polishing
Shape and polish in the usual manner to complete the inlay or the onlay.
- Staining pits and fissures
Stain the pits and the fissures, and light cure STAIN with a dental curing unit (See table "Light curing time") for the specified length of time shown on the table "Light curing time".

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proved to be defective.
Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

[NOTE]

"CHROMA ZONE", "EPRICORD" and "ESTENIA" are trademarks of KURARAY CO., LTD.
"Dentacolor" is a trademark of Heraeus Kulzer GmbH.

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Clear (C), White (W), Pale Pink (PP), Yellow (Y), Russet Brown (RBR), Orange (O), Red (R), Brown (BR), Dark Brown (DBR), Blue (BLE) et Black (BL)

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