CLEARFIL™ LINER BOND F

EN	ENGLISH	English	DENTAL BONDING AGENT
FR	FRANÇAIS	French	AGENT DE LIAISON DENTAIRE
ES	ESPAÑOL	Spanish	AGENTE DE ADHESIÓN DE USO DENTAL
IT	ITALIANO	Italian	AGENTE DI ADESIONE DENTALE
NL	NEDERLANDS	Dutch	DENTALE BONDING
DE	DEUTSCH	German	Dentaler Haftvermittler
SV	SVENSKA	Swedish	DENTALT BONDINGSMATERIAL
NO	NORSK	Norwegian	DENTALT BINDINGSMIDDEL
FI	SUOMI	Finnish	HAMPAAN SIDOSAINE
DA	DANSK	Danish	DENTALT BONDINGMATERIALE
PT	PORTUGUÊS	Portuguese	AGENTE DE ADESÃO DENTÁRIO
EL	ΕΛΛΗΝΙΚΑ	Greek	ΟΔΟΝΤΙΚΟΣ ΣΥΓΚΟΛΛΗΤΙΚΟΣ ΠΑΡΑΓΟΝΤΑΣ
TR	TÜRKÇE	Turkish	DENTAL BONDING AJAN
PL	POLSKI	Polish	STOMATOLOGICZNY SYSTEM WIAŻĄCY
RO	ROMÂNĂ	Romanian	AGENT DENTAR ADEZIV
HR	HRVATSKI	Croatian	DENTALNI ADHEZIV
HU	MAGYAR	Hungarian	FOGÁSZATI KÖTŐANYAG
SL	SLOVENŠČINA	Slovenian	DENTALNO VEZIVNO SREDSTVO
CS	ČESKY	Czech	DENTÁLNÍ POJIVO
SK	SLOVENSKY	Slovak	DENTÁLNE TMELIACE ČINIDLO
BG	БЪЛГАРСКИ	Bulgarian	СТОМАТОЛОГИЧЕН БОНДИНГ АГЕНТ
UK	УКРАЇНСЬКА	Ukrainian	СТОМАТОЛОГІЧНИЙ АДГЕЗИВ
ET	EESTI	Estonian	DENTAALNE ADHESIIV
LV	LATVIEŠU	Latvian	ZOBĀRSTNIECĪBAS SAISTVIELA
LT	LIETUVIŠKAI	Lithuanian	ODONTOLOGINIS RIŠIKLIS

Kuraray Noritake Dental Inc.

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

EMERGO EUROPE

Westervoortsedijk 60,
6827 AT Arnhem, The Netherlands

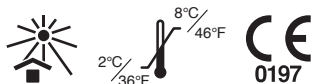
Kuraray Europe GmbH (EU Importer)

Philipp-Reis-Str. 4,
65795 Hattersheim am Main, Germany
Phone: +49 (0)69 305 35835 Fax: +49 (0)69 305 98 35835
URL: <https://www.kuraraynoritake.eu>

CLEARFIL™ LINER BOND F

I. INTRODUCTION

CLEARFIL LINER BOND F is a light-cure bonding system, and consists of a self-etching primer and a fluoride-releasing bonding agent. The primer simultaneously treats both dentin and enamel. The general clinical benefit of this product is to restore tooth function for the following INDICATIONS FOR USE.

II. INDICATIONS FOR USE

CLEARFIL LINER BOND F is indicated for the following uses :

- [1] Direct filling restorations using light-curing composite or compomer
- [2] Cavity sealing as a pretreatment for indirect restorations
- [3] Treatment of hypersensitive and/or exposed root surfaces
- [4] Intraoral repairs of fractured facing crowns made of porcelain, hybrid ceramics or composite resin using light-curing composite
- [5] Surface treatment of prosthetic appliances made of porcelain, hybrid ceramics and cured composite resin

III. CONTRAINDICATIONS

Patients with a history of hypersensitivity to methacrylate monomers

IV. POSSIBLE SIDE EFFECTS

The oral mucosal membrane may turn whitish when contacted by the product due to the coagulation of protein. This is usually a temporary phenomenon that will disappear in a few days. Instruct patients to avoid irritating the affected area while brushing.

V. INCOMPATIBILITIES

- [1] Do not use eugenol-containing materials for pulp protection or temporary sealing, since the eugenol can retard the curing process.
- [2] Do not use hemostatics containing ferric compounds since these materials may impair adhesion and may cause discoloration of the tooth margin or surrounding gingiva due to remaining ferric ions.

VI. PRECAUTIONS

1. Safety precautions

1. This product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylate monomers or any other components.
2. If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching or numbness, discontinue use of the product and seek medical attention.
3. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect them in the event of splashing material.
4. If the product comes in contact with human body tissues, take the following actions:
 - <If the product gets in the eye>
Immediately wash the eye with copious amounts of water and consult a physician.
 - <If the product comes in contact with the skin or the oral mucosa>
Immediately wipe the area with a cotton pellet or a gauze pad moistened with alcohol, and rinse with copious amounts of water.
5. Use caution to prevent the patient from accidentally swallowing the product.
6. Avoid looking directly at the dental curing light when curing the product.
7. Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomers or any other components.
8. Avoid using the same disposable brush tip for different patients to prevent cross infection. Discard the tip after use and sterilize the brush tip handle.
9. Dispose of this product as a medical waste to prevent infection.

2. Handling and manipulation precautions

[CLEARFIL LINER BOND F]

1. The product must not be used for any purposes other than specified in [II.INDICATIONS FOR USE].
2. PRIMER and BOND will set to a gel if left under an operating light or natural light (sunlight from windows). Use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 3 minutes from dispensing.
3. If the adherent surface extends to uncut enamel, apply a phosphoric acid (e.g. K-ETCHANT Syringe) to the enamel surface and leave it in place for 10 seconds, then wash and dry.
4. The use of this product is restricted to dental professionals.
5. To prevent poor performance or poor handling characteristics, observe the specified light-curing times and other handling requirements.
6. Clean the cavity sufficiently to prevent poor bonding. If the adherent surface is contaminated with saliva or blood, wash it thoroughly and dry before bonding.
7. Do not use BOND in conjunction with other bonding agents. Intermixing materials may cause a change in physical properties, including a possible decrease, from the properties expected results.
8. Do not use the mixture of PRIMER in conjunction with other silane coupling agents except CLEARFIL PORCELAIN BOND ACTIVATOR.
9. If the treated surface is contaminated, wash it with water, dry, or clean with alcohol, and treat again with the product.

[Dental light-curing unit]

1. Low light intensity causes poor adhesion. Check the lamp for service life and the dental curing light guide tip for contamination. It is advisable to check the dental curing light intensity using an appropriate light evaluating device at appropriate

intervals.

2. The emitting tip of the dental curing unit should be held as near and vertical to the resin surface as possible. If a large resin surface has to be light-cured, it is advisable to divide the area into several sections and light-cure each section separately.

[Outer case]

1. Do not hold the press points when removing the cap.
2. When dispensing PRIMER or BOND, hold the outer case and point it vertically downward.
3. Do not store the outer case with the bottom up.

3. Storage precautions

1. If the liquid may not flow easily, since it has not been used for a long term, shake the container before use.
2. The product must be used by the expiration date indicated on the package.
3. The product must be refrigerated (2-8°C/ 36-46°F) when not in use, and should be brought to room temperature before using. After taking out of the refrigerator, the product must be left standing until it comes to room temperature; otherwise the liquid might drop more than necessary or ooze after use.
4. The product must be kept away from extreme heat or direct sunlight.
5. The container should be tightly capped immediately after use.
6. The product must be stored in a proper place where only dental practitioners can access.

VII. COMPONENTS

Please see the outside of the package for contents and quantity.

1) PRIMER (self-etching primer)

Principal ingredients:

- 10-Methacryloyloxydecyl dihydrogen phosphate (MDP) (5-25%)
- 2-Hydroxyethyl methacrylate (HEMA) (20-40%)
- Hydrophilic aliphatic dimethacrylate (5-15%)
- dl-Camphorquinone (< 1%)
- Accelerators (< 6%)
- Water (40-60%)

2) BOND (fluoride-bonding agent)

Principal ingredients:

- 10-Methacryloyloxydecyl dihydrogen phosphate (MDP) (3-7%)
- Bisphenol A diglycidylmethacrylate (Bis-GMA) (25-45%)
- 2-Hydroxyethyl methacrylate (HEMA) (20-40%)
- Hydrophobic aliphatic dimethacrylate (10-30%)
- dl-Camphorquinone (< 2%)
- Accelerators(< 3%)
- Colloidal silica (1-10%)
- Surface treated sodium fluoride (< 1%)

Units in parentheses are mass %.

3) Accessories

- Disposable brush tips
- Brush tip handle*
- Mixing dish*
- Light blocking plate*
- Outer case*

[NOTE]

The PRIMER and BOND can easily be dispensed using the outer case container.
*Consumables

VIII. CLINICAL PROCEDURES

A. Standard procedure I

- [1] Direct filling restorations using light-curing composite or compomer
- [2] Cavity sealing as a pretreatment for indirect restorations
- [3] Treatment of hypersensitive and/or exposed root surfaces

A-1. Cleaning tooth structure

An adequately cleaned cavity assures maximum adhesive performance. Be sure the cavity is adequately cleaned.

A-2. Moisture control

In order to produce optimal results, avoid contamination of the treatment area from saliva or blood. A rubber dam is recommended to keep the tooth clean and dry.

A-3. Cavity preparations

Remove any infected dentin using caries detection material (e.g. CRIES DETECTOR) as a guide and prepare the cavity in the usual manner.

A-4. Pulp protection

Any actual or near pulp exposure could be covered with a hard setting calcium hydroxide material. There is no need for cement lining or basing. Do not use eugenol materials for pulp protection.

A-5. Acid etching uncut enamel

If the adherent surface extends to uncut enamel, apply a phosphoric acid (e.g. K-ETCHANT Syringe) to the enamel surface and leave it in place for 10 seconds, then wash and dry.

A-6. Tooth surface treatment

1. Dispense the necessary amount of PRIMER into a well of the mixing dish immediately before application.

[CAUTION]

PRIMER will set to a gel if left under an operating light or natural light (sunlight from windows). Use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 3 minutes from dispensing.

2. Apply PRIMER to the entire cavity wall with a disposable brush tip. Leave it in place for 20 seconds. Use caution not to allow saliva or exudate to contact the treated surfaces for at least 20 seconds.
3. After conditioning the tooth surface for 20 seconds, evaporate the volatile ingredients with a mild oil-free air stream.

[CAUTION]

Avoid pooling of PRIMER. Do not wash after applying PRIMER. Observe the drying method and treatment time to ensure optimum adhesion. Avoid touching the treated surface. If the treated surface is contaminated, wash it with water, dry, or clean with alcohol, and treat with PRIMER again.

A-7. Bonding

1. Dispense the necessary amount of BOND into a well of the mixing dish.

[CAUTION]

BOND will set to a gel if left under an operating light or natural light (sunlight from windows). Use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 3 minutes from dispensing.

2. Apply BOND to the entire surface of the cavity with a disposable brush tip.
3. After applying BOND, create a uniform bond film using a gentle oil-free air flow.

[CAUTION]

A strong air blast will scatter the bonding agent, resulting in poor adhesion.

4. Light-cure the BOND for 10 seconds with a dental curing light (irradiation wave length : 400-515nm, light intensity : >300mW/cm²).

A-8. Either A-8a or A-8b

A-8a. Direct restorations using light-cured composite resin (or compomer)

Apply composite resin (e.g. CLEARFIL Performance Pro) or compomer into the cavity, light-cure, finish and polish according to the manufacturer's instructions.

A-8b. Cavity sealing and treatment of hypersensitive teeth or exposed root surfaces

Apply a thin coat of flowable composite resin (e.g. CLEARFIL MAJESTY ES Flow) to the tooth, and light-cure the composite according to its' Instructions for Use. Remove unpolymerized resin with a cotton pellet moistened with alcohol.

B. Standard procedure II

- [4] Intraoral repairs of fractured facing crowns made of porcelain, hybrid ceramics or composite resin using light-curing composite

B-1. Preparation of fractured surfaces

- 1) Facing material surface
Using a diamond point, remove a layer of the fractured surface, and place a bevel at the marginal area.
- 2) Metal surface
Roughen the metal surface with a diamond point.

B-2. Acid etching of facing material surface

Apply a phosphoric acid (e.g. K-ETCHANT Syringe) to the facing material surface and leave it in place for 5 seconds, then wash and dry. If the adherent surface extends to uncut enamel, apply a phosphoric acid (e.g. K-ETCHANT Syringe) to the enamel surface and leave it in place for 10 seconds, then wash and dry.

B-3. Precious metal surface

When precious metal is used, apply a metal primer (e.g. ALLOY PRIMER) according to the manufacturer's instructions.

B-4. Silane treatment

1. Dispense one drop each of PRIMER and CLEARFIL PORCELAIN BOND ACTIVATOR into a well of the mixing dish and mix them.

[NOTE]

PRIMER and CLEARFIL PORCELAIN BOND ACTIVATOR should be mixed immediately before application.

[CAUTION]

PRIMER or the mixture will set to a gel if left under an operating light or natural light (sunlight from windows). Use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 3 minutes from dispensing.

2. Apply mixture to the facing material and metal surfaces with a disposable brush tip.
3. Leave it in place for 5 seconds. Use caution not to allow saliva or exudate to contact the treated surfaces.
4. If the adherent surface extends to the tooth, leave mixture in place for 20 seconds.
5. After that, evaporate the volatile ingredients with a mild oil-free air stream. The treated metal surface must be sufficiently dried; if not, adhesion might be seriously impaired.

[CAUTION]

Do not wash treated surface. If the treated surface is contaminated with saliva, wash it with water, dry, or clean with alcohol, and treat with the mixture again.

B-5. Bonding

1. Dispense the necessary amount of BOND into a well of the mixing dish.

[CAUTION]

BOND will set to a gel if left under an operating light or natural light (sunlight from windows). Use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 3 minutes from dispensing.

2. Apply BOND to the entire cavity wall with a disposable brush tip.
3. After applying BOND, create a uniform bond film using a gentle oil-free air flow.
4. Light-cure the BOND for 10 seconds with a dental curing light (irradiation wave length : 400-515nm, light intensity : >300mW/cm²).

B-6. Light-curing composite filling

Apply composite resin (e.g. CLEARFIL Performance Pro) into the cavity, light-cure, finish and polish according to the manufacturer's instructions.

[NOTE]

Use opaque resin such as CLEARFIL ST OPAQUER for metal surface to prevent metal shine through.

C. Standard procedure III

- [5] Surface treatment of prosthetic appliances made of porcelain, hybrid ceramics and cured composite resin.

Silanating treatment of prosthetic appliances made of esthetic materials (porcelain, hybrid ceramics or composite resin) can be accomplished using PRIMER in conjunction with CLEARFIL PORCELAIN BOND ACTIVATOR.

C-1. Acid etching of adherent surface

Apply a phosphoric acid (e.g. K-ETCHANT Syringe) to the adherent surface and leave it in place for 5 seconds, then wash and dry.

C-2. Silane treatment

Use the same procedure described in B-4.

C-3. Cementing

Cement the prosthetic appliance using PANAVIA cement according to the manufacturer's instructions.

[NOTE]

If a serious incident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user/patient resides.

[NOTE]

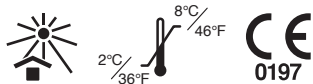
"CLEARFIL", "PANAVIA" and "CLEARFIL MAJESTY" are registered trademarks or trademarks of KURARAY CO., LTD.

004 1562R730R-EN 03/2024

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

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„CLEARFIL“, „PANAVIA“ ir „CLEARFIL MAJESTY“ yra „KURARAY CO., LTD“ registruotieji prekės ženklai arba prekės ženklai.