

CLEARFIL™ DC CORE PLUS & BOND KIT

Easy to use procedure

Apply without effort

Dual-cure



THE COMPLETE SOLUTION FOR CORE BUILD-UPS

CLEARFIL™ DC CORE PLUS

1

SIMPLE, STRESS-FREE PROCEDURE

Make every core build-up straightforward. Its easy-to-use design streamlines your workflow from start to finish.

2

DUAL-CURE CONFIDENCE

Achieve complete polymerization in all areas - even where light cannot reach - for a durable foundation.

3

OPTIMAL HANDLING & PRECISION CONTROL

Advanced thixotropy allows smooth flow under pressure while staying exactly where you place it.

4

SHAPE-STABLE, NON-SLUMPING FORMULA

Maintains its form during placement and sculpting, giving you precise core shaping every time.

5

OPTIMISED WORKING TIME

Approximately three minutes of working time lets you complete post cementation and core build-up efficiently with a single mixing tip.



BUILT FOR YOUR RELIABILITY

CLEARFIL™ Universal Bond Quick 2

QUICK AND STRAIGHTFORWARD PROCEDURE

With CLEARFIL™ Universal Bond Quick 2, the streamlined procedure enables quick application without any waiting time or the need for rubbing. It supports a short treatment time in core build-up procedures.

EASY APPLICATION


CLEARFIL™ Universal Bond Quick 2 is very easy to apply. The new filler dispersion method used makes it possible to enhance the product's flowability without lowering the filler content. Enhanced flowability provides for even spreading, creating a thin, uniform bonding layer. This minimizes the risk of pooling even in root canals.

VERSATILITY FOR VARIOUS APPLICATIONS

Suitable for core build-up procedures, as well as both direct and indirect restorations, this adhesive provides the flexibility needed for a wide range of clinical scenarios. It is also compatible with self-adhesive cements from other manufacturers, offering you greater choice.



Apply¹

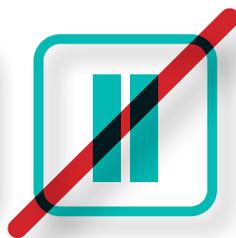

No waiting time



Dry



Light cure

CLEARFIL™ Universal Bond Quick 2 offers outstanding adhesion, making it a suitable match for CLEARFIL™ DC CORE PLUS. It adheres to dentin in the root channel and to the post. A separate primer is not needed.




¹ Apply with a rubbing motion and proceed

STRENGTH THAT WORKS WITH NATURE

CLEARFIL™ DC CORE PLUS is engineered to work in harmony with natural dentin. Its elastic modulus closely matches dentin, allowing occlusal forces to distribute evenly throughout the root canal rather than concentrating at a single point.


With its high mechanical strength, the core build-up remains stable under chewing stress, minimizing the risk of fractures in both the root and the restoration. The result is an outstanding foundation that gives clinicians confidence in every restoration.

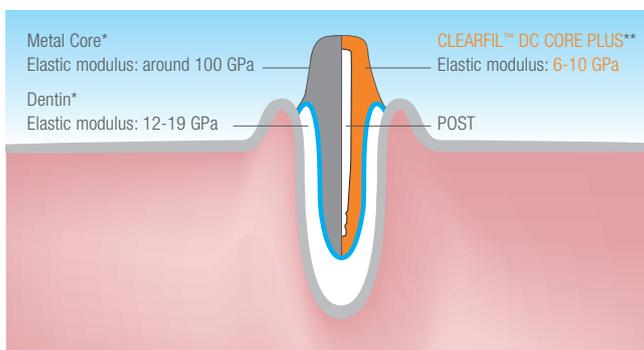

FLOWS WITH CONTROL

The paste dispenses smoothly and is easy to control, thanks to the unique filler surface treatment technology of CLEARFIL™ DC CORE PLUS.


Paste formability

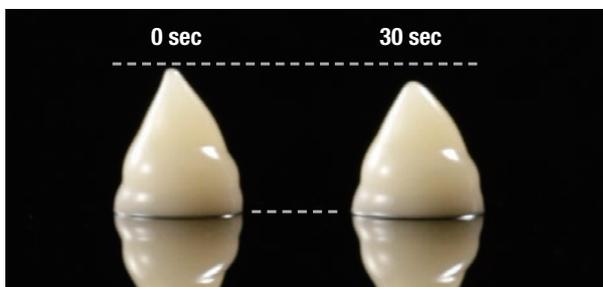


DUAL-CURING PROPERTIES FOR CORE BUILD-UPS

CLEARFIL™ Universal Bond Quick 2 is available for use in combination with CLEARFIL™ DC CORE PLUS. The specific accelerator for self-curing contained in CLEARFIL™ DC CORE PLUS PLUS provides for excellent results for areas that cannot be reached by light.

CLEARFIL™ Universal Bond Quick 2 creates a strong chemical bond to hydroxyapatite. Thanks to the Original MDP monomer, which is known for high bond strength and an excellent adhesion durability to the tooth structure.

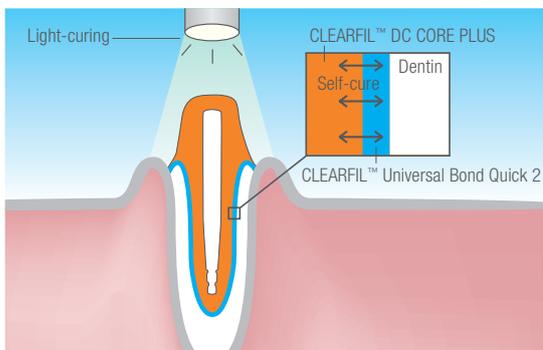


STRUCTURE OF ADHESIVE MONOMER MDP

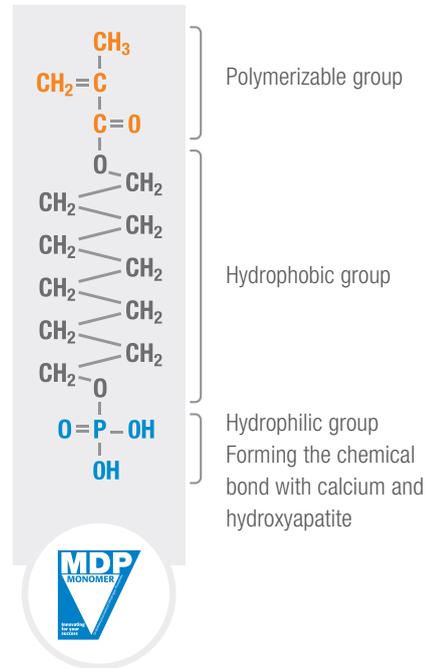

STRIKING BOND STRENGTH CONFIRMED!

Shear Bond Strength to bovine dentin

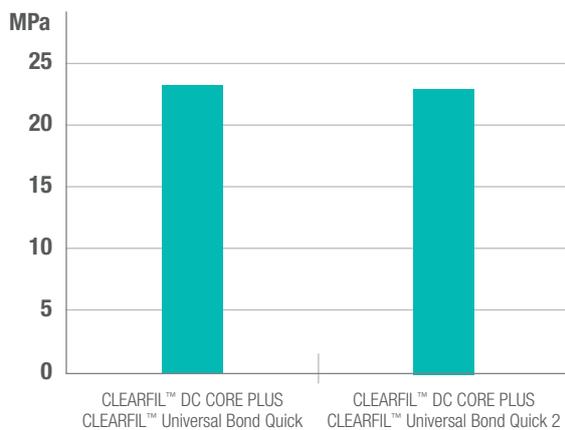

Test conditions:
Adherent surface: bovine dentin #600 polished surface,
Coated area: 2.38 mm dia. Irradiator: PenCure 2000 standard mode.
Stored in water at 37°C for 24 hours.

Source: Kuraray Noritake Dental Inc.

EASY TO USE, STRESS-LESS CLINICAL PROCEDURE

Post cementation and core build-up using CLEARFIL™ DC CORE PLUS according to the manufacturer's instructions for use.



Apply BOND with a rubbing motion.



Dry by blowing mild air and paper point until BOND does not move.



Light-cure.



Squeeze PASTE.



Insert the post and light-cure PASTE.



Place PASTE around the post.

NOTE: This is an abbreviated version. Please follow the Instructions for Use supplied with the product.

SHADES

CLEARFIL™ DC CORE PLUS & BOND Kit is available in two shades:



Dentin



White

PRODUCT ASSORTMENT

CLEARFIL™ DC CORE PLUS & BOND Kit

1 syringe (9 ml), CLEARFIL™ Universal Bond Quick 2: 1 bottle (1 ml)
Accessories: 20 mixing tips, 10 guide tips (L), 10 guide tips (S),
50 applicator brushes (endo), 1 dispensing dish, 1 light blocking plate

#4049-EU DENTIN

#4050-EU WHITE



CLEARFIL™ DC CORE PLUS Refill

1 syringe (9 ml), 20 mixing tips, 10 guide tips (L), 10 guide tips (S)

#2942-EU DENTIN

#2943-EU WHITE



CLEARFIL™ Universal Bond Quick 2 Bottle Refill

(5 ml)

#4042-EU



CLEARFIL™ Universal Bond Quick 2 Unit Dose Standard Pack

Unit Dose (50 pcs x 0.1 ml), Applicator Brush (x 50 pcs)

#4046-EU



CLEARFIL™ Universal Bond Quick 2 Bottle Value Pack

(3 x 5 ml)

#4044-EU



K-ETCHANT SYRINGE

K-ETCHANT Syringe (2 x 3 ml), Needle Tip (x 40 pcs)

#3252-EU

NEEDLE TIPS

Needle Tip (x 20 pcs)

#3253-EU





EU Importer

Kuraray Europe GmbH

Philipp-Reis-Strasse 4,
65795 Hattersheim am Main, Germany

Phone +49 (0)69 305 35 835

Fax +49 (0)69 305 98 35 835

www.kuraraynoritake.eu

centralmarketing@kuraray.com

- Before using this product, be sure to read the Instructions for Use supplied with the product.
- The specifications and appearance of the product are subject to change without notice.
- Printed color can be slightly different from actual color.

“CLEARFIL” is a registered trademark or trademark of KURARAY CO., LTD.



Kuraray Noritake Dental Inc.

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan
Website www.kuraraynoritake.com



01/2026 MKA00551