CARIES DETECTOR

EN	ENGLISH	English	CARIES DETECTION DEVICE
FR	FRANÇAIS	French	DISPOSITIF DE DÉTECTION DES CARIES
ES	ESPAÑOL	Spanish	DISPOSITIVO DETECTOR DE CARIES
IT	ITALIANO	Italian	DISPOSITIVO DI RILEVAZIONE DELLA CARIE
NL	NEDERLANDS	Dutch	CARIES DETECTOR
DE	DEUTSCH	German	KARIESDETEKTOR
SV	SVENSKA	Swedish	KARIESDETEKTERINGSSYSTEM
NO	NORSK	Norwegian	KARIESDETEKTOR
FI	SUOMI	Finnish	KARIESINDIKAATTORI
DA	DANSK	Danish	CARIES-DETEKTOR
PT	PORTUGUÊS	Portuguese	DETETOR DE CÁRIES
EL	ΕΛΛΗΝΙΚΑ	Greek	ΣΥΣΚΕΥΗ ΕΝΤΟΠΙΣΜΟΥ ΤΕΡΗΔΟΝΑΣ
TR	TÜRKÇE	Turkish	ÇÜRÜK TEŞHİS CİHAZI
PL	POLSKI	Polish	DETEKTOR PRÓCHNICY
RO	ROMÂNĂ	Romanian	DETECTOR DE CARII
HR	HRVATSKI	Croatian	UREĐAJ ZA DETEKCIJU KARIJESA
HU	MAGYAR	Hungarian	FOGSZUVASODÁS-DETEKTOR
SL	SLOVENŠČINA	Slovenian	NAPRAVA ZA ODKRIVANJE KARIJESA
CS	ČESKY	Czech	DETEKTOR KAZŮ
SK	SLOVENSKY	Slovak	DETEKTOR ZUBNÉHO KAZU
BG	БЪЛГАРСКИ	Bulgarian	УСТРОЙСТВО ЗА ОТКРИВАНЕ НА КАРИЕСИ
UK	УКРАЇНСЬКА	Ukrainian	ДЕТЕКТОР КАРИЄСУ
ET	EESTI	Estonian	KAARIESE DETEKTOR
LV	LATVIEŠU	Latvian	KARIESA NOTEIKŠANAS IERĪCE
LT	LIETUVIŠKAI	Lithuanian	ĖDUONIES APTIKIMO ĮTAISAS

Kuraray Noritake Dental Inc.

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

EMERGO EUROPE

Westervoortsedijk 60,
6827 AT Arnhem, The Netherlands

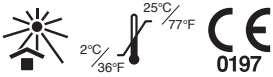
Kuraray Europe GmbH (EU Importer)

Philipp-Reis-Str. 4,
65795 Hattersheim am Main, Germany
Phone:+49 (0)69 305 35835 Fax:+49 (0)69 305 98 35835
URL:<https://www.kuraraynoritake.eu>

1561J259R

CARIES DETECTOR

I. INTRODUCTION

CARIES DETECTOR clearly differentiates clinically the two layers of carious dentin by staining the outer carious dentin scarlet red yet not staining either the inner carious dentin or normal dentin. The distinct staining of only the outer, infected carious layer provides a visible, clinically precise guide for caries removal. The general clinical benefit of this product is to restore tooth function for the following INDICATIONS FOR USE.

II. INDICATIONS FOR USE

CARIES DETECTOR is indicated for the following uses:

- Detecting carious dentine for cavity preparation

III. PRECAUTIONS

1. Safety precautions

1. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using product, cover the patient eyes with a towel to protect the patient's eyes from splashing material.
2. If the product comes in contact with the human body, take the following actions:
 - <If the product gets in the eye>
Immediately wash the eye with copious amounts of water and consult a physician.
 - <If the product comes in contact with the skin>
Immediately wipe it off with a cotton pledget moistened with alcohol or gauze and wash with copious amounts of water.
 - <If the product comes in contact with the mucous membrane>
The product might stay between other medical devices (e.g.cotton rolls) and the mucous membrane. Immediately wipe it off with cotton pledget moistened with alcohol or gauze and wash with copious amounts of water.
3. Use caution when using the product to prevent swallowing.
4. Do not use the same disposable brush for different patients to prevent cross contamination. Discard the brush after use.
5. Avoid direct contact with the skin and / or soft tissue to prevent hypersensitivity. Wear gloves or take appropriate precautions when using the product.
6. Do not apply the product directly from the bottle to carious cavity.
7. Dispose of this product as a medical waste to prevent infection.

2. Handling and manipulation precautions

1. The use of this product is restricted to dental professionals.
2. Provide moisture and contamination control with the use of a rubber dam.
3. Do not mix the product with any other dental materials.
4. Do not use the product for any purposes other than those specified in INDICATIONS FOR USE.

3. Storage precautions

1. The product must be used by the expiration date indicated on the package.
2. The product must be stored at 2 - 25°C / 36 - 77°F when not in use.
3. Keep away from extreme heat or direct sunlight.
4. The product must be stored in proper places where only dental practitioner can access it.

IV. COMPONENTS

Please see the outside of the package for contents and quantity.

1. Components

CARIES DETECTOR

2. Ingredients

- Propane-1,2-diol
- Dye

V. CLINICAL PROCEDURES

- (1) Isolate carious tooth with a rubber dam prior to application of the product.
- (2) After washing and drying the carious tooth, place one drop of the product on a disposable brush and apply it to the cavity.
- (3) After 10 seconds, wash it with water.
- (4) Remove the exposed outer carious dentin which is stained red.
- (5) Repeat detection and reduction a few more times to insure complete removal of the outer carious dentin.

[NOTE]

1. Recommended method for removal of carious dentin:
 - Only the non-sensitive stainable tissue should be removed, carefully avoiding cutting the sensitive unstainable tissue.
 - A low-speed rotary instrument should be used for carious dentin removal since the friction heat generated by a high-speed tool may conduct pain sensation even before reaching the sensitive unstained tissue.
2. Cavity preparation procedure using the product:

The product can be used in two ways:

 - The product is applied to the cavity area before opening the cavity. This procedure is suggested for a shallow open cavity because earlier disclosure of undermined free enamel may facilitate opening the cavity.
 - The product is used to disclose the residual outer carious dentin after opening a cavity and removing heavily discolored and highly softened dentin. This procedure is suggested for a deep cavity with a large amount of carious dentin for expedience. Careless gross reduction before application of the product, however, may cause pain from unintentional over-reduction.

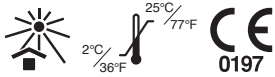
[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proved to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

[NOTE]

If a serious incident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user/patient resides.

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