EN	ENGLISH	English	Dental Ceramic
FR	FRANÇAIS	French	Céramique Dentaire
ES	ESPAÑOL	Spanish	Cerámica Dental
IT	ITALIANO	Italian	Ceramica Dentale
NL	NEDERLANDS	Dutch	DENTAAL PORSELEIN
DE	DEUTSCH	German	Dentalkeramik
SV	SVENSKA	Swedish	Dentalkeramik
NO	NORSK	Norwegian	Dentalkeramikk
FI	SUOMI	Finnish	Hammaskeramiikka
DA	DANSK	Danish	Dental Keramik
PT	PORTUGUÊS	Portuguese	Cerâmica Dentária
EL	ΕΛΛΗΝΙΚΑ	Greek	Οδοντικό Κεραμικό Υλικό
TR	TÜRKÇE	Turkish	Dental Seramik
PL	POLSKI	Polish	Dentystyczny materiał ceramiczny
RO	ROMÂNĂ	Romanian	Ceramică dentară
HR	HRVATSKI	Croatian	Dentalna keramika
HU	MAGYAR	Hungarian	Fogászati kerámia
SL	SLOVENŠČINA	Slovenian	Dentalna keramika
CS	ČESKY	Czech	Dentální keramika
SK	SLOVENSKY	Slovak	Dentálna keramika
BG	БЪЛГАРСКИ	Bulgarian	Стоматологична керамика
UK	УКРАЇНСЬКА	Ukrainian	Стоматологична кераміка
ET	EESTI	Estonian	Dentaalkeraamika
LV	LATVIEŠU	Latvian	Zobārstniecības keramika
LT	LIETUVIŠKAI	Lithuanian	Odontologinė keramika

<b>UK</b> <b>REP</b>	<b>EN</b>	UK Responsible Person	<b>PL</b>	Osoba odpowiedzialna w Wielkiej Brytanii
	<b>FR</b>	Personne responsable au Royaume-Uni	<b>RO</b>	Persoana responsabilă din Marea Britanie
	<b>ES</b>	Responsable en el Reino Unido	<b>HR</b>	Osoba odgovorna za Ujedinjeno Kraljevstvo (UK)
	<b>IT</b>	Responsabile per il Regno Unito	<b>HU</b>	Felelős személy az Egyesült Királyságban
	<b>NL</b>	Verantwoordelijke in het VK	<b>SL</b>	Odgovorna oseba v Združenem kraljestvu
	<b>DE</b>	Für UK verantwortliche Person	<b>CS</b>	Odpovědná osoba ve Spojeném království
	<b>SV</b>	Ansvarig person i Storbritannien	<b>SK</b>	Zodpovedná osoba v Spojenom kráľovstve
	<b>NO</b>	Ansvarlig person i Storbritannia	<b>BG</b>	Отговорно лице за Обединеното кралство
	<b>FI</b>	Vastuushenkilö Yhdistyneessä kuningaskunnassa	<b>UK</b>	Відповідальна особа у Великій Британії
	<b>DA</b>	Ansvarlig person i Det Forenede Kongerige	<b>ET</b>	Suurbritannia vastutav isik
	<b>PT</b>	Responsável no Reino Unido	<b>LV</b>	Apvienotās Karalistes atbildīgā persona
	<b>EL</b>	Υπεύθυνο Πρόσωπο για το Ηνωμένο Βασίλειο	<b>LT</b>	Atsakingas asmuo Jungtinėje Karalystėje
	<b>TR</b>	Birleşik Krallık Sorumlusu		

**Kuraray Noritake Dental Inc.**

300 Higashiyama, Miyoshi-cho, Miyoshi, Aichi 470-0293, Japan  
TEL +81-561-32-8953 FAX +81-561-32-8976

**EMERGO EUROPE**

Westervoortsedijk 60, 6827 AT Arnhem, The Netherlands

**Kuraray Europe GmbH (EU Importer)**

Philipp-Reis-Str. 4,  
65795 Hattersheim am Main, Germany  
Phone:+49 (0)69 305 35835 Fax:+49 (0)69 305 98 35835  
URL:https://www.kuraraynoritake.eu

# CERABIEN™ MiLai

## I. Introduction

This product is a dental porcelain, which has a potassium-aluminosilicate glass formulation as its primary chemical component. This product is suitable for a wide range of clinical applications, with variable shades. The general clinical benefit of this product is to restore tooth function in the following Indications for Use.

## II. Indications for Use

For use in prosthetic dentistry to create an all-ceramic prosthesis.

- Ceramic for coverage of a zirconia substructure
- Ceramic for coverage of a lithium disilicate substructure  
[CTE: 9.5-11.0x10<sup>-6</sup>/K (50-500°C/122-932°F)]

### [Intended purpose]

It is used as a dental porcelain for fabricating dental restorations.

### [Intended user]

- Dentists (chairside fabrication of restorations; clinical workflow)
- Dental laboratory technicians (fabrication of restorations in the dental laboratory)

## III. Components

For a complete list of the components (including shades and accessories) for this product, see "CERABIEN™ MiLai Baking Schedule".

## IV. Composition

Potassium-aluminosilicate glass (>70 %), inorganic pigments (0-30 %)

LF LIQUID: Water (>99 %), oxidizing agents (<1 %)

IS LIQUID: 2-Phenoxyethanol (>99 %)

Units in parentheses are mass %.

## V. Type and Class (ISO6872: 2015)

Type:1 / Class:1

## VI. Physical Properties

See Table 2.

## VII. Directions for Use

See Table 1 for the Baking schedule.

See Table 3 for instructions on the specified liquids.

1. Preparation of zirconia framework
  - Condition and adjust the ceramic framework according to the instructions provided with the product.
2. Checking for cracks and cleaning
  - Check the zirconia framework for cracks.
  - After making sure the framework does not have any cracks, sandblast the surface with aluminum oxide (50µm) at a pressure of 0.2MPa (30psi / 2 Bar) or less, followed by approximately 10 minutes of ultrasonic cleaning.
  - If cracks are found, do not use the zirconia framework.
3. Wash baking
 

Thin, one-layer application of product used as the first layer, and baking is performed. (For baking described in this instructions for use, follow the baking schedule.)
4. Color adjustment
 

Internal Stain can be used to make color adjustments and characterization, if necessary. Bake it according to the Baking Schedule for Internal Stain listed in Table 1.
5. Building-up/ Application, Baking
  - Build-up or apply to the desired part and bake according to the baking schedule listed on Table 1.
  - After baking, the surface of a correctly baked restoration should have a slight luster or sheen.
6. Adjusting for morphological corrections, glazing, and completion
  - Correct and contour the morphological shape of the restoration, followed by cleaning and glazing. Stains and glazes on surfaces should be applied using "CERABIEN™ ZR" FC Paste Stain.

## VIII. Remarks on Handling

### Contraindications:

If the patient is hypersensitive to potassium-aluminosilicate glass or any other components, this product must not be used.

### Caution and Warning:

1. This product should not be used when malocclusion, clenching or bruxism conditions are present.
2. If the patient or the dental professional demonstrates a hypersensitivity reaction, such as rash, dermatitis, etc., discontinue use of the product and seek medical attention immediately.
3. When mixing materials, or grinding a restoration being fabricated, use an approved dust mask and vacuum with an air filter to protect your lungs from inhaling either dust or liquid.
4. When mixing materials or grinding a restoration being fabricated, use safety glasses to prevent dust or liquid from getting into your eyes. If dust or liquid gets into your eyes, immediately rinse with copious amounts of water and seek medical attention.
5. Do not use for any purposes except for fabricating dental restorations. This product is for dental application only.
6. The use of this product is restricted to dental professionals.
7. The restoration is very hot immediately after baking. Do not touch it with your bare hands for a considerable amount of time after removing it from the furnace.
8. IS LIQUID is flammable. Do not use near fire or high-temperature objects.
9. IS LIQUID is a stimulant liquid. When using it, have good ventilation and if necessary, wear a protective mask, safety glasses and gloves.
10. Stains and plaque can accumulate on a restoration after it has been placed in the patient's oral cavity, depending on the patient's eating habits or food choices. Instruct patients about the importance of cleaning their restorations.

### Precautions in use:

1. Do not mix with other dental porcelains at any stage of the clinical procedure.
2. Porcelain furnace temperature variations may be more significant in the lower ranges. Determine the exact baking schedule by making a preliminary test before baking an actual case restoration.

3. The correct baking temperature for this product varies with the size of the restorations to be baked. Adjust the furnace's baking temperature according to the baking schedule.
4. Internal Stain should be mixed with the specified liquid. (Use of liquids other than those specified may result in a darker restoration color than desired, after baking.)
5. This product is a low-temperature baking porcelain. To prevent discoloration of the porcelain, observe the following conditions:
  - If tissue paper fibers get incorporated into the porcelain slurry during water sorption in the condensation procedure, the fibers will not burn off completely. After drying, check to make sure that no residual tissue fibers remain before baking.
  - Always use freshly mixed porcelain.
6. The layers applied with this product should not exceed the thickness of the ceramic framework.
7. Keep the interior of the porcelain furnace clean by periodically firing it above 1,000 °C (1,832°F) while empty.
8. Dispose of this product as a medical waste to prevent infection.
9. Do not use Internal Stain externally.
10. Do not use IS LIQUID near fire or high-temperature objects.

### Storage:

1. The product must be stored in a cool and dry place. Keep away from direct sunlight.
2. The product should be stored at 1-30°C (33.8-86°F).
3. The product must be stored in an appropriate place where only dental personnel have access.
4. The product must be used by the expiration date indicated on the package.
5. Do not store IS LIQUID near fire or high-temperature objects.

### [WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

### [NOTE]

1. If a serious incident attributable to this product occurs, report it to the manufacturer through the EU Importer, and to the regulatory authorities of the country in which the user/patient resides.
2. Refills do not include instructions for use (IFU). We provide the IFU in electronic format on the dedicated eIFU website.
3. "CERABIEN" is a registered trademark or trademark of NORITAKE CO., LIMITED.

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\*1 96 kPa = 72 cmHg (29 inchesHg)

\*2 The lower baking limit is based on the assumption of a single crown. When baking multiple or large restorations, adjust the dry-out time and temperature according to the upper baking limits.

\*3 This is the baking temperature for cases where Liner, Opacious Body, Body, Enamel, Translucent, Luster, Clear Cervical, and Mamelon are additionally applied.

Note: The above program is only a guideline. Baking temperature may be varied with the peculiarities of different furnace.

Porcelain furnace temperature variations may be more significant in the lower ranges. Determine the exact baking schedule by making a preliminary test before baking an actual case restoration.

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Table 3. Specified liquids

Type	Liquid
Internal Stain	IS LIQUID
Porcelain	LF LIQUID or purified water