# CLEARFIL MAJESTY™ ES-2

<b>EN</b>	ENGLISH	English	DENTAL LIGHT-CURED RESTORATIVE COMPOSITE
<b>FR</b>	FRANÇAIS	French	COMPOSITE DE RESTAURATION DENTAIRE PHOTOPOLYMERISABLE
<b>ES</b>	ESPAÑOL	Spanish	COMPOSITE DE RESTAURACIÓN FOTOPOLIMERIZABLE DE USO DENTAL
<b>IT</b>	ITALIANO	Italian	COMPOSITO FOTOPOLIMERIZZABILE PER RESTAURI DENTALI
<b>NL</b>	NEDERLANDS	Dutch	DENTAAL LICHTUITHARDEND RESTAURATIEF COMPOSIT
<b>DE</b>	DEUTSCH	German	DENTALES lichthärtendes Komposit-Restaurationsmaterial
<b>SV</b>	SVENSKA	Swedish	DENTAL LJUSHÄRDANDE RESTAURERINGSKOMPOSIT
<b>NO</b>	NORSK	Norwegian	DENTAL LYSHERDET RESTAURERINGSKOMPOSIT
<b>FI</b>	SUOMI	Finnish	HAMPAAN VALOKOVETTEINEN YHDISTELMÄMUOVI
<b>DA</b>	DANSK	Danish	DENTAL LYSHÆRDENDE RESTAURERINGSKOMPOSIT
<b>PT</b>	PORTUGUÊS	Portuguese	COMPÓSITO FOTOPOLIMERIZAVEL PARA USO DENTÁRIO
<b>EL</b>	ΕΛΛΗΝΙΚΑ	Greek	ΟΔΟΝΤΙΚΗ ΦΩΤΟΠΟΛΥΜΕΡΙΖΟΜΕΝΗ ΣΥΝΘΕΤΗ ΡΗΤΙΝΗ ΑΠΟΚΑΤΑΣΤΑΣΗΣ
<b>TR</b>	TÜRKÇE	Turkish	IŞIKLA SERTLEŞEN DENTAL RESTORATİF KOMPOZİT
<b>PL</b>	POLSKI	Polish	ŚWIATŁOUTWARDZALNY DENTYSTYCZNY KOMPOZYT DO UZUPEŁNIEŃ
<b>RO</b>	ROMÂNĂ	Romanian	COMPOZIT DE RESTAURARE DENTARĂ FOTOPOLIMERIZABIL
<b>HR</b>	HRVATSKI	Croatian	DENTALNI SVJETLOSNO POLIMERIZIRANI RESTAURACIJSKI KOMPOZIT
<b>HU</b>	MAGYAR	Hungarian	FOGÁSZATI, FÉNYRE KEMÉNYEDŐ RESTAURÁCIÓS KOMPOZIT
<b>SL</b>	SLOVENŠČINA	Slovenian	DENTALNI SVETLOBNO STRJUJOČI KOMPOZITNI MATERIAL ZA RESTAVRACIJE
<b>CS</b>	ČESKY	Czech	DENTÁLNÍ SVĚTLEM TUHNOUCÍ KOMPOZITNÍ MATERIÁL PRO NÁHRADY
<b>SK</b>	SLOVENSKY	Slovak	DENTÁLNY SVETLOM VYTVRDZOVANÝ VÝPLŇOVÝ KOMPOZIT
<b>BG</b>	БЪЛГАРСКИ	Bulgarian	СТОМАТОЛОГОЧЕН ФОТОПОЛИМЕРИЗИРАЩ ВЪЗСТАНОВИТЕЛЕН КОМПОЗИТ
<b>UK</b>	УКРАЇНСЬКА	Ukrainian	СТОМІТОЛОГІЧНИЙ ФОТОПОЛІМЕРНИЙ РЕСТАВРАЦІЙНИЙ КОМПОЗИТ
<b>ET</b>	EESTI	Estonian	DENTAALNE VALGUSKÕVASTUV KOMPOSIIT-RESTAURATSIOONIMATERJAL
<b>LV</b>	LATVIEŠU	Latvian	ZOBĀRSTNIECĪBAS GAISMĀ CIETĒJOŠS RESTAURĀCIJAS KOMPOZĪTMATERIĀLS
<b>LT</b>	LIETUVIŠKAI	Lithuanian	ODONTOLOGINIS ŠVIESA KIETINAMAS RESTAURACINIS KOMPOZITAS

**Kuraray Noritake Dental Inc.**

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

**EMERGO EUROPE**

Westervoortsedijk 60,  
6827 AT Arnhem, The Netherlands

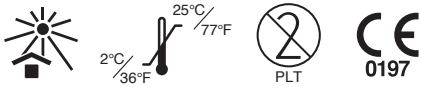
**Kuraray Europe GmbH (EU Importer)**

Philipp-Reis-Str. 4,  
65795 Hattersheim am Main, Germany  
Phone:+49 (0)69 305 35835 Fax:+49 (0)69 305 98 35835  
URL:<https://www.kuraraynoritake.eu>

1563R238R

# CLEARFIL MAJESTY™ ES-2

## I. INTRODUCTION

CLEARFIL MAJESTY ES-2 is a light-cure, radiopaque restorative material which provides accurate color matching, high polishability and excellent physical properties, making it ideal for both anterior and posterior restorations. It is formulated with optimal viscosity, assuring easy handling and placement. It is packaged in traditional syringes or Pre-loaded tips (PLTs).

The "lightness/ value"- based unique shade taking system of the CLEARFIL MAJESTY ES-2 Shade guide provides shade taking options for single (Classic) and dual-(Premium) shade layering for highly esthetic restorations. Universal shades cover all Classic and Premium shades with a limited number of syringes/PLTs depending on cavity class. It has radiopacity equal to or greater than 1mm aluminum, and is classified as a Type 1 and Class 2 (Group 1) material by ISO 4049.

The general clinical benefit of this product is to restore tooth function for the following INDICATIONS FOR USE.

## II. INDICATIONS FOR USE

CLEARFIL MAJESTY ES-2 is indicated for the following restorative applications:

- [1] Direct restorations for all cavity classes in anterior and posterior teeth
- [2] Direct veneers
- [3] Correction of tooth position and tooth shape (e.g. diastema closure, dwarfed tooth, etc.)
- [4] Intraoral repairs of fractured restorations

## III. CONTRAINDICATIONS

Patients with a history of hypersensitivity to methacrylate monomers.  
Patients known to be allergic to any of the ingredients contained in this product.

## IV. INCOMPATIBILITIES

Do not use eugenol containing materials for pulp protection or temporary sealing since the eugenol can retard the curing process.

## V. PRECAUTIONS

### 1. Safety precautions

1. This product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylate monomers or any other components.
2. If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching or numbness, discontinue use of the product and seek medical attention.
3. Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomers or any other components.
4. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect them in the event of splashing material.
5. If the product comes in contact with human body tissues, take the following actions:
  - <If the product gets in the eye>  
Immediately wash the eye with copious amounts of water and consult a physician.
  - <If the product comes in contact with the skin or the oral mucosa>  
Immediately wipe the area with a cotton pellet or a gauze moistened with alcohol, and rinse with copious amounts of water.
6. Use caution to prevent the patient from accidentally swallowing the product.
7. Syringe : Avoid sharing the same paste among different patients to prevent cross-contamination. Dispense the paste onto a mixing pad before use. Do not try to apply it directly from the syringe onto the patient's tooth.
8. PLT : Single- use only. Do not reuse a tip to prevent cross- contamination. Discard it after use and disinfect the dispenser according to its instructions for use.
9. PLT : Before dispensing the paste in the oral cavity, it is recommended to dispense a small amount of paste onto a mixing pad with a composite dispenser. During the preliminary dispensing, handle the dispenser quickly to ensure that the PLT does not fall out. A worn or malfunctioning composite dispenser may cause the PLT to dislodge and fall into the oral cavity, causing a possibility of injury or accidental ingestion. The dispenser should be checked regularly for excessive wear.
10. Dispose of this product as medical waste to prevent infection.

### 2. Handling and manipulation precautions

[CLEARFIL MAJESTY ES-2]

1. The product must not be used for any purposes other than specified in [I.INDICATIONS FOR USE].
2. The use of this product is restricted to dental professionals.
3. Use a pulp capping agent in a cavity close to the pulp or in the event of accidental pulp exposure.
4. Use a rubber dam to prevent contamination and to control moisture.
5. Do not use the product in conjunction with other composite materials. The mixed materials may cause a change in physical properties, including a possible decrease, from the expected results.
6. The paste should be used immediately after being dispensed from the syringe or the tip. If the paste is to be left for a time before use, it should be covered with a light-blocking plate.
7. When using a PLT, use the dispenser with slow and steady pressure. Excessive force is not necessary.
8. When using the shade guide, take the following precautions to prevent cross-contamination.
  - Cover the entire shade guide with a disposable barrier (e.g. a poly bag) to prevent saliva and blood contamination.

- Disinfect before and after use by wiping it thoroughly with an absorbent cotton moistened with alcohol.

[Dental light-curing unit]

1. Do not look directly at the light source. Protective glasses are recommended.
2. Low light intensity causes poor adhesion. Check the lamp for service life and the dental curing light guide tip for contamination. It is advisable to check the dental curing light intensity using an appropriate light evaluating device at appropriate intervals.
3. The emitting tip of the dental curing unit should be held as near and vertical to the resin surface as possible. If a large resin surface is to be light-cured, it is advisable to divide the area into several sections and light-cure each section separately.
4. Check the conditions required to cure the paste by referring to the light-curing times listed in these instructions for use before using the product.

### 3. Storage precautions

1. The product must be used by the expiration date indicated on the package.
2. The product and shade guide must be stored at 2-25°C/ 36-77°F when not in use.
3. When the product is stored in the refrigerator, it should be left at room temperature for more than 15 minutes before each use. Failure to do this might cause breakage of the syringe or the tip.
4. The product and shade guide must be kept away from extreme heat or direct sunlight.
5. After the paste has been dispensed, the syringe should be capped securely as soon as possible to prevent hardening by ambient light of the material, and to prevent foreign matter from entering the syringe.
6. The product must be stored in a proper place where only dental practitioners can access.

## VI. SHADES SYSTEM AND COMPONENTS

### 1. Shades

CLEARFIL MAJESTY ES-2 is available in the following 18 shades of Classic, 14 shades of Premium, 4 shades of Translucent and 4 shades of Universal:

- 1) Classic : XW, W, A1, A2, A3, A3.5, A4, KA6, B1, B2, B3, B4, C1, C2, C3, C4, D2, D3
- 2) Premium
  - Enamel : XWE, WE, A1E, A2E, A3E, A3.5E, A4E
  - Dentin : XWD, WD, A1D, A2D, A3D, A3.5D, A4D
- 3) Translucent : Clear, Amber, Blue, Gray
- 4) Universal : U, UW, UL, UD
  - \*KA6, A3D, A3.5D and A4D are opaque shades.

### Shade Description

Shades are arranged according to the "lightness/ value" and recommended layering method.

Classic shades are designed for single-shade layering and Premium shades are designed for a dual-shade layering method. Combination of Premium A shades covers 3 Classic (VITA) shades.

Table 1 High ← Lightness/ Value → Low

Layering method	Shade category	Shade															
		XW	W	B1	A1	B2	D2	A2	C1	C2	A3	D3	B3	A3.5	B4	C3	A4
Single-shade	Classic																
		Enamel		A1E		A2E		A3E		A3.5E		A4E					
Dual-shade	Premium	Dentin		A1D		A2D		A3D		A3.5D		A4D					

[NOTE]

Translucent shades can be used for fine characterization of the restoration. KA6 has the lowest lightness in Classic shades.

Universal shades are designed to cover all cavity classes with shades as shown in Table 2.

Table 2

Shade category	Cavity Class	Shade							
		W (XW)	A1 (B1, B2)	A2 (D2, C1)	A3 (C2, D3)	A3.5 (B3, B4)	A4 (C3, C4)		
Universal	I	U* (Universal)							
	II	U* (Universal)							
	III	UW (Universal White)	UL (Universal Light)			UD (Universal Dark)			
	IV								
	V								

\*Use UW paste for the tooth determined as W/XW shade, if the restoration requires a high lightness/value.

### 2. Components

#### Main contents

- CLEARFIL MAJESTY ES-2
- Accessories
  - Shade guide (All shades except Universal shades)
  - Shade guide compact (Classic shades except KA6)

Please see the outside of the package for detailed contents and quantity of each component.

### 3. Ingredients

Silanated barium glass filler (30-80%)/Pre-polymerized organic filler (< 40%)/ Bisphenol A diglycidylmethacrylate (Bis-GMA) (5-15%)/ Hydrophobic aromatic dimethacrylate (5-15%)/ Silanated colloidal silica (1-5%)/Hydrophobic aliphatic dimethacrylate (1-5%)/ dl-Camphorquinone (< 0.1%)

Units in parentheses are mass %.

[NOTE]

The total amount of filler is approx. 78 wt%, including inorganic filler of 40 vol%.  
The particle size of inorganic fillers ranges from 0.37µm to 1.5µm.

**VII. CLINICAL PROCEDURES**

**Shade taking and restorative technique**

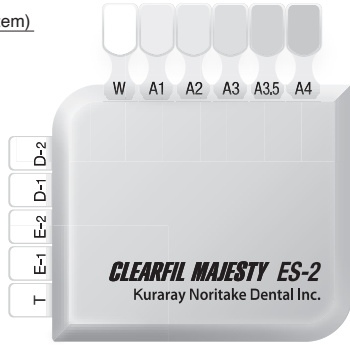
Shade presentation in the shade guide: Classic shades except KA6 are arranged in the shade taking tabs as described below in the "Single-shade Technique" and "Dual-shade Technique".

Translucent, Premium Enamel, Premium Dentin and KA6 shades are arranged in the tabs marked T, E-1, E-2, D-1 and D-2.

**1. Single-shade Technique (Classic system)**


Step 1:

Select the closest "lightness/value" tab from 6 exposed tab options of the shade guide.


Step 2:

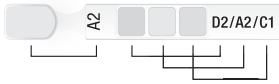Pull out the chosen tab from the holder and select the corresponding shade. For example, if A2 tab is chosen, the shade D2, A2 or C1 can be selected as shown below.

**2. Dual-shade Technique (Premium system)**

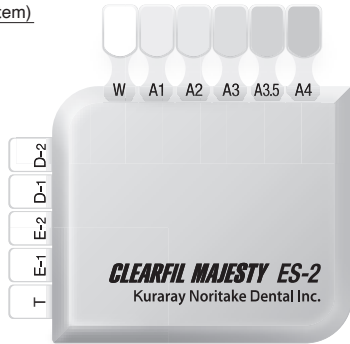Select the closest "lightness/value" tab from 6 exposed tab options of the shade guide. For the teeth lighter than "W" shade, pull-out the tab for "XW" shade taking.

- Select the corresponding pastes of Premium shades (Enamel/Dentin) as shown in Table 1. For example, if A1 tab is chosen, then select A1E and A1D shades.

- Replace dentin space with a respective Dentin shade, and enamel space with a respective Enamel shade.

**3. Universal shade Technique (Universal system)**

Select the closest "lightness/value" tab from 6 exposed tab options of the shade guide.

- Select the corresponding paste of Universal shades depending on cavity class as shown in Table 2.

[NOTE]

When the W/XW tab is selected from the procedure above in class I or II cavity cases, select UW paste (the highest value in this system) if the restoration requires a high lightness/value.

**A. Standard procedure I (INDICATIONS FOR USE [1] to [3])**

- [1] Direct restorations for all cavity classes in anterior and posterior teeth
- [2] Direct veneers
- [3] Correction of tooth position and tooth shape (e.g. diastema closure, dwarfed tooth, etc.)

**A-1. Shade taking**

Clean the tooth with pumice and water to remove surface stains, then select the appropriate shade according to the above "Shade taking and restorative technique".

**A-2. Isolation and Moisture control**

Avoid contamination of the treatment area from saliva or blood to produce optimal results. A rubber dam is recommended to keep the tooth clean and dry.

**A-3. Cavity preparations**

Remove any infected dentin and prepare the cavity in the usual manner.

**A-4. Pulp protection**

Any actual or near pulp exposure could be covered with a hard setting calcium hydroxide material or other indicated materials. Do not use eugenol materials for pulp protection.

**A-5. Tooth surface treatment and bonding**

Tooth surface treatment and bonding should be performed according to the instructions for use of the bonding system used.

**A-6. Dispensing of the paste**

- 6-1. Syringe : Dispense the necessary amount of paste from the syringe onto the mixing pad by turning the handle slowly in a clockwise manner.
- 6-2. PLT : Insert a PLT into the dispenser barrel and extrude the paste into the cavity according to the instructions for use of the dispenser.

[CAUTION]

Use the dispenser with a slow and steady pressure. Excessive force is not necessary. Discard the tip after use.

**A-7. Placement and light-curing of CLEARFIL MAJESTY ES-2**

- 7-1. The application of a flowable composite resin or an opaque resin (e.g. CLEARFIL ST OPAQUER) to the cavity as an initial layer is optional. When adapting it, refer to its instructions for use.
- 7-2. Place the chosen shade of the paste into the cavity and light-cure with a dental curing unit. Considering the depth of cure, incremental curing may be required. Refer to the following table for the relationship between curing time and depth of cure with a visible light source.

Table 3: Relationship between curing time and depth of cure for type of light source.

Type of light source (Light intensity)	Curing time	Shade and depth of cure				
		Clear Amber Gray Blue	XW,W,A1,A2,A3,A3.5 B1,B2,B3,B4 C1,C2,C3,D2,D3 XWE,WE,A1E,A2E A3E,A3.5E,A4E U,UL,UD	A4,C4 WD,XWD A1D,A2D UW	KA6 A3D A3.5D A4D	
High-intensity BLUE LED* (More than 1500 mW/cm <sup>2</sup> )	Twice for 3 or 5 sec.	3.0 mm	2.0 mm	1.5 mm	1.0 mm	
Middle-intensity BLUE LED* (1100-1400 mW/cm <sup>2</sup> )	10 sec.					
Low-intensity BLUE LED* (800-1000 mW/cm <sup>2</sup> )	20 sec.					
High-intensity Halogen lamp (More than 800 mW/cm <sup>2</sup> )	10 sec.					
Low-intensity Halogen lamp (400-700 mW/cm <sup>2</sup> )	20 sec.					

The effective wavelength range of each dental curing unit must be 400 - 515 nm.

\*Peak of emission spectrum: 450 - 480 nm

**A-8. Finishing**

Contour the restoration and adjust the occlusion using a fine diamond point. Polish with silicon rubber points or polishing discs in the usual manner.

**B. Standard procedure II (INDICATIONS FOR USE [4])**

[4] Intraoral repairs of fractured restorations

**B-1. Shade taking**

Follow the same procedure described in "A-1".

**B-2. Preparation of fractured surfaces**

If necessary, bevel the margin with a diamond polishing point. Then follow the surface treatment procedures of the fractured area based on the surface material (e.g. metal-oxide ceramic, silica-based ceramic, composite, metal or tooth) according to the instructions for use for bonding, silane coupling agents or a metal adhesive primer.

**B-3. Tooth surface treatment and bonding**

Follow the same procedure described in "A-5".

**B-4. Dispensing of the paste**

Follow the same procedure described in "A-6".

**B-5. Placement and light-curing of CLEARFIL MAJESTY ES-2**

- 5-1. The application of an opaque resin (e.g. CLEARFIL ST OPAQUER) to the fractured surfaces as an initial layer is optional. When adapting it, refer to its instructions for use.
- 5-2. Place the chosen shade of the paste into the fractured surfaces and light-cure with a dental curing unit. Considering the depth of cure, incremental curing may be required. Refer to the table in section "A-7" for the relationship between curing time and depth of cure with a visible light source.

**B-6. Finishing**

Follow the same procedure described in "A-8".

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

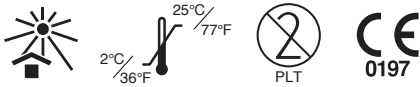
[NOTE]

If a serious incident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user/patient resides.

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"CLEARFIL", "CLEARFIL MAJESTY" and "CLEARFIL ST" are registered trademarks or trademarks of KURARAY CO., LTD.
"VITA" is a trademark of VITA Zahnfabrik H. Rauter GmbH & Co. KG.