# CLEARFIL MAJESTY™ Flow

EN	ENGLISH	English	DENTAL LIGHT-CURED RESTORATIVE COMPOSITE
FR	FRANÇAIS	French	COMPOSITE DE RESTAURATION DENTAIRE PHOTOPOLYMERISABLE
ES	ESPAÑOL	Spanish	COMPOSITE DE RESTAURACIÓN FOTOPOLIMERIZABLE DE USO DENTAL
IT	ITALIANO	Italian	COMPOSITO FOTOPOLIMERIZZABILE PER RESTAURI DENTALI
NL	NEDERLANDS	Dutch	DENTAAL LICHTUITHARDEND RESTAURATIEF COMPOSIT
DE	DEUTSCH	German	DENTALES lichthärtendes Komposit-Restaurationsmaterial
SV	SVENSKA	Swedish	DENTAL LJUSHÄRDANDE RESTAURERINGSKOMPOSIT
NO	NORSK	Norwegian	DENTALT LYSHERDET RESTAURERINGSKOMPOSITT
FI	SUOMI	Finnish	HAMPAAN VALOKOVETTEINEN YHDISTELMÄMUOVI
DA	DANSK	Danish	DENTALT LYSHÆRDENDE RESTAURERINGSKOMPOSIT
PT	PORTUGUÊS	Portuguese	COMPÓSITO FOTOPOLIMERIZAVEL PARA USO DENTÁRIO
EL	ΕΛΛΗΝΙΚΑ	Greek	ΟΔΟΝΤΙΚΗ ΦΩΤΟΠΟΛΥΜΕΡΙΖΟΜΕΝΗ ΣΥΝΘΕΤΗ ΡΗΤΙΝΗ ΑΠΟΚΑΤΑΣΤΑΣΗΣ
TR	TÜRKÇE	Turkish	IŞIKLA SERTLEŞEN DENTAL RESTORATİF KOMPOZİT
PL	POLSKI	Polish	ŚWIATŁOUTWARDZALNY DENTYSTYCZNY KOMPOZYT DO UZUPEŁNIEŃ
RO	ROMÂNĂ	Romanian	COMPOZIT DE RESTAURARE DENTARĂ FOTOPOLIMERIZABIL
HR	HRVATSKI	Croatian	DENTALNI SVJETLOSNO POLIMERIZIRANI RESTAURACIJSKI KOMPOZIT
HU	MAGYAR	Hungarian	FOGÁSZATI, FÉNYRE KEMÉNYEDŐ RESTAURÁCIÓS KOMPOZIT
SL	SLOVENŠČINA	Slovenian	DENTALNI SVETLOBNO STRJUJOČI KOMPOZITNI MATERIAL ZA RESTAVRACIJE
CS	ČESKY	Czech	DENTÁLNÍ SVĚTLEM TUHNOUCÍ KOMPOZITNÍ MATERIÁL PRO NÁHRADY
SK	SLOVENSKY	Slovak	DENTÁLNY SVETLOM VYTVRDZOVANÝ VÝPLŇOVÝ KOMPOZIT
BG	БЪЛГАРСКИ	Bulgarian	СТОМАТОЛОГОЧЕН ФОТОПОЛИМЕРИЗИРАЩ ВЪЗСТАНОВИТЕЛЕН КОМПОЗИТ
UK	УКРАЇНСЬКА	Ukrainian	РЕСТАВРАЦІЙНИЙ КОМПОЗИТНИЙ МАТЕРІАЛ
ET	EESTI	Estonian	DENTAALNE VALGUSKÕVASTUV KOMPOSIIT-RESTAURATSIOONIMATERJAL
LV	LATVIEŠU	Latvian	ZOBĀRSTNIECĪBAS GAISMĀ CIETĒJOŠS RESTAURĀCIJAS KOMPOZĪTMATERIĀLS
LT	LIETUVIŠKAI	Lithuanian	ODONTOLOGINIS ŠVIESA KIETINAMAS RESTAURACINIS KOMPOZITAS

**Kuraray Noritake Dental Inc.**

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

**EMERGO EUROPE**

Westervoortsedijk 60,  
6827 AT Arnhem, The Netherlands

**Kuraray Europe GmbH (EU Importer)**

Philipp-Reis-Str. 4,  
65795 Hattersheim am Main, Germany  
Phone:+49 (0)69 305 35835 Fax:+49 (0)69 305 98 35835  
URL:<https://www.kuraraynoritake.eu>

1561R400R

# CLEARFIL MAJESTY™ Flow

## I. INTRODUCTION

CLEARFIL MAJESTY Flow is a flowable, radiopaque restorative composite resin which provides accurate color matching, high polish ability and excellent physical properties, making it ideal for both anterior and posterior restorations. CLEARFIL MAJESTY Flow can be used alone or together with CLEARFIL AP-X, CLEARFIL MAJESTY Esthetic and CLEARFIL MAJESTY Posterior. It has radiopacity equal to or greater than 1mm aluminum, and is classified as a Type 1 and Class 2 (Group 1) material by ISO 4049. The general clinical benefit of this product is to restore tooth function in the following INDICATIONS FOR USE.

## II. INDICATIONS FOR USE

CLEARFIL MAJESTY Flow is indicated for the following restorative applications:

- Direct restorations for anterior and posterior teeth (Class I - III, V cavities, cervical caries, root erosion)
- Cavity base / liner
- Intraoral repairs of fractured crowns / bridges / composite resin

## III. CONTRAINDICATION

- Patients with a history of hypersensitivity to methacrylate monomers
- The needle tips must not be reused.

## IV. INCOMPATIBILITIES

Do not use eugenol containing materials for pulp protection or temporary sealing, since the eugenol could retard the bonding system curing process.

## V. PRECAUTIONS

### 1. Safety precautions

1. Avoid use of the product for patients with a history of hypersensitivity to methacrylate monomers.
2. If any hypersensitivity occurs, such as dermatitis, discontinue use of the product and consult a physician.
3. Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomer.
4. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect the patient's eyes from splashing material.
5. If the product comes in contact with human body tissues, take the following actions:
  - < If the product gets in the eye>  
Immediately rinse the eye with copious amounts of water and consult a physician.
  - < If the product comes in contact with the skin or the oral mucosa>  
Immediately wipe it off with a cotton pledget moistened with alcohol or gauze and rinse with copious amounts of water.
6. Use caution to prevent the patient from accidentally swallowing the product.
7. Avoid looking directly at the dental curing unit when curing the product.
8. To prevent cross infection, avoid sharing the same paste among different patients.
9. Do not use the same needle tip for different patients to prevent cross infection.
10. Dispose of this product as a medical waste to prevent infection. The needle tip must be disposed of after covering the tip of the needle to prevent injury.

### 2. Handling and manipulation precautions

1. Do not use the product in conjunction with the other composite resin. Mixing materials may result in a physical property change, possibly a decrease, from the properties expected.
2. Amalgam or other lining materials remaining in the cavity will prevent the passage of light and the polymerization of the product. Completely remove any lining material when preparing the cavity.
3. When light curing the product, note the light curing depth in this Instructions for Use.
4. The emitting tip of the dental curing unit should be held as near and vertical to the resin surface as possible. If a large resin surface is to be light cured, it is advisable to divide the area into several sections and light-cure each section separately.
5. Low light intensity causes poor adhesion. Check the lamp for service life and the dental curing unit guide tip for contamination. It is advisable to check the dental curing unit intensity using an appropriate lightevaluating device at appropriate intervals.
6. The product should be returned to room temperature before dispensing when it has been taken out of a refrigerator. Failure to do this could cause difficulty using the plunger.
7. Use caution not to get your fingers injured by any sharp edge that might occur due to damage.
8. The use of this product is restricted to dental professionals.

### 3. Storage precautions

1. Do not use the product after the expiration date. Note the expiration date on the outside of the package.
2. The product must be stored at 2 - 25°C / 36 - 77°F when not in use.
3. Keep away from extreme heat, direct sunlight.
4. After the paste has been dispensed, the syringe should be capped securely as soon as possible to prevent the paste inside the syringe from being hardened by ambient light and to keep foreign matter from entering the syringe.
5. The product must be stored in proper places where only dental practitioners can access it.

## VI. SHADES SYSTEM AND COMPONENTS

### 1. Shades

CLEARFIL MAJESTY Flow is available in 9 shades; A1, A2, A3, A3.5, A4, B2, C3, Cv (Cervical)\*, OA3\*

\*Cv and OA3 are opaque shades.

## 2. Components

See outer package for the items included in the package.

- CLEARFIL MAJESTY Flow
- Needle tips
- Shade guide

## 3. Ingredients

Silanated barium glass filler (55-90%)/  
Hydrophobic aromatic dimethacrylate (5-25%)/  
Silanated colloidal silica (1-10%)/  
Triethyleneglycol dimethacrylate (TEGDMA) (< 10%)/  
dl-Camphorquinone (< 1%)

Units in parentheses are mass %.

## [NOTE]

The total amount of inorganic filler is approx. 62vol%. The particle size of inorganic fillers ranges from 0.02 µm to 19 µm.

## VII. CLINICAL PROCEDURES

### Method. A

- Direct restorations for anterior and posterior teeth (Class I - III, V cavities, cervical caries, root erosion)
- Cavity base / liner

### 1. Shade selection

Select an appropriate product shade corresponding with the shade of the tooth to be treated using the product shade guide. Two or more shades may be combined to achieve a better overall match.

### 2. Syringe preparation

Remove the cap from the selected syringe and attach a needle tip securely.

### 3. Cavity preparation

Be sure the cavity is adequately cleaned. An adequately cleaned cavity assures maximum adhesive performance.

### 4. Moisture control and pulp protection

Avoid contamination of the treatment area from saliva or blood to assure optimal results. A rubber dam is recommended to keep the tooth clean and dry. Any actual or near pulp exposure could be covered with a hard setting calcium hydroxide material. There is no need for cement lining or basing. Do not use eugenol materials for pulp protection.

### 5. Tooth surface treatment and bonding

Tooth surface treatment and bonding should be performed according to Instructions for Use of the bonding system used (e.g. CLEARFIL TRI-S BOND).

### 6. Placement and light curing of CLEARFIL MAJESTY Flow

Place the chosen shade of the product into the cavity, and light-cure the paste with a dental curing unit\* following the times in the table below. When using the product for a deep cavity, load and light-cure the paste in a series of increments, taking into consideration the relationship between the curing time and depth of cure. When using the product for a base and liner, the paste can be placed and light-cured and then followed by the placing of a light-cure composite resin, such as CLEARFIL AP-X, CLEARFIL MAJESTY Esthetic or CLEARFIL MAJESTY Posterior.

Table: Relation between curing time and depth of cure for each dental curing unit.

Type	Curing time (Sec)	Depth of cure (mm)	
		A1, A2, A3, A3.5, A4, B2, C3	Cv, OA3
Conventional halogen*	20	1,5	1,0
Fast halogen*	10		
Plasma arc*	5		
LED*	20		

### \*Dental curing unit

Type	Light source	Wavelength range and light intensity
Conventional halogen	Halogen lamp	Light intensity <sup>2)</sup> of 300 - 550 mW/cm <sup>2</sup> in wavelength range from 400 - 515 nm
Fast halogen	Halogen lamp	Light intensity <sup>2)</sup> of more than 550 mW/cm <sup>2</sup> in wavelength range from 400 - 515 nm
Plasma arc	Xenon lamp	Light intensity <sup>3)</sup> of more than 2000 mW/cm <sup>2</sup> in wavelength range from 400 - 515 nm, and light intensity of more than 450 mW/cm <sup>2</sup> in wavelength range from 400 - 430 nm.
LED	Blue LED <sup>1)</sup>	Light intensity <sup>2)</sup> of more than 300 mW/cm <sup>2</sup> in wavelength range from 400 - 515 nm

1) Peak of emission spectrum: 450 - 480 nm

2) Evaluated according to ISO 10650-1.

3) Wavelength distribution and light intensity values measured with a spectro-radiometer calibrated using an IEC or the NIST (National Institute of Standards and Technology) standard lamp

## [NOTE]

If two or more shades are selected, the paste of one shade should be applied and then light cured before the paste of another shade is applied and light cured. After all the pastes have been layered, the entire bulk of the product should be completely light cured again.

### 7. Finishing

Contour the restoration and adjust the occlusion using a fine diamond point. Polish with silicon rubber points or polishing discs in the usual manner.

## **Method. B**

- Intraoral repairs of fractured crowns / bridges / composite resin

### 1. Preparation of fractured surfaces

#### 1-a. Facing material surface

If necessary, remove a layer of the fractured surfaces and place a bevel at the marginal area using suitable instruments (e.g. diamond point).

#### 1-b. Metal surface

If the metal surface is not surface treated, roughen it by using sandblasting with alumina powder or using a diamond point. Rinse the surface with water and then dry.

### 2. Acid etching of facing material

Apply a phosphoric acid etching agent (e.g. K-ETCHANT GEL) to the facing material and leave it in place for 5 seconds before washing and drying.

### 3. Precious metal surface

If precious metal is used, apply a metal-adhesive primer (e.g. ALLOY PRIMER) according to the Instructions for Use of the product used.

### 4. Silane treatment

Apply a silane-coupling agent (e.g. CLEARFIL CERAMIC PRIMER) to the facing material according to the Instructions for Use of the product used.

### 5. Tooth surface treatment and bonding

Tooth surface treatment and bonding should be performed according to Instructions for Use of the bonding system used (e.g. CLEARFIL TRI-S BOND).

### 6. Syringe preparation

Remove the cap from the selected syringe and attach a needle tip securely.

### 7. Placement and light curing of CLEARFIL MAJESTY Flow

Place the chosen shade of the product into the cavity, and light-cure it with a dental curing unit for curing following the times specified in table in Method. A 6. The paste can be placed and light-cured in a few increments, if necessary, considering the relationship between the curing time and depth of cure.

#### [NOTE]

Use opaque resin (e.g. CLEARFIL ST OPAQUER) for metal surface to prevent metal shine through.

### 8. Finishing

Contour the restoration and adjust the occlusion using a fine diamond point. Polish with silicon rubber points or polishing discs in the usual manner.

#### [WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proved to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

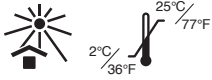
#### [NOTE]

- If a serious incident attributable to this product occurs, report it to the manufacturer through the EU Importer, and to the regulatory authorities of the country in which the user/patient resides.
- The Summary of Safety and Clinical Performance (SSCP) can be retrieved from the European Database on Medical Devices (EUDAMED) at <https://ec.europa.eu/tools/eudamed>.  
Basic UDI-DI: 458211076Composite132Z5

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"CLEARFIL MAJESTY", "CLEARFIL TRI-S BOND", "CLEARFIL AP-X", "CLEARFIL ST" and "CLEARFIL" are registered trademarks or trademarks of KURARAY CO., LTD.
„Kuraray Noritake Dental Inc.“ pakeičia visus produktus su įrodymais trūkumais.