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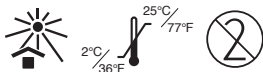
1561R521R

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I. INTRODUCTION

CLEARFIL MAJESTY Posterior PLT (PLT: Pre-loaded tip) is a light-cure, radiopaque restorative composite resin which provides accurate color matching, high polish ability and excellent physical properties, making it ideal for both anterior and posterior restorations. It is formulated with optimal viscosity assuring easy handling and placement. CLEARFIL MAJESTY Posterior PLT, with its' special dispensing system, can be quickly and conveniently placed directly into the cavity. It has radiopacity equal to or greater than 1mm aluminum, and is classified as a Type 1 and Class 2 (Group 1) material by ISO 4049. The general clinical benefit of this product is to restore tooth function in the following INDICATIONS FOR USE.

II. INDICATIONS FOR USE

CLEARFIL MAJESTY Posterior PLT is indicated for the following restorative applications:

- Direct restorations for anterior and posterior teeth (Class I - V cavities)
- Correction of tooth position and tooth shape (e.g. diastema closure, dwarfed tooth, etc.)
- Intraoral repairs of fractured crowns/ bridges

III. CONTRAINDICATIONS

Patients with a history of hypersensitivity to methacrylate monomers

IV. INCOMPATIBILITIES

Do not use eugenol containing materials for pulp protection or temporary sealing, since the eugenol could retard the bonding system curing process.

V. PRECAUTIONS

1. Safety precautions

1. Avoid use of the product for patients with a history of hypersensitivity to methacrylate monomers.
2. If any hypersensitivity occurs, such as dermatitis, discontinue use of the product and consult a physician.
3. Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomer.
4. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect the patient's eyes from splashing material.
5. If the product comes in contact with human body tissues, take the following actions:
 - <If the product gets in the eye>
Immediately rinse the eye with copious amounts of water or consult a physician.
 - <If the product comes in contact with the skin or the oral mucosa>
Immediately wipe it off with a cotton pledget moistened with alcohol or gauze and rinse with copious amounts of water.
6. Use caution to prevent the patient from accidentally swallowing the product.
7. Avoid looking directly at the dental curing light when curing the product.
8. Avoid using the same tip for different patients to prevent cross contamination. Discard the tip after use and sterilize the dispenser according to the dispenser Instruction for Use.
9. For infection control reasons, PLT tips are for single use only.
10. Dispose of this product as a medical waste to prevent infection.

2. Handling and manipulation precautions

1. Do not use the product in conjunction with the other composite resin. The mixture of materials may cause a change in physical properties, possibly a decrease, from the properties expected.
2. If the adherent surface extends to uncut enamel, apply a phosphoric acid etching agent (e.g. K-ETCHANT GEL) and leave it in place for 10 seconds before washing and drying.
3. Amalgam or other lining material remaining in the cavity will prevent the passage of light and the polymerization of the product. Completely remove any lining material when preparing the cavity.
4. When light curing the product, note the light curing depth in this Instructions for Use.
5. The emitting tip of the dental curing light should be held as near and vertical to the resin surface as possible. If a large resin surface is to be light cured, it is advisable to divide the area into several sections and light-cure each section separately.
6. Low light intensity causes poor adhesion. Check the lamp for service life and the dental curing light guide tip for contamination. It is advisable to check the dental curing light intensity using an appropriate lightevaluating device at appropriate intervals.
7. The paste should be used as soon as practicable after being dispensed from the tip. If the paste is to be left for a while before use, it should be covered with a light blocking plate.
8. The product should be returned to room temperature before dispensing, if it has been taken out of a refrigerator. Failure to do this might cause breakage of the tip.
9. Squeeze the dispenser with a slow, steady pressure. Excessive force is not necessary.
10. The use of this product is restricted to dental professionals.

3. Storage precautions

1. Do not use the product after the expiration date. Note the expiration date on the outside of package.
2. The product must be stored at 2 - 25°C/ 36 - 77°F when not in use.
3. Keep away from extreme heat, direct sunlight.
4. The product must be stored in proper places where only dental practitioners can access it.

VI. SHADES SYSTEM AND COMPONENTS

1. Shades

CLEARFIL MAJESTY Posterior PLT is available in 6 shades: A2, A3, A3.5, B2, XL, OA3
*OA3 is opaque shade.

2. Components

- See outer package for the items included in the package.
- CLEARFIL MAJESTY Posterior PLT
 - CLEARFIL MAJESTY Posterior shade guide

3. Ingredients

Silanated glass ceramics (60-80%)/
Surface treated alumina micro filler (10-20%)/
Hydrophobic aromatic dimethacrylate (1-10%)/
Bisphenol A diglycidylmethacrylate (Bis-GMA) (< 3%)/
Triethyleneglycol dimethacrylate (TEGDMA) (< 3%)/
Silanated silica filler (< 3%)/dl-Camphorquinone (< 0.1%)

Units in parentheses are mass %.

[NOTE]

The total amount of inorganic filler is approx. 82vol%. The particle size of inorganic fillers ranges from 0.02 µm to 7.9 µm.

VII. CLINICAL PROCEDURES

Method. A

- Direct restorations for anterior and posterior teeth (Class I - V cavities)
 - Correction of tooth position and tooth shape (e.g. diastema closure, dwarfed tooth, etc.)
1. Cleaning tooth structure
Be sure the cavity is adequately cleaned. An adequately cleaned cavity assures maximum adhesive performance.
 2. Moisture control
Avoid contamination of the treatment area from saliva or blood to produce optimal results. A rubber dam is recommended to keep the tooth clean and dry.
 3. Cavity preparations
Remove any infected dentin and prepare the cavity in the usual manner.
 4. Pulp protection
Any actual or near pulp exposure could be covered with a hard setting calcium hydroxide material. There is no need for cement lining or basing. Do not use eugenol materials for pulp protection.
 5. Tooth surface treatment and bonding
Tooth surface treatment and bonding should be performed according to Instructions for Use of the bonding system used (e.g. CLEARFIL TRI-S BOND).
 6. Placement and light curing of CLEARFIL MAJESTY Posterior PLT
 - 6-1. If necessary, the application of a flowable composite resin (e.g. CLEARFIL MAJESTY Flow) to the cavity as an initial layer is optional (Refer to its' Instructions for Use). This layer should be polymerized prior to placing the first layer of CLEARFIL MAJESTY Posterior PLT.
 - 6-2. Select an appropriate shade corresponding with the shade of the tooth to be treated using CLEARFIL MAJESTY Posterior shade guide. Two or more shades may be combined to achieve a better overall shade match.
 - 6-3. Place a PLT tip of the product into the dispenser barrel according to the Instructions for Use for dispenser. The tip may be rotated to provide the proper angle for delivery into the cavity.
- [CAUTION]
Squeeze the dispenser with a slow, steady pressure. Excessive force is not necessary.
Discard the tip after use and sterilize the dispenser according to its Instructions for Use.
- 6-4. Place the chosen shade product into the cavity, and light-cure the resin with a dental curing unit* following the table.

Table: Relation between curing time and depth of cure for each dental curing unit.

| Type | Curing time (Sec) | Depth of cure (mm) | |
|-----------------------|-------------------|----------------------|-----|
| | | A2, A3, A3.5, B2, XL | OA3 |
| Conventional halogen* | 20 | 1,5 | 1,0 |
| Fast halogen* | 10 | | |
| Plasma arc* | 5 | | |
| LED* | 20 | | |

Dental curing unit

| Type | Light source | Wavelength range and light intensity |
|----------------------|------------------------|--|
| Conventional halogen | Halogen lamp | Light intensity ²⁾ of 300 - 550 mW/cm ² in wavelength range from 400 - 515 nm |
| Fast halogen | Halogen lamp | Light intensity ²⁾ of more than 550 mW/cm ² in wavelength range from 400 - 515 nm |
| Plasma arc | Xenon lamp | Light intensity ³⁾ of more than 2000 mW/cm ² in wavelength range from 400 - 515 nm, and light intensity of more than 450 mW/cm ² in wavelength range from 400 - 430 nm. |
| LED | Blue LED ¹⁾ | Light intensity ²⁾ of more than 300 mW/cm ² in wavelength range from 400 - 515 nm |

1) Peak of emission spectrum: 450 - 480 nm

2) Evaluated according to ISO 10650-1.

3) Wavelength distribution and light intensity values measured with a pectro-radiometer calibrated using an IEC or the NIST (National Institute of Standards and Technology) standard lamp

[NOTE]

- If two or more shades are selected, the paste of one shade should be applied and then light cured before the paste of another shade is applied and light cured. After all the pastes have been layered, the entire bulk of the product should be completely light cured again.
- If necessary, a thin layer of opaque resin (e.g. CLEARFIL ST OPAQUER) can be applied and light-cured according to its Instructions for Use.

7. Finishing

Contour the restoration and adjust the occlusion using a fine diamond point. Polish with silicon rubber points or polishing discs in the usual manner.

Method. B

Intraoral repairs of fractured crowns/ bridges

1. Preparation of fractured surfaces

1-a. Facing material surface

If necessary, remove a layer of the fractured surfaces and place a bevel at the marginal area using suitable instruments (e.g. diamond point).

1-b. Metal surface

If the metal surface is not surface treated, roughen it by using sandblasting with alumina powder or using a diamond point. Rinse the surface with water and then dry.

2. Acid etching of facing material

Apply a phosphoric acid etching agent (e.g. K-ETCHANT GEL) to the facing material and leave it in place for 5 seconds before washing and drying.

3. Precious metal surface

If precious metal is used, apply a metal-adhesive primer (e.g. ALLOY PRIMER) according to the Instructions for Use of the product used.

4. Silane treatment

Apply a silane-coupling agent (e.g. CLEARFIL CERAMIC PRIMER) to the facing material according to the Instructions for Use of the product used.

5. Tooth surface treatment and bonding

Tooth surface treatment and bonding should be performed according to Instructions for Use of the bonding system used (e.g. CLEARFIL TRI-S BOND).

6. Placement and light curing of CLEARFIL MAJESTY Posterior PLT

6-1. If necessary, the application of a flowable composite resin (e.g. CLEARFIL MAJESTY Flow) to the cavity as an initial layer is optional (Refer to its' Instructions for Use). This layer should be polymerized prior to placing the first layer of CLEARFIL MAJESTY Posterior PLT.

6-2. When using CLEARFIL MAJESTY Posterior shade guide, select an appropriate shade of the product corresponding to the fractured crown/ bridge to be repaired. Two or more shades can be selected and combined in order to give a better match to the shade of the fractured crown/ bridge.

6-3. Place a PLT tip of the product into the dispenser barrel according to the Instructions for Use for dispenser. The tip may be rotated to gain the proper angle for entrance into the cavity.

[CAUTION]

Squeeze the dispenser with a slow, steady pressure. Excessive force is not necessary. Discard the tip after use and sterilize the dispenser according to its Instructions for Use.

6-4. Load the product of the selected shade to the cavity and light-cure it with a dental curing unit for curing time specified on the table in **Method. A 6-4**. Considering the depth of cure, apply the resin in layers and polymerize each layer, if necessary. If two or more shades are to be applied, apply them in layers respectively, either polymerizing all the layers collectively, or each layer individually, depending on the total depth of cure.

[NOTE]

Use opaque resin (e.g. CLEARFIL ST OPAQUER) for metal surface to prevent metal shine through.

7. Finishing

Contour the restoration and adjust the occlusion using a fine diamond point. Polish with silicon rubber points or polishing discs in the usual manner.

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proved to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

[NOTE]

- If a serious incident attributable to this product occurs, report it to the manufacturer through the EU Importer, and to the regulatory authorities of the country in which the user/patient resides.
- The Summary of Safety and Clinical Performance (SSCP) can be retrieved from the European Database on Medical Devices (EUDAMED) at <https://ec.europa.eu/tools/eudamed>.
Basic UDI-DI: 458211076Composite145ZE

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"CLEARFIL MAJESTY", "CLEARFIL TRI-S BOND", "CLEARFIL ST" and "CLEARFIL" are registered trademarks or trademarks of KURARAY CO., LTD.
Kuraray Noritake Dental Inc.“ pakeičia visus produktus su įrodomais trūkumais.