# CLEARFIL™ CERAMIC PRIMER PLUS

<b>EN</b>	ENGLISH	English	DENTAL UNIVERSAL PROSTHETIC PRIMER
<b>FR</b>	FRANÇAIS	French	APPRÊT PROTHÉTIQUE UNIVERSEL DENTAIRE
<b>ES</b>	ESPAÑOL	Spanish	IMPRIMADOR PROTÉSICO DENTAL UNIVERSAL
<b>IT</b>	ITALIANO	Italian	PRIMER PROTESICO UNIVERSALE
<b>NL</b>	NEDERLANDS	Dutch	TANDHEELKUNDIGE UNIVERSELE PROTHETISCHE PRIMER
<b>DE</b>	DEUTSCH	German	DENTALER UNIVERSELL einsetzbarer PROTHETISCHER PRIMER
<b>SV</b>	SVENSKA	Swedish	DENTAL UNIVERSELL PROTESPRIMER
<b>NO</b>	NORSK	Norwegian	DENTAL UNIVERSELL PROTETISK PRIMER
<b>FI</b>	SUOMI	Finnish	HAMMASLÄKETEILISEN KÄYTTÖN TARKOITETTU RESTAURATODENESKÄSITELYNE
<b>DA</b>	DANSK	Danish	DENTAL UNIVERSAL PROTETISK PRIMER
<b>PT</b>	PORTUGUÊS	Portuguese	PRIMER UNIVERSAL PARA PRÓTESES DENTÁRIAS
<b>EL</b>	ΕΛΛΗΝΙΚΑ	Greek	ΟΔΟΝΤΙΑΤΡΙΑΚΟ ΠΡΟΣΘΕΤΙΚΟ PRIMER ΓΕΝΙΚΗΣ ΧΡΗΣΕΩΣ
<b>TR</b>	TÜRKÇE	Turkish	ÜNİVERSAL DENTAL PROTETİK PRİMER
<b>PL</b>	POLSKI	Polish	DENTYSTYCZNY UNIWERSALNY PRIMER PROTETYCZNY
<b>RO</b>	ROMÂNĂ	Romanian	PRIMER PROTETIC DENTAR UNIVERSAL
<b>HR</b>	HRVATSKI	Croatian	DENTALNI UNIVERZALNI PROTETIČKI PREMAZ
<b>HU</b>	MAGYAR	Hungarian	FOGÁSZATI UNIVERZÁLIS PROTETIKAI PRIMER
<b>SL</b>	SLOVENŠČINA	Slovenian	DENTALNI PROTETIČNI PRIMER ZA UNIVERZALNO UPORABO
<b>CS</b>	ČESKY	Czech	DENTÁLNÍ UNIVERZÁLNĚ POUŽITELNÝ PROTETICKÝ PRIMER
<b>SK</b>	SLOVENSKY	Slovak	DENTÁLNY UNIVERZÁLNY PROTETICKÝ PRIMER
<b>BG</b>	БЪЛГАРСКИ	Bulgarian	СТОМАТОЛОГИЧЕН УНИВЕРСАЛЕН ПРАЙМЕР ЗА ПРОТЕЗИ
<b>UK</b>	УКРАЇНСЬКА	Ukrainian	УНІВЕРСАЛЬНИЙ ЗУБОПРОТЕЗНИЙ ПРАЙМЕР
<b>ET</b>	EESTI	Estonian	DENTAALNE UNIVERSAALSELT KASUTATAV PROTEETILINE PRAIMER
<b>LV</b>	LATVIEŠU	Latvian	ZOBĀRSTNIECĪBAS UNIVERSĀLAIS PROTĒZES PRAIMERIS
<b>LT</b>	LIETUVIŠKAI	Lithuanian	ODONTOLOGINIS UNIVERSALUS PROTEZAVIMO GRUNTAS

**Kuraray Noritake Dental Inc.**  
1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

**EMERGO EUROPE**

Westervoortsedijk 60,  
6827 AT Arnhem, The Netherlands


**Kuraray Europe GmbH** (EU Importer)

Philipp-Reis-Str. 4,  
65795 Hattersheim am Main, Germany  
Phone:+49 (0)69 305 35835 Fax:+49 (0)69 305 98 35835  
URL:<https://www.kuraraynoritake.eu>

1562R589R

# CLEARFIL™ CERAMIC PRIMER PLUS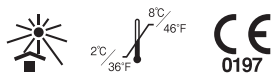

## I. INTRODUCTION

CLEARFIL CERAMIC PRIMER PLUS is a dental universal prosthetic primer that provides an enhanced adhesive surface to ceramic (e.g. conventional porcelain, lithium disilicate, zirconia), hybrid ceramics (e.g. ESTENIA C&B), composite resin and metal. The general clinical benefit of this product is to restore tooth function for the following INDICATIONS FOR USE.

## II. INDICATIONS FOR USE

CLEARFIL CERAMIC PRIMER PLUS is indicated for the following uses:

- [1] Surface treatment of prosthetic restorations made of ceramic, hybrid ceramics, composite resin or metal
- [2] Intraoral repairs of fractured restorations made of ceramics, hybrid ceramics, composite resin or metal

## III. CONTRAINDICATION

Patients with a history of hypersensitivity to methacrylate monomers

## IV. POSSIBLE SIDE EFFECT

The oral mucosal membrane may turn whitish when contacted by the product due to the coagulation of protein. This is usually a temporary phenomenon that will disappear in a few days. Instruct patients to avoid irritating the affected area while brushing.

## V. PRECAUTIONS

### 1. Safety precautions

1. This product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylate monomers or any other components.
2. If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching, or numbness, discontinue use of the product and seek medical attention.
3. Avoid direct contact with the skin and / or soft tissue to prevent hypersensitivity. Wear gloves or take appropriate precautions when using the product.
4. Exercise caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel or safety glasses to protect them in the event of splashing material.
5. If the product comes in contact with human body tissues, take the following actions:
  - <If the product gets in the eye>  
Immediately wash the eye with copious amounts of water and consult a physician.
  - <If the product comes in contact with the skin or the oral mucosa>  
Immediately wipe the area with a cotton pellet or a gauze pad moistened with alcohol, and rinse with copious amounts of water.
6. Exercise caution to prevent the patient from accidentally swallowing the product.
7. Do not reuse the Applicator brush to prevent cross-contamination. Applicator brush is single-use only. Discard it after use.
8. Dispose of this product as a medical waste to prevent infection.

### 2. Handling and manipulation precautions

1. The product must not be used for any purposes other than specified in [II.INDICATIONS FOR USE].
2. The use of this product is restricted to dental professionals.
3. Do not mix the product with bonding agents or use in conjunction with other products; combining materials may cause a change in physical properties, including a possible decrease in bond strength.
4. The product should be used shortly after dispensing. It contains volatile ethanol. As the solvent evaporates, the viscosity increases and may make it difficult to apply.
5. Perform bonding (or cementation) soon after treating the restoration surface with the product.
6. If the treated surface is contaminated with saliva or blood, wash it with water, dry, clean with an etching agent (e.g. K-ETCHANT Syringe), and re-treat.
7. If the product gets deposited on the nozzle, wipe it away using a piece of gauze pad or a cotton pellet moistened with alcohol.
8. If the container of this product is damaged, protect yourself from any danger and immediately discontinue their use.
9. Etching the vital dentin may cause post-operative sensitivity.
10. Apply ALLOY PRIMER when the adherent surface of the precious metal is unable to be roughened intraorally during repairs.

### 3. Storage precautions

1. The product must be used by the expiration date indicated on the package.
2. The product must be stored (2-8°C/ 36-46°F) when not in use, and should be brought to room temperature for 15 minutes before using; otherwise, excess liquid may be dispensed or ooze after use.
3. The product must be kept away from extreme heat, direct sunlight or a flame.
4. The bottle cap should be replaced as soon as the liquid has been dispensed from the bottle. This prevents evaporation of volatile ingredients.
5. The product must be stored in a proper place where only dental practitioners can access.

## VI. COMPONENTS

Please see the outside of the package for contents and quantity.

<Principal ingredients>

- 3-Methacryloxypropyl trimethoxysilane (< 10%)
- 10-Methacryloyloxydecyl dihydrogen phosphate (MDP) (< 2%)
- Ethanol (> 80%)

Units in parentheses are mass %.

## VII. CLINICAL PROCEDURES

### A. Surface treatment of prosthetic restorations made of ceramic, hybrid ceramics, composite resin or metal

#### A-1. Moisture control

If necessary, place a rubber dam to accomplish an appropriate isolation for adhesive procedures.

#### A-2. Surface preparation

*If the adherent surface is metal oxide ceramic (e.g. KATANA Zirconia) or metal:*

- (1) After trial fitting, roughen the adherent surface by blasting with alumina powder (30-50 μm) using air pressure of 0.1-0.4 MPa (1-4 kgf/cm<sup>2</sup>, 15-58 psi). The air pressure and powder size should be properly adjusted to suit the material and/ or shape of the prosthetic restoration, using caution to prevent chipping.
- (2) Clean the prosthetic restoration in an ultrasonic cleaning unit for 2 minutes, followed by drying it with an air stream.

*If the adherent surface is silica-based ceramic (e.g. conventional porcelain, lithium disilicate), hybrid ceramics or composite resin:*

Based on the type of restoration, Acid Treatment or Blasting Treatment may be used:

*Acid Treatment (e.g. conventional porcelain, lithium disilicate):*

- (1) Etch the adherent surface with a hydrofluoric acid solution in accordance with the Instructions for Use of the restoration material, or apply a phosphoric acid agent (e.g. K-ETCHANT Syringe) over the adherent surface and leave it for 5 seconds.
- (2) Clean the adherent surface with water and dry.

*Blasting Treatment (e.g. composite resin):*

- (1) Roughen the adherent surface by blasting with alumina powder (30-50 μm) using air pressure of 0.1-0.2 MPa (1-2 kgf/cm<sup>2</sup>, 15-29 psi). The air pressure and powder size should be properly adjusted to suit the material and/ or shape of the prosthetic restoration, using caution to prevent chipping.
- (2) Clean the adherent surface in an ultrasonic unit for 2 minutes, followed by drying it with an air stream.

### A-3. Application of CLEARFIL CERAMIC PRIMER PLUS

- (1) If the adherent surface is composite resin, apply an etching agent (e.g. K-ETCHANT Syringe) onto the adherent surface and leave it for 5 seconds. Then thoroughly wash and dry.
- (2) Dispense the necessary amount of CLEARFIL CERAMIC PRIMER PLUS into a well of the mixing dish immediately before application.
- (3) Apply CLEARFIL CERAMIC PRIMER PLUS to the adherent surface of the restoration with an applicator brush.
- (4) Dry the entire adherent surface sufficiently using mild, oil-free air flow.

### A-4. Cementation

Cement the prosthetic appliance using resin cement (e.g. PANAVIA V5, PANAVIA F2.0 or PANAVIA 21) according to relevant sections "Cementation" or "Placing" of the Instructions for Use.

## B. Intraoral repairs of fractured crowns/bridges made of ceramic, hybrid ceramics, composite resin or metal

### B-1. Moisture control

If necessary, place a rubber dam to accomplish an appropriate isolation for adhesive procedures.

### B-2. Preparation of adherent surfaces

- (1) Roughen adherent surface using a diamond point or blasting with alumina powder. The air pressure and powder size should be properly adjusted to suit the material and/ or shape of the prosthetic restoration.
- (2) Apply an etching agent (e.g. K-ETCHANT Syringe) to the adherent surfaces of the facing material and the metal frame according to the Instructions for Use of the etching agent and leave it in place for 5 seconds. Then thoroughly wash and dry the surface.

### B-3. Application of CLEARFIL CERAMIC PRIMER PLUS

- (1) Dispense the necessary amount of CLEARFIL CERAMIC PRIMER PLUS into a well of the mixing dish immediately before application.
  - (2) Apply CLEARFIL CERAMIC PRIMER PLUS to the facing material surface with an applicator brush.
- [NOTE]  
If the adherent surface is not roughened, apply ALLOY PRIMER to the precious metal surface for optimum performance; follow the Instructions for Use before applying CLEARFIL CERAMIC PRIMER PLUS.
- (3) Dry the entire adherent surface sufficiently by blowing mild oil-free air.

### B-4. Treatment of tooth surface

When there is tooth surface as part of the adherent surface, apply a bonding agent (e.g. CLEARFIL TRI-S BOND PLUS) to the area according to the Instructions for Use.

### B-5. Light-cure composite resin fillings

Apply a light-cure composite resin (e.g. CLEARFIL MAJESTY ES-2) into the cavity, light-cure, finish and polish the resin according to the Instructions for Use.

## [WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proved to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

## [NOTE]

If a serious incident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user/patient resides.

## [NOTE]

"CLEARFIL", "PANAVIA" and "ESTENIA" are registered trademarks or trademarks of KURARAY CO., LTD.

"KATANA" is a registered trademark or trademark of NORITAKE CO., LIMITED.

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„CLEARFIL“, „PANAVIA“, ir „ESTENIA“ yra „KURARAY CO., LTD“ registruotieji prekės ženklai arba prekės ženklai.