# CLEARFIL™ DC Activator

|           |             |            |                                    |
|-----------|-------------|------------|------------------------------------|
| <b>EN</b> | ENGLISH     | English    | DENTAL BONDING AGENT               |
| <b>FR</b> | FRANÇAIS    | French     | AGENT DE LIAISON DENTAIRE          |
| <b>ES</b> | ESPAÑOL     | Spanish    | AGENTE DE ADHESIÓN DE USO DENTAL   |
| <b>IT</b> | ITALIANO    | Italian    | AGENTE DI ADESIONE DENTALE         |
| <b>NL</b> | NEDERLANDS  | Dutch      | DENTALE BONDING                    |
| <b>DE</b> | DEUTSCH     | German     | Dentaler Haftvermittler            |
| <b>SV</b> | SVENSKA     | Swedish    | DENTALT BONDINGSMATERIAL           |
| <b>NO</b> | NORSK       | Norwegian  | DENTALT BINDINGSMIDDEL             |
| <b>FI</b> | SUOMI       | Finnish    | HAMPAAN SIDOSAINE                  |
| <b>DA</b> | DANSK       | Danish     | DENTALT BONDINGMATERIALE           |
| <b>PT</b> | PORTUGUÊS   | Portuguese | AGENTE DE ADESÃO DENTÁRIO          |
| <b>EL</b> | ΕΛΛΗΝΙΚΑ    | Greek      | ΟΔΟΝΤΙΚΟΣ ΣΥΓΚΟΛΛΗΤΙΚΟΣ ΠΑΡΑΓΟΝΤΑΣ |
| <b>TR</b> | TÜRKÇE      | Turkish    | DENTAL BONDING AJAN                |
| <b>PL</b> | POLSKI      | Polish     | STOMATOLOGICZNY SYSTEM WIĄŻĄCY     |
| <b>RO</b> | ROMÂNĂ      | Romanian   | AGENT DENTAR ADEZIV                |
| <b>HR</b> | HRVATSKI    | Croatian   | DENTALNI ADHEZIV                   |
| <b>HU</b> | MAGYAR      | Hungarian  | FOGÁSZATI KÖTŐANYAG                |
| <b>SL</b> | SLOVENŠČINA | Slovenian  | DENTALNO VEZIVNO SREDSTVO          |
| <b>CS</b> | ČESKY       | Czech      | DENTÁLNÍ POJIVO                    |
| <b>SK</b> | SLOVENSKY   | Slovak     | DENTÁLNE TMELIACE ČINIDLO          |
| <b>BG</b> | БЪЛГАРСКИ   | Bulgarian  | СТОМАТОЛОГИЧЕН БОНДИНГ АГЕНТ       |
| <b>UK</b> | УКРАЇНСЬКА  | Ukrainian  | БОНДИНГОВИЙ ДОДАТОК                |
| <b>ET</b> | EESTI       | Estonian   | DENTAALNE ADHESIIV                 |
| <b>LV</b> | LATVIEŠU    | Latvian    | ZOBĀRSTNIECĪBAS SAISTVIELA         |
| <b>LT</b> | LIETUVIŠKAI | Lithuanian | ODONTOLOGINIS RIŠIKLIS             |

## Kuraray Noritake Dental Inc.

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

## EMERGO EUROPE

Westervoortsedijk 60,  
6827 AT Arnhem, The Netherlands

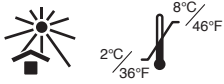
## Kuraray Europe GmbH (EU Importer)

Philipp-Reis-Str. 4,  
65795 Hattersheim am Main, Germany  
Phone:+49 (0)69 305 35835 Fax:+49 (0)69 305 98 35835  
URL:<https://www.kuraraynoritake.eu>

# CLEARFIL™ DC Activator

## I. INTRODUCTION

CLEARFIL DC Activator activates the dual-curing mechanism of the CLEARFIL BONDING SYSTEM (e.g. CLEARFIL SE BOND 2 or CLEARFIL SE Protect). The product is mixed with the BOND and can be used with dual-cure or self-cure composite filling materials, cements, or core build-up materials. The general clinical benefit of this product is to restore tooth function in the following INDICATIONS FOR USE.

## II. INDICATIONS FOR USE

When used with the CLEARFIL BONDING SYSTEM, the product is indicated for the following uses:

- [1] Core build-up in conjunction with self- or dual-cured core build-up materials
- [2] Cementation in conjunction with self- or dual-cured composite resin cements

## III. CONTRAINDICATIONS

Patients with a history of hypersensitivity to methacrylate monomers

## IV. PRECAUTIONS

### 1. Safety precautions

1. This product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylate monomers or any other components.
2. If the patient demonstrates a hypersensitivity reaction such as rash, eczema, features of inflammation, ulcer, swelling, itching, or numbness, discontinue use of the product and seek medical attention.
3. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect them in the event of splashing material.
4. If the product comes in contact with human body tissues, take the following actions:
  - <If the product gets in the eye>  
Immediately wash the eye with copious amounts of water and consult a physician.
  - <If the product comes in contact with the skin or the oral mucosa>  
Immediately wipe the area with a cotton pellet or a gauze pad moistened with alcohol, and rinse with copious amounts of water.
5. Use caution to prevent the patient from accidentally swallowing the product.
6. Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomers or any other components.
7. Dispose of this product as a medical waste to prevent infection.

### 2. Handling and manipulation precautions

1. The product must not be used for any purposes other than specified in [II.INDICATIONS FOR USE].
2. The use of this product is restricted to dental professionals.
3. Clean the cavity sufficiently to prevent poor bonding. If the adherend surface is contaminated with saliva or blood, wash it thoroughly and dry before bonding.
4. The product contains ethanol. Dry the entire adherend surface sufficiently by blowing mild air for more than 5 seconds until the mixture does not move. Adjust the air pressure according to the shape and size of the cavity and the prosthetic appliance. Use a vacuum aspirator to prevent the bond liquid from scattering.
5. After mixing the product with a bonding agent, use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 90 seconds after dispensing.
6. Do not use CLEARFIL DC Activator in conjunction with other bonding agents except those from the CLEARFIL BONDING SYSTEM.

### 3. Storage precautions

1. The product must be used by the expiration date indicated on the package.
2. The product must be refrigerated (2-8°C/ 36-46°F) when not in use, and should be brought to room temperature for more than 15 minutes before using. After taking out of the refrigerator, the product must be left standing until it comes to room temperature.
3. The container should be tightly capped immediately after use.
4. The product must be kept away from extreme heat, direct sunlight or a flame.
5. The product must be stored in a proper place where only dental practitioners can access.

## V. COMPONENTS AND INGREDIENTS

### 1) Components

See outer package for the items included in the package.

### 2) Ingredients

#### Principal ingredients

Ethanol (> 90%), Catalysts (< 5%), Accelerators (< 3%)

Units in parentheses are mass %.

## VI. CLINICAL PROCEDURES

CLEARFIL DC Activator can be used in combination with either CLEARFIL SE BOND 2 or CLEARFIL SE Protect. When using in conjunction with another CLEARFIL BONDING SYSTEM, refer to each specific bonding agent's instructions. Do not mix with any other adhesive not recommended in the Instructions for Use for CLEARFIL DC Activator.

### A. Standard procedure for core build-up

#### A-1. Isolation, moisture control, preparing root canal, post surface treatment, and acid-etching of tooth

Refer to the instructions for use of the core material or composite resin cement used.

#### A-2. Pretreatment of tooth

Apply the PRIMER of CLEARFIL SE BOND 2 or CLEARFIL SE Protect to the adherend surface, according to the manufacturer's instructions.

#### A-3. Bonding to tooth

1. Dispense one each drop of the product and the BOND of CLEARFIL SE BOND 2 or CLEARFIL SE Protect into a well of the dispensing dish and mix immediately before application.

#### [CAUTION]

Use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 90 seconds after dispensing.

2. Apply the mixture to the entire cavity wall with the applicator brush. Use caution not to allow saliva or exudate to contact the treated surfaces.
3. Dry the entire adherend surface thoroughly by blowing mild air for more than 5 seconds, until the bond does not move. Use a vacuum aspirator to prevent the bond liquid from scattering. Remove the excess mixture with a paper point. After wiping away the excess mixture, dry the adherend surface again, if necessary.

#### [NOTE]

To dry thoroughly, adjust the air pressure according to the shape and size of the adherend surface.

4. Light-cure is recommended for optimum performance. Regarding the light-curing time, refer to the instructions for use of the bonding agent used.

#### A-4. Placing the post and core build-up

Place the post and core build-up using a core material according to the manufacturer's instructions.

## B. Standard procedure for cementation

### B-1. Preparation of the cavity, prosthetic restorations

Refer to the instructions for use of the composite resin cement used.

### B-2. Pretreatment of tooth

Refer to section A-2.

### B-3. Bonding to tooth

Mix with the BOND, and apply it to the adherend surface referring to section A-3.

### B-4. Cementing

Cement the prosthetic restoration using composite resin cement according to the manufacturer's instructions for use.

#### [WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

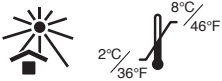
#### [NOTE]

- If a serious incident attributable to this product occurs, report it to the manufacturer through the EU Importer, and to the regulatory authorities of the country in which the user/patient resides.
- The Summary of Safety and Clinical Performance (SSCP) can be retrieved from the European Database on Medical Devices (EUDAMED) at <https://ec.europa.eu/tools/eudamed>.  
Basic UDI-DI: 458211076Conditioner1968M

#### [NOTE]

"CLEARFIL" and "SE BOND" are registered trademarks or trademarks of KURARAY CO., LTD.

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„CLEARFIL“ ir „SE BOND“ yra „KURARAY CO., LTD“ registruotieji prekės ženklai arba prekės ženklai.