# CLEARFIL™ DC CORE PLUS

|           |             |            |  |
|-----------|-------------|------------|--|
| <b>EN</b> | ENGLISH     | English    | DUAL-CURED CORE BUILD-UP MATERIAL                              |
| <b>FR</b> | FRANÇAIS    | French     | MATÉRIAU DE RECONSTITUTION CORONAIRE À PRISE DUALE             |
| <b>ES</b> | ESPAÑOL     | Spanish    | MATERIAL DE RESTAURACIÓN DE MUÑONES DE POLIMERIZACIÓN DUAL     |
| <b>IT</b> | ITALIANO    | Italian    | MATERIALE A POLIMERIZZAZIONE DUALE PER RICOSTRUZIONE MONCONI   |
| <b>NL</b> | NEDERLANDS  | Dutch      | DUAAL HARDEND STOMPPOUW MATERIAAL                              |
| <b>DE</b> | DEUTSCH     | German     | Dualhärtendes Stumpfaufbaumaterial                             |
| <b>SV</b> | SVENSKA     | Swedish    | DUBBELHÄRDANDE PELARUPPBYGGNADSMATERIAL                        |
| <b>NO</b> | NORSK       | Norwegian  | DOBBELTHERDET KJERNEOPPBYGGINGSMATERIALE                       |
| <b>FI</b> | SUOMI       | Finnish    | KAKSOISKOVETTEINEN PILARIN RAKENNUSMATERIAALI                  |
| <b>DA</b> | DANSK       | Danish     | DUALHÆRDENDE KERNOPBYGNINGSMATERIALE                           |
| <b>PT</b> | PORTUGUÊS   | Portuguese | MATERIAL DE POLIMERIZAÇÃO DUAL PARA CONSTRUÇÃO DE NÚCLEOS      |
| <b>EL</b> | ΕΛΛΗΝΙΚΑ    | Greek      | ΥΛΙΚΟ ΔΙΠΛΟΥ ΠΟΛΥΜΕΡΙΣΜΟΥ ΑΝΑΣΥΣΤΑΣΗΣ ΚΟΛΟΒΩΜΑΤΩΝ              |
| <b>TR</b> | TÜRKÇE      | Turkish    | DUAL SERTLEŞEN KOR YAPIM MATERYALİ                             |
| <b>PL</b> | POLSKI      | Polish     | PODWÓJNIE UTWARDZALNY MATERIAŁ DO ODBUDOWY ZREBU               |
| <b>RO</b> | ROMÂNĂ      | Romanian   | MATERIAL CU POLIMERIZARE DUBLĂ PENTRU RECONSTRUCȚIA BONTURILOR |
| <b>HR</b> | HRVATSKI    | Croatian   | DVOJNO STVRDNJAVAJUĆI MATERIJAL ZA KORONARNU NADOGRADNJU       |
| <b>HU</b> | MAGYAR      | Hungarian  | DUÁL KÖTÉSŰ CSONKFELÉPÍTŐ ANYAG                                |
| <b>SL</b> | SLOVENŠČINA | Slovenian  | DVOJNO STRJUJOČI MATERIAL ZA DOGRAJEVANJE KRNA                 |
| <b>CS</b> | ČESKY       | Czech      | DUÁLNĚ TUHNOUCÍ MATERIÁL PRO DOSSTAVBU PAHÝLU                  |
| <b>SK</b> | SLOVENSKY   | Slovak     | DUÁLNE VYTVRDZOVANÝ NADSTAVBOVÝ MATERIÁL NA PAHÝLE             |
| <b>BG</b> | БЪЛГАРСКИ   | Bulgarian  | ДВОЙНОПОЛИМЕРИЗИРАЩ МАТЕРИАЛ ЗА ИЗГРАЖДАНЕ НА ПЪНЧЕТА          |
| <b>UK</b> | УКРАЇНСЬКА  | Ukrainian  | МАТЕРІАЛ ДЛЯ НАРОЩУВАННЯ КУЛЬТИ ПОДВІЙНОГО ТВЕРДІННЯ           |
| <b>ET</b> | EESTI       | Estonian   | KAKSIKÕVASTUV HAMBAKÕNDI ÜLESEHITUSMATERJAL                    |
| <b>LV</b> | LATVIEŠU    | Latvian    | DIVĒJĀDI CIETĒJOŠS STUMBRA IZVEIDES MATERIĀLS                  |
| <b>LT</b> | LIETUVIŠKAI | Lithuanian | DVIGUBO KIETĖJIMO KULTIES ATKŪRIMO MEDŽIAGA                    |

**Kuraray Noritake Dental Inc.**

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

**EMERGO EUROPE**

Westervoortsedijk 60,  
6827 AT Arnhem, The Netherlands

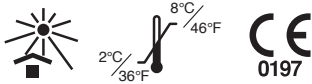
**Kuraray Europe GmbH** (EU Importer)

Philipp-Reis-Str. 4,  
65795 Hattersheim am Main, Germany  
Phone:+49 (0)69 305 35835 Fax:+49 (0)69 305 98 35835  
URL:<https://www.kuraraynoritake.eu>

# CLEARFIL™ DC CORE PLUS

## I. INTRODUCTION

CLEARFIL DC CORE PLUS is a dual-cured (light-cured with self-curing property), radiopaque two-component core build-up material supplied in an automix delivery system. CLEARFIL DC CORE PLUS is available in two shades: Dentin and White. The general clinical benefit of CLEARFIL DC CORE PLUS is to restore tooth function for the following INDICATIONS FOR USE.

## II. INDICATIONS FOR USE

Post cementation and core build-up

## III. CONTRAINDICATIONS

Patients with a history of hypersensitivity to methacrylate monomers

## IV. INCOMPATIBILITIES

Do not use eugenol-containing materials for temporary sealing since the eugenol can retard the curing process.

## V. PRECAUTIONS

### 1. Safety precautions

- The product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylate monomers or any other components.
- If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching, or numbness, discontinue the use of the product and seek medical attention.
- Avoid direct contact with the skin and/or soft tissue to prevent hypersensitivity. Wear gloves or take appropriate precautions when using the product.
- Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect the patient's eyes from splashing material.
- If the product comes in contact with human body tissues, take the following actions:
  - <If the product gets in the eye>  
Immediately wash the eye with copious amounts of water and consult a physician.
  - <If the product comes in contact with the skin or the oral mucosa>  
Immediately wipe the area with a cotton pledget or a gauze moistened with alcohol; rinse with copious amounts of water.
- Use caution when using the product to prevent swallowing.
- Avoid looking directly at the dental curing light when curing the product.
- Do not reuse the mixing tip and the guide tip to prevent cross-contamination. The mixing tip and the guide tip are single use only. Discard them after use.
- Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomers or any other components in the product.
- If the instruments associated with this product are damaged, use caution and protect yourself; immediately discontinue use.
- Dispose of this product as a medical waste to prevent infection.

### 2. Handling and manipulation precautions

[CLEARFIL DC CORE PLUS]

- CLEARFIL Universal Bond Quick, CLEARFIL TRI-S BOND PLUS or dual-cured bonding agents can be used with the product. When using in conjunction with another light-cured CLEARFIL BONDING SERIES, refer to each specific bonding agent's instructions. Do not use with any other adhesive not recommended in the Instructions for Use of bonding agents. In the areas where the light cannot reach easily (e.g. adherent areas masked by the post), "CLEARFIL Universal Bond Quick" and "CLEARFIL TRI-S BOND PLUS" can be self-cured in contact with the product.
- The product should not be used for purposes other than specified in [II. INDICATIONS FOR USE].
- The use of this product is restricted to dental professionals.
- Provide moisture and contamination control with the use of a rubber dam.
- Amalgam, temporary sealing material, lining material etc. remaining in the root canal will prevent the passage of light and the polymerization of the product. Completely remove any lining material when preparing the root canal.
- Clean the root canal sufficiently to prevent poor bonding. If the adherent surface is contaminated with saliva or tissue exudates, wash it thoroughly and dry before cementation or bonding.
- The product should be brought to room temperature for 15 minutes or more after it is removed from the refrigerator. If the syringe is not brought up to room temperature, you will find it difficult to squeeze out the paste.
- Do not use a lentulo spiral to insert the paste into the root canal; this can accelerate the polymerization of the paste beyond the desirable limit.
- Do not use the product in conjunction with other composite resins for core build-up. Intermixing materials may cause a change in physical properties, including a possible decrease, from the properties expected.
- Be careful to prevent unnecessary exposure to direct sunlight or surgery operating lights, otherwise the paste inside the tip may harden, leading to a shortened working time.
- Although the product is a dual-cured core build-up material, it must be light-cured according to the instructions.
- When light-curing the product, note the light-curing depth in the Instructions for Use.
- While putting the product into the root canal, be careful to avoid cross-contamination. Cover the entire syringe with a disposable barrier (e.g. a poly bag) to prevent saliva and blood contamination. Disinfect the syringe by wiping them with an absorbent cotton with alcohol both before and after use.
- Do not immerse the syringe in a disinfectant solution.

- Be careful not to cut your fingers on the sharp edges of instruments.

[Dental light-curing unit]

- Low light intensity causes poor adhesion. Check the lamp for service life and the dental curing light guide tip for contamination. It is advisable to check the dental curing light intensity using an appropriate light evaluating device at appropriate intervals.
- The emitting tip of the dental curing unit should be held as near and vertical to the resin surface as possible. If a large resin surface is to be light-cured, it is advisable to divide the area into several sections and light-cure each section separately.

## 3. Storage precautions

- The product must be used by the expiration date indicated on the package.
- The product must be refrigerated (2-8°C / 36-46°F) when not in use, and should be brought to room temperature before using.
- Keep away from extreme heat or direct sunlight.
- The product must be stored in proper places where only dental practitioners can access it.

## VI. COMPONENTS

Please see the outside of the package for contents and quantity.

- A Paste and B Paste :Dentin or White

Principal ingredients

- A Paste

Bisphenol A diglycidylmethacrylate (Bis-GMA, 1-10%), Hydrophobic aliphatic dimethacrylate (1-15%), Hydrophilic aliphatic dimethacrylate (1-10%), Hydrophobic aromatic dimethacrylate (1-10%), Silanated barium glass filler (50-80%), Silanated colloidal silica (1-10%), Colloidal silica (< 5%), dl-Camphorquinone (< 1%), Benzoyl peroxide (BPO, < 1%), Initiators (< 0.1%), Pigments (< 0.1%)

- B Paste

Triethyleneglycol dimethacrylate (< 5%), Hydrophilic aliphatic dimethacrylate (1-10%), Hydrophobic aromatic dimethacrylate (5-25%), Silanated barium glass filler (50-80%), Silanated colloidal silica (< 5%), Aluminum oxide filler (< 5%), Accelerators (< 5%)

The total amount of inorganic filler is approx. 52 vol%.

The particle size of inorganic fillers ranges from 0.01 µm to 20 µm.

Units in parentheses are mass %.

The mass % of each ingredient is content in the total amount of the two pastes, since the two pastes are in one container of dual-syringe.

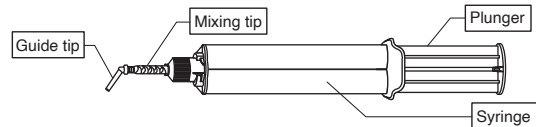
- Accessories

- Mixing tip
- Guide tip(L)
- Guide tip(S)


## VII. CLINICAL PROCEDURES

### SYRINGE PREPARATION


The device components.

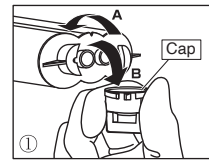
- Turn the cap 1/4 of a turn counterclockwise to align the projections on the cap with the grooves (A) on the syringe. Holding the base of the cap, remove the cap by twisting and pressing downward (B).

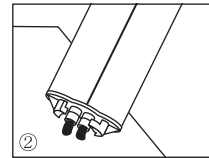
- Squeeze out small amounts of the two pastes, making sure equal amounts are being dispensed through the two outlets of the syringe. For the next and all subsequent uses, make sure equal amounts of the two pastes are being dispensed. If equal amounts of paste are not used, there is a possibility of poor polymerization.

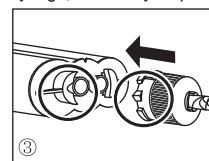
- Attach a mixing tip to the syringe, aligning the projections on it with the grooves in the syringe, as closely as possible.

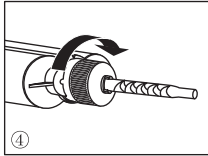
4. Turn the mixing tip 1/4 of a turn clockwise to lock in place.

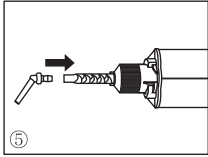5. Insert a guide tip into the mixing tip until it clicks.

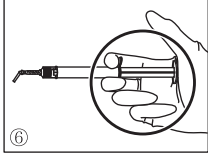6. The mixed pastes will be dispensed through the guide tip when the plunger is pressed.

[NOTE]

- After use, the syringe should be stored with the cap. When you put the cap back on the syringe before storage, make sure the cap is free of paste.
- When replacing an old mixing tip with a new one, turn the mixing tip 1/4 of a turn counterclockwise to align the projections of the mixing tip with the grooves on the syringe. Remove the mixing tip from the syringe by twisting and pressing downward.
- If the paste has hardened, making it difficult to squeeze the paste mixture out of the syringe, remove it using an appropriate instrument.
- When attaching the guide tip to the mixing tip or rotating the guide tip after attaching, use caution and hold the root of the guide tip to avoid bending.

**POST CEMENTATION AND CORE BUILD-UP**

1. Moisture control

In order to produce optimal results, avoid contamination of the treatment area with saliva or blood. A rubber dam is recommended to keep the tooth clean and dry.

2. Preparing root canal

Prepare and clean the root canal opening in the usual manner.

3. Try-in of the post

A post of the proper diameter is fitted to the prepared root canal and the length of the post adjusted.

4. Treatment of post surface

When using glass fiber posts

Treat the surface of the post according to the silane coupling agent's instructions (e.g. CLEARFIL CERAMIC PRIMER PLUS or CLEARFIL Universal Bond Quick).

When using metal posts

Treat the post surface according to the metal adhesive primer's instructions (e.g. ALLOY PRIMER or CLEARFIL Universal Bond Quick).

5. Syringe preparation

Select the desired shade of the product and prepare the syringe and accessories according to "SYRINGE PREPARATION".

6. Post cementation

6-a. When the following adhesives are used (Post cementation by the product)

- 1) Tooth surface treatment and bonding should be performed according to the instructions for use of CLEARFIL Universal Bond Quick, CLEARFIL TRI-S BOND PLUS or dual-cured bonding agents (e.g. CLEARFIL LINER BOND 2V or CLEARFIL PHOTO BOND).

[CAUTION]

Do not use with any other light-cured adhesive not recommend in the Instructions for Use of bonding agents.

- 2) Carefully squeeze the paste from the syringe directly into the root canal, being sure to avoid entrapment of air. You must start step 3) within 1 minute after application of the paste, since self-cure of the product is accelerated on contact with a surface on which the bonding agent has been applied.

[CAUTION]

- Timing for CLEARFIL DC CORE PLUS:  
<Working time (23°C / 73°F) : 3 minutes>  
<Setting time (37°C / 99°F) : 6 minutes>
- While putting the product into the root canal, be careful to avoid cross-contamination. Cover the entire syringe with a disposable barrier (e.g. a poly bag) to prevent saliva and blood contamination. Disinfect the syringe by wiping them with an absorbent cotton with alcohol both before and after use.

- 3) Insert the post into the canal.

- 4) Cure the paste using the dental curing unit\* according indicated on the table below. It is advisable to light-cure the margins well for strong bond.

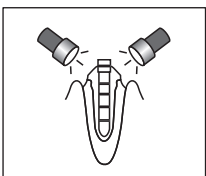
Table: Relationship between curing time and curing depth for each dental curing unit.

| Type    | Curing time | Curing depth |
|---------|-------------|--------------|
| Halogen | 10 seconds  | 1.5 mm       |
|         | 20 seconds  | 2.0 mm       |
| LED     | 10 seconds  | 1.5 mm       |
|         | 20 seconds  | 2.0 mm       |

\*Dental curing unit

| Type    | Light source           | Wavelength range and light intensity  |
|---------|------------------------|---|
| Halogen | Halogen lamp           | Light intensity <sup>2)</sup> of more than 300mW/cm <sup>2</sup> in wavelength range from 400 - 515 nm  |
| LED     | Blue LED <sup>1)</sup> | Light intensity <sup>3)</sup> of more than 300 mW/cm <sup>2</sup> in wavelength range from 400 - 515 nm |

1) Peak of emission spectrum: 450 - 480 nm

2) Evaluated according to ISO 10650-1.

3) Evaluated according to ISO 10650-2.

6-b. When PANAVIA F 2.0, PANAVIA V5 or PANAVIA SA Cement Universal is used.

Cement the post into the root canal according to the instructions for use of PANAVIA F 2.0, PANAVIA V5 or PANAVIA SA Cement Universal.

6-c. When another luting cement is used.

- 1) Cement the post into the root canal according to the instructions for use of the luting cement.
- 2) Apply the bonding agent to the entire adherent surface according to the Instructions for Use of the bonding system (e.g. CLEARFIL Universal Bond Quick, CLEARFIL TRI-S BOND PLUS, CLEARFIL DC BOND, CLEARFIL LINER BOND 2V or CLEARFIL PHOTO BOND).

7. Core build-up

- 1) After cementing the post in the root canal, squeeze the paste from the syringe directly around the post.

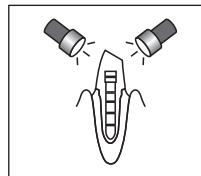
[CAUTION]

While squeezing the paste around the post, be careful to avoid cross-contamination. Cover the entire syringe with a disposable barrier (e.g. a poly bag) to prevent saliva and blood contamination. Disinfect the syringe by wiping them with an absorbent cotton with alcohol both before and after use.

[NOTE]

If necessary, a commercial matrix can be used to stabilize PASTE or tack curing with light for 5 seconds may be used to perform the initial polymerization of incremental paste placement.

- 2) After placing the paste around the post, light-cure the paste from both the lingual and the labial (buccal) sides, using a dental curing unit. See table "Relationship between curing time and curing depth for each dental curing unit" in 6-a. Leave the paste more than 6 minutes after light-curing when the thickness of the paste is greater than the specified curing depth on the table.

- 3) Preparation of an abutment tooth

After determining that the paste is completely cured, complete the construction of the core in the usual manner.

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proved to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.


[NOTE]

If a serious incident attributable to this product occurs, report it to the manufacturer and the regulatory authorities of the country in which the user/patient resides.

[NOTE]

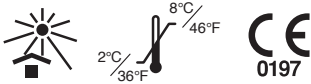"CLEARFIL" and "PANAVIA" are registered trademarks or trademarks of KURARAY CO., LTD.

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„CLEARFIL“ ir „PANAVIA“ yra „KURARAY CO., LTD“ registruotieji prekės ženklai arba prekės ženklai.