# CLEARFIL™ Performance Pro

EN	ENGLISH	English	DENTAL LIGHT-CURED RESTORATIVE COMPOSITE
FR	FRANÇAIS	French	COMPOSITE DE RESTAURATION DENTAIRE PHOTOPOLYMERISABLE
ES	ESPAÑOL	Spanish	COMPOSITE DE RESTAURACIÓN FOTOPOLIMERIZABLE DE USO DENTAL
IT	ITALIANO	Italian	COMPOSITO FOTOPOLIMERIZZABILE PER RESTAURI DENTALI
NL	NEDERLANDS	Dutch	DENTAAL LICHTUITHARDEND RESTAURATIEF COMPOSIT
DE	DEUTSCH	German	DENTALES lichthärtendes Komposit-Restaurationsmaterial
SV	SVENSKA	Swedish	DENTAL LJUSHÄRDANDE RESTAURERINGSKOMPOSIT
NO	NORSK	Norwegian	DENTALT LYSHERDET RESTAURERINGSKOMPOSIT
FI	SUOMI	Finnish	HAMPAAN VALOKOVETTEINEN YHDISTELMÄMUOVI
DA	DANSK	Danish	DENTALT LYSHÆRDENDE RESTAURERINGSKOMPOSIT
PT	PORTUGUÊS	Portuguese	COMPÓSITO FOTOPOLIMERIZAVEL PARA USO DENTÁRIO
EL	ΕΛΛΗΝΙΚΑ	Greek	ΟΔΟΝΤΙΚΗ ΦΩΤΟΠΟΛΥΜΕΡΙΖΟΜΕΝΗ ΣΥΝΘΕΤΗ ΡΗΤΙΝΗ ΑΠΟΚΑΤΑΣΤΑΣΗΣ
TR	TÜRKÇE	Turkish	IŞIKLA SERTLEŞEN DENTAL RESTORATİF KOMPOZİT
PL	POLSKI	Polish	ŚWIATŁOUTWARDZALNY DENTYSTYCZNY KOMPOZYT DO UZUPEŁNIEŃ
RO	ROMÂNĂ	Romanian	COMPOZIT DE RESTAURARE DENTARĂ FOTOPOLIMERIZABIL
HR	HRVATSKI	Croatian	DENTALNI SVJETLOSNO POLIMERIZIRANI RESTAURACIJSKI KOMPOZIT
HU	MAGYAR	Hungarian	FOGÁSZATI, FÉNYRE KEMÉNYEDŐ RESTAURÁCIÓS KOMPOZIT
SL	SLOVENŠČINA	Slovenian	DENTALNI SVETLOBNO STRJUJOČI KOMPOZITNI MATERIAL ZA RESTAVRACIJE
CS	ČESKY	Czech	DENTÁLNÍ SVĚTLEM TUHNOUCÍ KOMPOZITNÍ MATERIÁL PRO NÁHRADY
SK	SLOVENSKY	Slovak	DENTÁLNY SVETLOM VYTVRDZOVANÝ VÝPLŇOVÝ KOMPOZIT
BG	БЪЛГАРСКИ	Bulgarian	СТОМАТОЛОГОЧЕН ФОТОПОЛИМЕРИЗИРАЩ ВЪЗСТАНОВИТЕЛЕН КОМПОЗИТ
UK	УКРАЇНСЬКА	Ukrainian	РЕСТАВРАЦІЙНИЙ КОМПОЗИТНИЙ МАТЕРІАЛ
ET	EESTI	Estonian	DENTAALNE VALGUSKÕVASTUV KOMPOSIIT-RESTAURATSIOONIMATERJAL
LV	LATVIEŠU	Latvian	ZOBĀRSTNIECĪBAS GAISMĀ CIETĒJOŠS RESTAURĀCIJAS KOMPOZĪTMATERIĀLS
LT	LIETUVIŠKAI	Lithuanian	ODONTOLOGINIS ŠVIESA KIETINAMAS RESTAURACINIS KOMPOZITAS

**Kuraray Noritake Dental Inc.**

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

**EMERGO EUROPE**

Westervoortsedijk 60,  
6827 AT Arnhem, The Netherlands

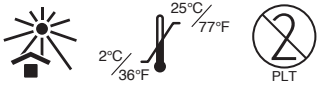
**Kuraray Europe GmbH (EU Importer)**

Philipp-Reis-Str. 4,  
65795 Hattersheim am Main, Germany  
Phone:+49 (0)69 305 35835 Fax:+49 (0)69 305 98 35835  
URL:<https://www.kuraraynoritake.eu>

1562R686R

# CLEARFIL™ Performance Pro

## I. INTRODUCTION

CLEARFIL Performance Pro is a light-cure, radiopaque restorative material which provides accurate color matching, high polishability and excellent physical properties, making it ideal for both anterior and posterior restorations. It is formulated with optimal viscosity, assuring easy handling and placement. It is packaged in traditional syringes or Pre-loaded tips (PLTs).

It has radiopacity equal to or greater than 1mm aluminum, and is classified as a Type 1 and Class 2 (Group 1) material by ISO 4049.

The general clinical benefit of this product is to restore tooth function in the following INDICATIONS FOR USE.

## II. INDICATIONS FOR USE

CLEARFIL Performance Pro is indicated for the following restorative applications:

- [1] Direct restorations for all cavity classes in anterior and posterior teeth
- [2] Direct veneers
- [3] Correction of tooth position and tooth shape (e.g. diastema closure, dwarfed tooth, etc.)
- [4] Intraoral repairs of fractured restorations

## III. CONTRAINDICATIONS

Patients with a history of hypersensitivity to methacrylate monomers

## IV. INCOMPATIBILITIES

Do not use eugenol containing materials for pulp protection or temporary sealing since the eugenol can retard the curing process.

## V. PRECAUTIONS

### 1. Safety precautions

1. This product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylate monomers or any other components.
2. If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching or numbness, discontinue use of the product and seek medical attention.
3. Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomers or any other components.
4. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect them in the event of splashing material.
5. If the product comes in contact with human body tissues, take the following actions:
  - <If the product gets in the eye>  
Immediately wash the eye with copious amounts of water and consult a physician.
  - <If the product comes in contact with the skin or the oral mucosa>  
Immediately wipe the area with a cotton pellet or a gauze moistened with alcohol, and rinse with copious amounts of water.
6. Use caution to prevent the patient from accidentally swallowing the product.
7. Syringe : Avoid sharing the same paste among different patients to prevent cross-contamination. Dispense the paste onto a mixing pad before use. Do not try to apply it directly from the syringe onto the patient's tooth.
8. PLT : Single- use only. Do not reuse a tip to prevent cross- contamination. Discard it after use and disinfect the dispenser according to its instructions for use.
9. PLT : Before dispensing the paste in the oral cavity, it is recommended to dispense a small amount of paste onto a mixing pad with a composite dispenser. During the preliminary dispensing, handle the dispenser quickly to ensure that the PLT does not fall out. A worn or malfunctioning composite dispenser may cause the PLT to dislodge and fall into the oral cavity, causing a possibility of injury or accidental ingestion. The dispenser should be checked regularly for excessive wear.
10. Dispose of this product as a medical waste to prevent infection.

### 2. Handling and manipulation precautions

[CLEARFIL Performance Pro]

1. The product must not be used for any purposes other than specified in [I.INDICATIONS FOR USE].
2. The use of this product is restricted to dental professionals.
3. Use a pulp capping agent in a cavity close to the pulp or in the event of accidental pulp exposure.
4. Use a rubber dam to prevent contamination and to control moisture.
5. Do not use the product in conjunction with other composite materials. The mixed materials may cause a change in physical properties, including a possible decrease, from the expected results.
6. The paste should be used immediately after being dispensed from the syringe or the tip. If the paste is to be left for a time before use, it should be covered with a light-blocking plate.
7. When using a PLT, use the dispenser with slow and steady pressure. Excessive force is not necessary.

### [Dental light-curing unit]

1. Do not look directly at the light source. Protective glasses are recommended.
2. Low light intensity causes poor adhesion. Check the lamp for service life and the dental curing light guide tip for contamination. It is advisable to check the dental curing light intensity using an appropriate light evaluating device at appropriate intervals.

3. The emitting tip of the dental curing unit should be held as near and vertical to the resin surface as possible. If a large resin surface is to be light-cured, it is advisable to divide the area into several sections and light-cure each section separately.
4. Check the conditions required to cure the paste by referring to the light-curing times listed in these instructions for use before using the product.

### 3. Storage precautions

1. The product must be used by the expiration date indicated on the package.
2. The product and shade guide must be stored at 2-25°C/ 36-77°F when not in use.
3. When the product is stored in the refrigerator, it should be left at room temperature for more than 15 minutes before each use. Failure to do this might cause breakage of the syringe or the tip.
4. The product must be kept away from extreme heat or direct sunlight.
5. After the paste has been dispensed, the syringe should be capped securely as soon as possible to prevent hardening by ambient light of the material, and to prevent foreign matter from entering the syringe.
6. The product must be stored in a proper place where only dental practitioners can access.

## VI. SHADES SYSTEM AND COMPONENTS

### 1. Shades

CLEARFIL Performance Pro is available in the following 4 shades:  
A2, A3, A3.5, B2

### 2. Components

See outer package for the items included in the package.

### 3. Ingredients

Silanated barium glass filler (30-80%)/Pre-polymerized organic filler (< 40%)/ Bisphenol A diglycidylmethacrylate (Bis-GMA) (5-15%)/ Hydrophobic aromatic dimethacrylate (5-15%)/Silanated colloidal silica (1-5%)/ Hydrophobic aliphatic dimethacrylate (1-5%)/ dl-Camphorquinone (< 0.1%)

Units in parentheses are mass %.

### [NOTE]

The total amount of filler is approx. 78 wt%, including inorganic filler of 40 vol%.  
The particle size of inorganic fillers ranges from 0.37µm to 1.5µm.

## VII. CLINICAL PROCEDURES

### A. Standard procedure 1 (INDICATIONS FOR USE [1] to [3])

- [1] Direct restorations for all cavity classes in anterior and posterior teeth
- [2] Direct veneers
- [3] Correction of tooth position and tooth shape (e.g. diastema closure, dwarfed tooth, etc.)

#### A-1. Shade taking

Clean the tooth with pumice and water to remove surface stains, then select the appropriate shade using the VITA classical A1-D4 shade guide.

#### A-2. Isolation and Moisture control

Avoid contamination of the treatment area from saliva or blood to produce optimal results. A rubber dam is recommended to keep the tooth clean and dry.

#### A-3. Cavity preparations

Remove any infected dentin and prepare the cavity in the usual manner.

#### A-4. Pulp protection

Any actual or near pulp exposure could be covered with a hard setting calcium hydroxide material or other indicated materials. Do not use eugenol materials for pulp protection.

#### A-5. Tooth surface treatment and bonding

Tooth surface treatment and bonding should be performed according to the instructions for use of the bonding system used.

#### A-6. Dispensing of the paste

- 6-1. Syringe : Dispense the necessary amount of paste from the syringe onto the mixing pad by turning the handle slowly in a clockwise manner.
- 6-2. PLT : Insert a PLT into the dispenser barrel and extrude the paste into the cavity according to the instructions for use of the dispenser .

### [CAUTION]

Use the dispenser with a slow and steady pressure. Excessive force is not necessary. Discard the tip after use.

#### A-7. Placement and light-curing of CLEARFIL Performance Pro

- 7-1. The application of a flowable composite resin or a opaque resin (e.g. CLEARFIL ST OPAQUER) to the cavity as an initial layer is optional. When adapting it, refer to its instructions for use.
- 7-2. Place the chosen shade of the paste into the cavity and light-cure with a dental curing unit. Considering the depth of cure, incremental curing may be required. Refer to the following table for the relationship between curing time and depth of cure with a visible light source.

Table: Relationship between curing time and depth of cure for type of light source.

Type of light source (Light intensity)	Curing time	Shade and depth of cure
		A2, A3, A3.5, B2
High-intensity BLUE LED* (More than 1500 mW/cm <sup>2</sup> )	Twice for 3 or 5 sec.	2.0 mm
Middle-intensity BLUE LED* (1100-1400 mW/cm <sup>2</sup> )	10 sec.	
Low-intensity BLUE LED* (800-1000 mW/cm <sup>2</sup> )	20 sec.	
High-intensity Halogen lamp (More than 800 mW/cm <sup>2</sup> )	10 sec.	

The effective wavelength range of each dental curing unit must be 400 - 515 nm.

\*Peak of emission spectrum: 450 - 480 nm

#### A-8. Finishing

Contour the restoration and adjust the occlusion using a fine diamond point. Polish with silicon rubber points or polishing discs in the usual manner.

### **B. Standard procedure II (INDICATIONS FOR USE [4])**

[4] Intraoral repairs of fractured restorations

#### B-1. Shade taking

Follow the same procedure described in "A-1".

#### B-2. Preparation of fractured surfaces

If necessary, bevel the margin with a diamond polishing point. Then follow the surface treatment procedures of the fractured area based on the surface material (e.g. metal-oxide ceramic, silica-based ceramic, composite, metal or tooth) according to the instructions for use for bonding, silane coupling agents or a metal adhesive primer.

#### B-3. Tooth surface treatment and bonding

Follow the same procedure described in "A-5".

#### B-4. Dispensing of the paste

Follow the same procedure described in "A-6".

#### B-5. Placement and light-curing of CLEARFIL Performance Pro

- 5-1. The application of a opaque resin (e.g. CLEARFIL ST OPAQUER) to the fractured surfaces as an initial layer is optional. When adapting it, refer to its instructions for use.
- 5-2. Place the chosen shade of the paste into the fractured surfaces and light-cure with a dental curing unit. Considering the depth of cure, incremental curing may be required. Refer to the table in section "A-7" for the relationship between curing time and depth of cure with a visible light source.

#### B-6 . Finishing

Follow the same procedure described in "A- 8".

#### [WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

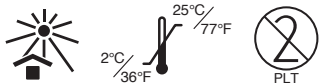
#### [NOTE]

- If a serious incident attributable to this product occurs, report it to the manufacturer through the EU Importer, and to the regulatory authorities of the country in which the user/patient resides.
- The Summary of Safety and Clinical Performance (SSCP) can be retrieved from the European Database on Medical Devices (EUDAMED) at <https://ec.europa.eu/tools/eudamed>.  
Basic UDI-DI: 458211076Composite178ZV

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"CLEARFIL" and "CLEARFIL ST" are registered trademarks or trademarks of KURARAY CO., LTD.
"VITA classical A1-D4" is a trademark of VITA Zahnfabrik H. Rauter GmbH & Co. KG.
„Kuraray Noritake Dental Inc.“ pakeičia visus produktus su įrodomais trūkumais.
„CLEARFIL“ ir „CLEARFIL ST“ yra „KURARAY CO., LTD“ registruotieji prekės ženklai arba prekės ženklai.
„VITA classical A1-D4“ yra prekės ženklas, priklausantis „VITA Zahnfabrik H. Rauter GmbH & Co. KG“.