# **CLEARFIL™** **PHOTO POSTERIOR**

**CE**  
0197

<b>EN</b>	ENGLISH	English	LIGHT- CURED COMPOSITE RESIN
<b>FR</b>	FRANÇAIS	French	RÉSINE COMPOSITE PHOTOPOLYMERISABLE
<b>ES</b>	ESPAÑOL	Spanish	RESINA COMPOSITE FOTOPOLIMERIZABLE
<b>IT</b>	ITALIANO	Italian	RESINA COMPOSITA FOTOPOLIMERIZZABILE
<b>NL</b>	NEDERLANDS	Dutch	LICHTUITHARDEND COMPOSITET
<b>DE</b>	DEUTSCH	German	Lichthärtendes Komposit
<b>SV</b>	SVENSKA	Swedish	LJUSHÄRDANDE KOMPOSITRESIN
<b>NO</b>	NORSK	Norwegian	LYSHERDET KOMPOSITRESIN
<b>FI</b>	SUOMI	Finnish	VALOKOVETTEINEN YHDISTELMÄMUOVI
<b>DA</b>	DANSK	Danish	LYSHÆRDENDE KOMPOSITRESIN
<b>PT</b>	PORTUGUÊS	Portuguese	RESINA DE COMPÓSITO FOTOPOLIMERIZAVEL
<b>EL</b>	ΕΛΛΗΝΙΚΑ	Greek	ΦΩΤΟΠΟΛΥΜΕΡΙΖΟΜΕΝΗ ΣΥΝΘΕΤΗ ΠΗΤΙΝΗ
<b>TR</b>	TÜRKÇE	Turkish	İŞIKLA SERTLEŞEN KOMPOZİT REZİN

**Kuraray Noritake Dental Inc.**  
1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

**EMERGO EUROPE**

Westervoortsedijk 60,  
6827 AT Arnhem, The Netherlands

**Kuraray Europe GmbH** (EU Importer)


Philipp-Reis-Str. 4,  
65795 Hattersheim am Main, Germany

Phone: +49 (0)69 305 35835 Fax: +49 (0)69 305 98 35835  
URL: <https://www.kuraraynoritake.eu>

1561R493R

# CLEARFIL™ PHOTO POSTERIOR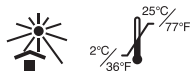

## I. INTRODUCTION

CLEARFIL PHOTO POSTERIOR is a light-curing restorative composite for posterior teeth. A range of 6 shades permits the construction of esthetic and durable posterior restorations. It has radiopacity equal to or greater than 1mm aluminum, and is classified as a Type 1 and Class 2 (Group 1) material by ISO 4049.

The general clinical benefit of this product is to restore tooth function in the following INDICATIONS FOR USE.

## II. INDICATIONS FOR USE

Composite resin restoration for posterior tooth; Class I, II and V cavities

## III. CONTRAINDICATION

Patients with a history of hypersensitivity to methacrylate monomers

## IV. INCOMPATIBILITY

Do not use eugenol-containing materials for pulp protection or temporary sealing since the eugenol can retard the curing process.

## V. PRECAUTIONS

### 1. Safety precautions

- The product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylate monomers or any other components.
- If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching or numbness, discontinue the use of the product and seek medical attention.
- Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomers or any other components.
- Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect them in the event of splashing material.
- If the product comes in contact with human body tissues, take the following actions:
  - <If the product gets in the eye>  
Immediately wash the eye with copious amounts of water and consult a physician.
  - <If the product comes in contact with the skin or the oral mucosa>  
Immediately wipe the area with a cotton pledget or a gauze moistened with alcohol, and rinse with copious amounts of water.
- Use caution to prevent the patient from accidentally swallowing the product.
- Avoid looking directly at the dental curing light when curing the product.
- Avoid sharing the same paste among different patients to prevent cross contamination.
- Dispense the paste on a mixing paper before use.
- Do not try to apply the paste directly from the syringe onto the patient's tooth.
- Dispose of this product as a medical waste to prevent infection.

### 2. Handling and manipulation precautions

- The use of this product is restricted to dental professionals.
- The product should be returned to room temperature before dispensing, if it has been taken out of a refrigerator. Failure to do this might cause breakage of the syringe.
- Use the composite within 5 minutes after dispensing. Exposure to the operating light will cause the composite to cure.
- Do not use the product in conjunction with the other composite resin. The mixed materials may cause a change in physical properties, possibly a decrease, from the properties expected.
- When light curing the product, note the depth of cure in this Instructions for Use.
- The emitting tip of the dental curing unit should be held as near and vertical to the resin surface as possible. If a large resin surface is to be light cured, it is advisable to divide the area into several sections and light-cure each section separately.
- Low light intensity causes poor adhesion. Check the lamp for service life and the dental curing light guide tip for contamination. It is advisable to check the dental curing light intensity using an appropriate light-evaluating device at appropriate intervals.

### 3. Storage precautions

- The product must be used by the expiration date indicated on the package.
- The product must be stored at 2 - 25°C / 36 - 77°F when not in use.
- The product must be kept away from extreme heat or direct sunlight.
- After the paste has been dispensed, turn the plunger counter-clockwise a half-turn to prevent excess composite from oozing out. Then, the syringe should be capped securely as soon as possible to prevent the paste inside the syringe from being hardened by ambient light, and to keep foreign matter from entering the syringe.
- The product must be stored in a proper place where only dental practitioners can access it.

## VI. SHADES SYSTEM AND COMPONENTS

### 1. Shades

CLEARFIL PHOTO POSTERIOR is available in the following 6 shades.  
US, UL, XL, UY, B and G

### 2. Components

See outer package for the items included in the package.

## 3. Ingredients

Silanated silica filler (25-50%)/  
Silanated barium glass filler (25-50%)/  
Bisphenol A diglycidylmethacrylate (Bis-GMA) (< 10%)/  
Silanated colloidal silica (< 10%)/  
Urethane tetramethacrylate (< 5%)/  
Triethyleneglycol dimethacrylate (TEGDMA) (< 5%)/  
d-Camphorquinone (< 1%) / Benzoyl peroxide (< 1%) / Accelerators (< 1%) /  
Pigments (< 0.1%)

## [NOTE]

Units in parentheses are mass %. The total amount of inorganic filler is approx. 72 vol%. The particle size of inorganic fillers ranges from 0.04 µm to 54 µm.

## VII. CLINICAL PROCEDURES

### 1. Shade taking

Choose a suitable shade of the product using the shade guide of CLEARFIL PHOTO BRIGHT. Two or more shades can be used to achieve a better colour match for the treating tooth.

### 2. Controlling the moisture

For the best results, prevent the treating surface from contamination by saliva or blood. A rubber dam is recommended to keep the working area clean and dry.

### 3. Preparing the cavity

Remove any infected dentin and prepare the cavity in the conventional manner.

### 4. Protecting the pulp

Any actual or near pulp exposure could be covered with a hard setting calcium hydroxide material. A liner or other under-layer is not required. Do not use eugenol materials for pulp protection.

### 5. Treating tooth surface and bonding

See the instructions for the bonding system (e.g. CLEARFIL TRI-S BOND, CLEARFIL SE BOND or CLEARFIL PHOTO BOND).

### 6. Applying and curing CLEARFIL PHOTO POSTERIOR

Apply the paste to the cavity and cure it with a dental curing unit according to the time on the table 1.

Table 1: Relation between curing time and depth of cure for each dental curing unit\*

Type	Curing time (Sec.)	Depth of cure (mm)	
		US, UL, XL	UY, B, G
Halogen	20	2.5	2.0
	40	3.5	3.0

\*Dental curing unit

Type	Light source	Range of wavelength and light intensity
Halogen	Halogen lamp	Light intensity <sup>1)</sup> of more than 300 mW/cm <sup>2</sup> in wavelength range from 400 - 515 nm

1) Evaluated according to ISO 10650-1.

## [NOTE]

If the cavity is Class II or deeper than the indicated depth of cure, incremental placement and light-curing should be performed.

### 7. Finishing

Finish the restoration with a fine diamond point. Polish with a silicone-rubber tip or polishing discs in the conventional manner.

## [WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

## [NOTE]

- If a serious incident attributable to this product occurs, report it to the manufacturer through the EU Importer, and to the regulatory authorities of the country in which the user/patient resides.
- The Summary of Safety and Clinical Performance (SSCP) can be retrieved from the European Database on Medical Devices (EUDAMED) at <https://ec.europa.eu/tools/eudamed>.  
Basic UDI-DI: 458211076Composite016Z2

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"CLEARFIL TRI-S BOND", "CLEARFIL SE BOND", "CLEARFIL PHOTO BOND" and "CLEARFIL" are registered trademarks or trademarks of KURARAY CO., LTD.