# CLEARFIL™ PORCELAIN BOND ACTIVATOR

<b>EN</b>	ENGLISH	English	SILANE COUPLING AGENT
<b>FR</b>	FRANÇAIS	French	AGENT DE COUPLAGE SILANE
<b>ES</b>	ESPAÑOL	Spanish	AGENTE SILANO
<b>IT</b>	ITALIANO	Italian	AGENTE DI ACCOPPIAMENTO DEL SILANO
<b>NL</b>	NEDERLANDS	Dutch	SILANISEERMIDDEL
<b>DE</b>	DEUTSCH	German	SILANHAFTVERMITTLER
<b>SV</b>	SVENSKA	Swedish	KOPPLINGSÄMNE MED SILAN
<b>NO</b>	NORSK	Norwegian	SILANBONDINGMATERIALE
<b>FI</b>	SUOMI	Finnish	SILAANI OR KERAMIAN ESİKÄSITTELYNESTE
<b>DA</b>	DANSK	Danish	SILAN-PRIMER
<b>PT</b>	PORTUGUÊS	Portuguese	AGENTE À BASE DE SILANO
<b>EL</b>	ΕΛΛΗΝΙΚΑ	Greek	ΣΥΓΚΟΛΛΗΤΙΚΟΣ ΠΑΡΑΓΟΝΤΑΣ ΣΙΛΑΝΙΟΥ
<b>TR</b>	TÜRKÇE	Turkish	SILAN BAĞLAMA AJANI
<b>PL</b>	POLSKI	Polish	SILANOWY ŚRODEK SPRZĘGAJĄCY
<b>RO</b>	ROMÂNĂ	Romanian	AGENT DE CUPLARE SILANIC
<b>HR</b>	HRVATSKI	Croatian	SILANSKO SREDSTVO ZA VEZIVANJE
<b>HU</b>	MAGYAR	Hungarian	SZILÁN KÖTŐANYAG
<b>SL</b>	SLOVENŠČINA	Slovenian	SILANSKO SPOJNO SREDSTVO
<b>CS</b>	ČESKY	Czech	SILANOVÉ POJIVO
<b>SK</b>	SLOVENSKY	Slovak	SILÁNOVÉ SPOJIVO
<b>BG</b>	БЪЛГАРСКИ	Bulgarian	СИЛАНОВ СВЪРЗВАЩ АГЕНТ
<b>UK</b>	УКРАЇНСЬКА	Ukrainian	СИЛАНОВИЙ АПРЕТ
<b>ET</b>	EESTI	Estonian	SILAANI SIDUSAINE
<b>LV</b>	LATVIEŠU	Latvian	SILĀNA SAISTVIELA
<b>LT</b>	LIETUVIŠKAI	Lithuanian	SILANO SUJUNGIMO AGENTAS

**Kuraray Noritake Dental Inc.**

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

**EMERGO EUROPE**

Westervoortsedijk 60,  
6827 AT Arnhem, The Netherlands

**Kuraray Europe GmbH (EU Importer)**

Philipp-Reis-Str. 4,  
65795 Hattersheim am Main, Germany  
Phone:+49 (0)69 305 35835 Fax:+49 (0)69 305 98 35835  
URL:<https://www.kuraraynoritake.eu>

1561J069R

**CLEARFIL™ PORCELAIN BOND ACTIVATOR****I. INTRODUCTION**

CLEARFIL PORCELAIN BOND ACTIVATOR is used in combination with the CLEARFIL BONDING SYSTEM (e.g. CLEARFIL TRI-S BOND, CLEARFIL SE BOND, CLEARFIL SE Protect, CLEARFIL NEW BOND and CLEARFIL PHOTO BOND) to enhance bonding strength to porcelain or cured composite resin. The general clinical benefit of this product is to restore tooth function for the following INDICATIONS FOR USE.

**II. INDICATIONS FOR USE**

CLEARFIL PORCELAIN BOND ACTIVATOR is indicated for the following uses:

- Repair of porcelain or porcelain fused-to-metal crowns and bridges.
- Restoration of gingival porcelain areas where bonding to porcelain is necessary.
- Increased retention of porcelain inlays, onlays, crowns, and veneers.
- Increased retention of composite inlays and onlays.

**III. CONTRAINDICATION**

Patients with a history of hypersensitivity to methacrylate monomers

**IV. PRECAUTIONS** **1. Safety precautions**

1. Avoid using the product on patients with a known history of hypersensitivity to methacrylate monomers.
2. If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching, or numbness, discontinue the use of the product and seek medical attention.
3. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using product, cover the patient's eyes with a towel to protect the patient's eyes from splashing materials.
4. If the product comes in contact with the human body, take the following actions:
  - <If the product gets in the eye>  
Immediately wash the eye with copious amounts of water and consult a physician.
  - <If the product comes in contact with the skin>  
Immediately wipe it off with a cotton pledget moistened with alcohol or gauze and wash with copious amounts of water.
  - <If the product comes in contact with the mucous membrane>  
The product might stay between other medical devices (e.g. cotton rolls) and the mucous membrane. Immediately wipe it off with cotton pledget moistened with alcohol or gauze and wash with copious amounts of water.
5. Use caution when using the product to prevent swallowing.
6. Do not use the same disposable brush tip for different patients to prevent cross contamination. Discard the tip after use and sterilize the brush tip handle after each patient.
7. Avoid direct contact with the skin and / or soft tissue to prevent hypersensitivity. Wear gloves or take appropriate precautions when using the product.
8. During visible light-curing, avoid looking directly at the light. Wear protective glasses.
9. Dispose of this product as a medical waste to prevent infection.

**2. Handling and manipulation precautions**

1. The use of this product is restricted to dental professionals.
2. If the treated adherent surface is contaminated with saliva or blood, rinse with water, dry, clean the area with an etching agent (e.g. K-ETCHANT GEL), and re-treat again.
3. If the product accumulates on the nozzle or the container, remove it by using a piece of gauze or a cotton pledget saturated with alcohol.
4. Do not use the product for any purposes other than those specified in INDICATIONS FOR USE.

**3. Storage precautions**

1. The product must be used by the expiration date indicated on the package.
2. The product must be refrigerated (2-8°C/36-46°F) when not in use, and should be brought to room temperature before using. After taking out of the refrigerator, the product must be left standing until it comes to room temperature.
3. Keep away from extreme heat or direct sunlight.
4. The product should be tightly capped immediately after use.
5. The product must be stored in proper places where only dental practitioners can access it.

**V. COMPONENTS**

Please see the outside of the package for contents and quantity.

**CLEARFIL PORCELAIN BOND ACTIVATOR**

Principal ingredients:

- Hydrophobic aromatic dimethacrylate (30-70%)
- 3-Methacryloxypropyl trimethoxysilane (30-70%)

Units in parentheses are mass %.

**VI. CLINICAL PROCEDURES**

CLEARFIL PORCELAIN BOND ACTIVATOR must be used in combination with either CLEARFIL TRI-S BOND, CLEARFIL SE BOND, CLEARFIL SE Protect, CLEARFIL NEW BOND or CLEARFIL PHOTO BOND.

**1. SURFACE PREPARATION****Phosphoric acid treatment of porcelain surface:**

Apply an etching agent (e.g. K-ETCHANT GEL) to the adherent surface according to the manufacturer's instructions and leave it in place for 5 seconds. The surface is then washed with water and dried.

**2. MIXING CLEARFIL PORCELAIN BOND ACTIVATOR WITH CLEARFIL BONDING SYSTEM****(1) CLEARFIL TRI-S BOND**

Dispense one drop each of BOND and CLEARFIL PORCELAIN BOND ACTIVATOR into a well of the mixing dish and mix them.

**(2) CLEARFIL SE BOND, CLEARFIL SE Protect**

Dispense one drop each of PRIMER and CLEARFIL PORCELAIN BOND ACTIVATOR into a well of the mixing dish and mix them.

**(3) CLEARFIL NEW BOND, CLEARFIL PHOTO BOND**

Dispense a drop of Catalyst Liquid, a drop of Universal Liquid and a drop of CLEARFIL PORCELAIN BOND ACTIVATOR into the mixing dish.

**3. APPLICATION OF THE MIXTURE**

Apply the mixture to the porcelain and metal surface with a disposable brush tip. After application, dry the entire adherent surface sufficiently by blowing mild oil-free air. When CLEARFIL TRI-S BOND or CLEARFIL PHOTO BOND is used, light-cure the mixture according to the manufacturer's instructions.

**4. FINISHING THE REPAIR, RESTORATION OR CEMENTATION****A. Repair or Restoration**

Apply composite resin (e.g. CLEARFIL MAJESTY Esthetic, CLEARFIL MAJESTY Flow and CLEARFIL MAJESTY Posterior) into the cavity, cure, finish and polish according to the manufacturer's instructions.

**B. Cementation**

Cement the prosthetic appliance using resin cement (e.g. PANAVIA F 2.0) according to the manufacturer's instructions.

**[WARRANTY]**

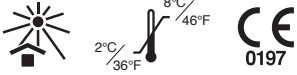
Kuraray Noritake Dental Inc. will replace any product that is proved to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

**[NOTE]**

If a serious incident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user/patient resides.

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"CLEARFIL", "CLEARFIL TRI-S BOND", "SE BOND", "CLEARFIL SE Protect", "CLEARFIL MAJESTY" and "PANAVIA" are registered trademarks or trademarks of KURARAY CO., LTD.