
ADHESION PRIMER

# CLEARFIL™ SA PRIMER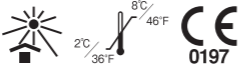

## ENGLISH INSTRUCTIONS FOR USE

**I. Introduction**

CLEARFIL SA PRIMER is an adhesion primer and contains a salicylic acid derivative monomer to promote the affinity to collagen fibers. This material can be used in conjunction with CLEARFIL PHOTO BOND for composite resin restorations. The general clinical benefit of this product is to restore tooth function for the following INDICATIONS FOR USE.

**II. INDICATIONS FOR USE**

CLEARFIL SA PRIMER in conjunction with K-ETCHANT GEL and CLEARFIL PHOTO BOND is indicated for composite resin restorations.

**III. Contraindications**

Patients with a history of hypersensitivity to methacrylate monomers

**IV. Incompatibility**

Do not use eugenol containing materials for pulp protection and temporary sealing, as they retard the curing process of CLEARFIL SA PRIMER.

**V. Precautions**  **Safety precautions**

- This product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylate monomers or any other components.
- If any hypersensitivity such as dermatitis occurs, discontinue use of the product and consult a physician.
- The mucous membrane may turn whitish when contacted with primer due to the coagulation of protein. This is a temporary phenomenon that will disappear in a few days.
- Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect the patient eyes from splashing material.
- If the product comes in contact with the human body, take the following measures:
  - <If the product gets in the eye> Immediately wash the eye with copious amounts of water and consult a physician.
  - <If the product comes in contact with the skin> Immediately wipe it off with a cotton pledget moistened with alcohol or gauze and wash with copious amounts of water.
- Use caution to prevent the patient from accidentally swallowing the product.
- Avoid looking directly at the curing light when curing the product.
- Keep the product out of reach of chidren. Others

Note that CLEARFIL SA PRIMER is flammable. Do not use or store it near open flames.

- Dispose of this product as a medical waste to prevent infection.

**Handling and manipulation precautions**

- The emitting tip of the visible light-curing activator should be held as near and vertical to the resin surface as possible. If a large resin surface is to be light cured, it is advisable to divide the area into several sections and light-cure each section separately.
- Low intensity of light causes poor adhesion. Check the lamp for service life and the visible light guide tip for contamination. It is advisable to check the light intensity of light-curing activator using an appropriate light-checker at periodic intervals.
- The use of this product is restricted to dental professionals.

**Storage precautions**

- Do not use after the expiration date. Note expiration date on the outside of package.
- The product must be refrigerated (2°C-8°C) when not in use; bring it to room temperature before using.
- Keep away from extreme heat or direct sunlight.
- Replace the bottle cap as soon as possible after liquid has been dispensed from the bottle.

**VI. COMPONENTS**

Please see the outside of the package for contents and quantity.

**Ingredients**

Principal ingredients:
Ethanol (70-90%)
Water (10-30%)
N-Methacryloyl-5-aminosalicylic acid (< 5%)
Units in parentheses are mass %.

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"CLEARFIL" es una marca registrada o marca comercial de KURARAY CO., LTD.