CLEARFIL™ SE BOND

EN	ENGLISH	English	RESIN-BASED DENTAL ADHESIVE SYSTEM
FR	FRANÇAIS	French	SYSTÈME DE COLLAGE DENTAIRE À BASE DE RÉSINE
ES	ESPAÑOL	Spanish	SISTEMA ADHESIVO DENTAL CON BASE DE RESINA
IT	ITALIANO	Italian	SISTEMA ADESIVO DENTALE A BASE DI RESINA
NL	NEDERLANDS	Dutch	DENTAAL ADHESIEF SYSTEEM OP BASIS VAN KUNSTHARS
DE	DEUTSCH	German	DENTALES ADHÄSIVSYSTEM AUF KUNSTHARZBASIS
SV	SVENSKA	Swedish	RESINBASERAT DENTALT ADHESIVSYSTEM
NO	NORSK	Norwegian	DENTALT ADHESIVSYSTEM BASERT PÅ RESIN
FI	SUOMI	Finnish	YHDISTELMÄMUOVIPOHJAINEN HAMPAAAN KIINNITYSAINE
DA	DANSK	Danish	RESINBASERET DENTALT ADHÆSIVSYSTEM
PT	PORTUGUÊS	Portuguese	SISTEMA ADESIVO DENTÁRIO À BASE DE RESINA
EL	ΕΛΛΗΝΙΚΑ	Greek	ΟΔΟΝΤΙΚΟ ΣΥΓΚΟΛΛΗΤΙΚΟ ΣΥΣΤΗΜΑ ΜΕ ΒΑΣΗ ΤΗ ΡΗΤΙΝΗ
TR	TÜRKÇE	Turkish	REZİN BAZLI DENTAL ADEZİV SİSTEM
PL	POLSKI	Polish	DENTYSTYCZNY SYSTEM ADHEZYJNY NA BAZIE ŻYWICY
RO	ROMÂNĂ	Romanian	SISTEM ADEZIV DENTAR PE BAZĂ DE RĂȘINI
HR	HRVATSKI	Croatian	DENTALNI ADHEZIVNI SUSTAV NA BAZI SMOLE
HU	MAGYAR	Hungarian	MŰGYANTA ALAPÚ FOGÁSZATI ADHEZÍV RENDSZER
SL	SLOVENŠČINA	Slovenian	DENTALNI ADHEZIVNI SISTEM NA OSNOVI SMOLE
CS	ČESKY	Czech	DENTÁLNÍ ADHEZIVNÍ SYSTÉM NA BÁZI UMĚLÉ PRYSKYŘICE
SK	SLOVENSKY	Slovak	DENTÁLNY ADHÉZNY SYSTÉM NA BÁZE UMELEJ ŽIVICE
BG	БЪЛГАРСКИ	Bulgarian	СТОМАТОЛОГИЧНА АДХЕЗИВНА СИСТЕМА НА БАЗАТА НА СМОЛА
UK	УКРАЇНСЬКА	Ukrainian	БОНДИНГОВА СИСТЕМА
ET	EESTI	Estonian	DENTAALNE VAIGUPÕHINE ADHESIIVSÜSTEEM
LV	LATVIEŠU	Latvian	ZOBĀRSTNIECĪBAS SVEĶUS SATUROŠA ADHEZĪVA SISTĒMA
LT	LIETUVIŠKAI	Lithuanian	DERVOS PAGRINDO ODONTOLOGINĖ ADHEZYVINĖ SISTEMA

Kuraray Noritake Dental Inc.

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

EMERGO EUROPE

Westervoortsedijk 60,
6827 AT Arnhem, The Netherlands

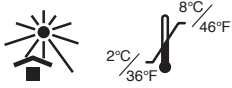
Kuraray Europe GmbH (EU Importer)

Philipp-Reis-Str. 4,
65795 Hattersheim am Main, Germany
Phone: +49 (0)69 305 35835 Fax: +49 (0)69 305 98 35835
URL: <https://www.kuraraynoritake.eu>

CLEARFIL™ SE BOND

I. INTRODUCTION

CLEARFIL SE BOND is a light-curing bonding system, and consists of a selfetching primer and a bonding agent. The primer offers simultaneous treatment of both dentine and enamel. The general clinical benefit of this product is to restore tooth function in the following INDICATIONS FOR USE.

II. INDICATIONS FOR USE

CLEARFIL SE BOND is indicated for the following cases :

- [1] Direct filling restorations using light-curing composite or compomer
- [2] Cavity sealing as a pretreatment for indirect restorations
- [3] Treatment of hypersensitive and/or exposed root surfaces
- [4] Intraoral repairs of fractured facing crowns made of porcelain, hybrid ceramics or composite resin using light-curing composite
- [5] Surface treatment of prosthetic appliances made of porcelain, hybrid ceramics and cured composite resin

III. CONTRAINDICATIONS

Patients with a history of hypersensitivity to methacrylate monomers

IV. POSSIBLE SIDE EFFECTS

The mucous membrane may turn whitish when contacted with PRIMER due to the coagulation of protein. This is a temporary phenomenon that will disappear in a few days.

V. INCOMPATIBILITY

Do not use eugenol containing materials for pulp protection and temporary sealing, as they retard the curing process.

VI. PRECAUTIONS

1. Safety precautions

1. Avoid use of the product for patients with a history of hypersensitivity to methacrylate monomers.
2. If any hypersensitivity such as dermatitis occurs, discontinue use of the product and consult a physician.
3. Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomer.
4. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect the patient's eyes from splashing material.
5. If the product comes in contact with the human body, take the following measures :
 - <If the product gets in the eye>
 - Immediately wash the eye with copious amounts of water and consult a physician.
 - <If the product comes in contact with the skin or the oral mucosa>
 - Immediately wipe it off with a cotton pledget moistened with alcohol or gauze and wash with copious amounts of water.
6. Use caution to prevent the patient from accidentally swallowing the product.
7. Avoid looking directly at the curing light when curing the product.
8. Keep the product out of reach of children.
9. Dispose of this product as a medical waste to prevent infection.
10. Avoid using the same product dispensed into a well of the dispensing dish and the same disposable brush tip on different patients to prevent cross contamination. The disposable brush tip is single-use only. Discard the disposable brush tip after use.

2. Handling and manipulation precautions

1. PRIMER and BOND polymerize by visible light (especially ultraviolet rays). Use the light blocking plate to avoid operating light or natural light (sunlight from windows) and use within 3 minutes from dispensing.
2. The emitting tip of the dental curing light should be held as near and vertical to the resin surface as possible. If a large resin surface is to be light cured, it is advisable to divide the area into several sections and light-cure each section separately.
3. Low intensity of light causes poor adhesion. Check the lamp for service life and the dental curing light guide tip for contamination. It is advisable to check the light intensity of light-curing activator using an appropriate light-checker at periodic intervals.
4. If the adherent surface extends to uncut enamel, apply a phosphoric acid etching agent (e.g. K-ETCHANT GEL) and leave it in place for 10 seconds before washing and drying.
5. The use of this product is restricted to dental professionals.

3. Storage precautions

1. Do not use after the expiration date. Note expiration date on the outside of package.
2. The product must be refrigerated (2-8°C/36-46°F) when not in use; bring it to room temperature before using.
3. Keep away from extreme heat or direct sunlight.
4. Replace the bottle cap as soon as possible after resin has been dispensed from the bottle.
5. The product must be stored in proper places where only dental practitioners can access it.

VII. COMPONENTS

1. Components

See outer package for the items included in the package.

- 1) CLEARFIL SE BOND
 - PRIMER
 - BOND

- 2) Accessories
 - Disposable brush tips
 - Brush tip handles*
 - Mixing dish*
 - Light blocking plate*
 - Outer case*

The PRIMER and BOND can be easily dispensed using this outer case.
*Consumables

2. Ingredients

Principal ingredients :

- 1) CLEARFIL SE BOND
 - PRIMER
 - 10-Methacryloyloxydecyl dihydrogen phosphate (MDP) (10-30%)
 - 2-Hydroxyethyl methacrylate (HEMA) (10-30%)
 - Hydrophilic aliphatic dimethacrylate (5-10%)
 - di-Camphorquinone (< 1%)

N,N-Diethanol-p-toluidine (< 5%)
Water (40-60%)

• BOND

10-Methacryloyloxydecyl dihydrogen phosphate (MDP) (< 10%)
Bisphenol A diglycidylmethacrylate (Bis-GMA) (25-45%)
2-Hydroxyethyl methacrylate (HEMA) (20-40%)
Hydrophobic aliphatic dimethacrylate (10-30%)
di-Camphorquinone (< 1%)
Accelerators (< 3%)
N,N-Diethanol-p-toluidine (< 1%)
Colloidal silica (5-15%)

Units in parentheses are mass %.

VIII. CLINICAL PROCEDURES

- A. [1] Direct filling restorations using light-curing composite or compomer
- [2] Cavity sealing as a pretreatment for indirect restorations
- [3] Treatment of hypersensitive and/or exposed root surfaces
- A-1. Cleaning tooth structure
An insufficiently cleaned cavity may impair the adhesive performance. Make sure the cavity is sufficiently cleaned.
- A-2. Moisture control
In order to produce optimal results, avoid contamination of the treatment area from saliva or blood. A rubber dam is recommended to keep the tooth dry.
- A-3. Cavity preparations
Remove any infected dentine using CARIES DETECTOR and prepare the cavity in the usual manner.
- A-4. Pulp protection
Any actual or near pulp exposure should be covered with a hard setting calcium hydroxide material. There is no need for cement lining or basing. Do not use eugenol materials for pulp protection.
- A-5. Acid etching uncut enamel
If there is a possibility of resin spreading over uncut enamel, apply K-ETCHANT GEL to the enamel, let it remain for 10 seconds, wash with water, and then dry.

[CAUTION]

The use of PRIMER alone does not sufficiently condition uncut enamel. Overfilling of resin onto unetched, uncut enamel could cause marginal discoloration.

A-6. Tooth surface treatment

1. Dispense the necessary amount of PRIMER into a well of the mixing dish immediately before application.

[CAUTION]

PRIMER polymerizes by visible light (especially ultraviolet rays). PRIMER will set to a gel if left under the natural light or operating light. Use the light blocking plate to avoid operating light or natural light (sunlight from windows) and use within 3 minutes from dispensing.

2. Apply PRIMER to the entire cavity wall with a sponge or a disposable brush tip. Leave it in place for 20 seconds. Use caution not to allow saliva or exudate to contact the treated surfaces for at least 20 seconds.
3. After conditioning the tooth surface for 20 seconds, evaporate the volatile ingredients with a mild oil-free air stream.

[CAUTION]

Avoid pooling of PRIMER. Also do not wash. Observe the drying method, and treatment time to ensure optimum adhesion. Avoid touching treated surface. If the treated surface is contaminated, wash it with water, dry, or clean with alcohol, and treat with PRIMER again.

A-7. Bonding

1. Dispense the necessary amount of BOND into a well of the mixing dish.

[CAUTION]

BOND polymerizes by visible light (especially ultraviolet rays). BOND will set to a gel if left under the operating light or natural light. Use the light blocking plate to avoid operating light or natural light (sunlight from windows) and use within 3 minutes from dispensing.

2. Apply BOND to the entire surface of the cavity with a sponge or a disposable brush tip.
3. After application, make the bond film as uniform as possible using a gentle oil-free air stream.

[CAUTION]

A strong air blast will scatter the bonding agent, resulting in poor adhesion.

4. Light-cure the BOND for 10 seconds with a dental curing light (irradiation wave length : 400-515nm, light intensity : >300mW/cm²).

A-8a. [1] Direct filling restorations using light-curing composite or compomer
Apply composite (e.g. CLEARFIL MAJESTY Esthetic, CLEARFIL MAJESTY Posterior) or compomer into the cavity, light-cure, finish and polish according to the manufacturer's instructions for use.

A-8b. [2] Cavity sealing as a pretreatment for indirect restorations
[3] Treatment of hypersensitive and/or exposed root surfaces
Apply a thin coat of flowable composite resin (e.g. CLEARFIL MAJESTY Flow) to the tooth, and light-cure the composite according to its' Instructions for Use. Remove unpolymerized resin with a cotton pledget moistened with alcohol.

B. [4] Intraoral repairs of fractured facing crowns made of porcelain, hybrid ceramics or composite resin using light-curing composite

B-1. Preparation of fractured surfaces

- 1) Facing material surface
Using a diamond point, remove a layer of the fractured surface, and place a bevel at the marginal area.
- 2) Metal surface
Roughen the metal surface with a diamond point.

B-2. Acid etching of facing material surface
Apply K-ETCHANT GEL to the facing material surface and leave it in place for 5 seconds before washing and drying. If the adherent surface extends to uncut enamel, apply K-ETCHANT GEL and leave it in place for 10 seconds before washing and drying.

B-3. Precious metal surface

When precious metal is used, apply ALLOY PRIMER according to the manufacturer's Instructions for Use.

B-4. Silane treatment

1. Dispense one drop each of PRIMER and CLEARFIL PORCELAIN BOND ACTIVATOR into a well of the mixing dish and mix them.

[NOTE]

PRIMER and CLEARFIL PORCELAIN BOND ACTIVATOR should be mixed immediately before application.

[CAUTION]

PRIMER polymerizes by visible light (especially ultraviolet rays). PRIMER or the mixture will set to a gel if left under the operating light or natural light. Use the light blocking plate to avoid operating light or natural light (sunlight from windows) and use within 3 minutes from dispensing.

2. Apply mixture to the facing material and metal surfaces with a sponge or a disposable brush tip.
3. Leave it in place for 5 seconds. Use caution not to allow saliva or exudate to contact the treated surfaces.
4. If the adherent surface extends to the tooth, leave mixture in place for 20 seconds.
5. After applying PRIMER, evaporate the volatile ingredients with a mild oilfree air stream. The treated metal surface must be sufficiently dried; if not, adhesion might be seriously impaired.

[CAUTION]

Do not wash treated surface. If the treated surface is contaminated with saliva, wash it with water, dry, or clean with alcohol, and treat with the mixture again.

B-5. Bonding

1. Dispense the necessary amount of BOND into a well of the mixing dish.

[CAUTION]

BOND polymerizes by visible light (especially ultraviolet rays). BOND will set to a gel if left under the operating light or natural light. Use the light blocking plate to avoid operating light or natural light (sunlight from windows) and use within 3 minutes from dispensing.

2. Apply BOND to the entire cavity wall with a sponge or a disposable brush tip.
3. Use a light air stream to make the bond film as uniform as possible.
4. Light-cure the BOND for 10 seconds with a dental curing light (irradiation wave length : 400-515nm, light intensity : >300mW/cm²).

B-6. Light-curing composite filling

Apply composite (e.g. CLEARFIL MAJESTY Esthetic, CLEARFIL MAJESTY Posterior) into the cavity, light-cure, finish and polish according to the manufacturer's instructions for use.

[NOTE]

Use opaque resin (e.g. CLEARFIL ST OPAQUER) for metal surface to prevent metal shine through.

- C. [5] Surface treatment of prosthetic appliances made of porcelain, hybrid ceramics and cured composite resin.

Silanating treatment on prosthetic appliances made of esthetic materials (porcelain, hybrid ceramics or composite resin) can be accomplished using PRIMER in conjunction with CLEARFIL PORCELAIN BOND ACTIVATOR.

C-1. Acid etching of adherent surface

Apply K-ETCHANT GEL to the adherent surface and leave it in place for 5 seconds before washing and drying.

C-2. Silane treatment

Use the same procedure described in B-4.

C-3. Cementing

Cement the prosthetic appliance using PANAVIA cement according to the manufacturer's instructions for use.

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proved to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

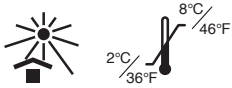
[NOTE]

- If a serious incident attributable to this product occurs, report it to the manufacturer through the EU Importer, and to the regulatory authorities of the country in which the user/patient resides.
- The Summary of Safety and Clinical Performance (SSCP) can be retrieved from the European Database on Medical Devices (EUDAMED) at <https://ec.europa.eu/tools/eudamed>. Basic UDI-DI: 458211076Bonding0644C

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"CLEARFIL", "SE BOND", "CLEARFIL MAJESTY", "CLEARFIL ST" and "PANAVIA" are registered trademarks or trademarks of KURARAY CO., LTD.