CLEARFIL™ SE Protect

EN	ENGLISH	English	DENTAL BONDING AGENT
FR	FRANÇAIS	French	AGENT DE LIAISON DENTAIRE
ES	ESPAÑOL	Spanish	AGENTE DE ADHESIÓN DE USO DENTAL
IT	ITALIANO	Italian	AGENTE DI ADESIONE DENTALE
NL	NEDERLANDS	Dutch	DENTALE BONDING
DE	DEUTSCH	German	Dentaler Haftvermittler
SV	SVENSKA	Swedish	DENTALT BONDINGSMATERIAL
NO	NORSK	Norwegian	DENTALT BINDINGSMIDDEL
FI	SUOMI	Finnish	HAMPAAN SIDOSAINE
DA	DANSK	Danish	DENTALT BONDINGMATERIALE
PT	PORTUGUÊS	Portuguese	AGENTE DE ADESÃO DENTÁRIO
EL	ΕΛΛΗΝΙΚΑ	Greek	ΟΔΟΝΤΙΚΟΣ ΣΥΓΚΟΛΛΗΤΙΚΟΣ ΠΑΡΑΓΟΝΤΑΣ
TR	TÜRKÇE	Turkish	DENTAL BONDING AJAN
PL	POLSKI	Polish	STOMATOLOGICZNY SYSTEM WIĄŻĄCY
RO	ROMÂNĂ	Romanian	AGENT DENTAR ADEZIV
HR	HRVATSKI	Croatian	DENTALNI ADHEZIV
HU	MAGYAR	Hungarian	FOGÁSZATI KÖTŐANYAG
SL	SLOVENŠČINA	Slovenian	DENTALNO VEZIVNO SREDSTVO
CS	ČESKY	Czech	DENTÁLNÍ POJIVO
SK	SLOVENSKY	Slovak	DENTÁLNE TMELIACE ČINIDLO
BG	БЪЛГАРСКИ	Bulgarian	СТОМАТОЛОГИЧЕН БОНДИНГ АГЕНТ
UK	УКРАЇНСЬКА	Ukrainian	СТОМАТОЛОГІЧНИЙ АДГЕЗИВ
ET	EESTI	Estonian	DENTAALNE ADHESIIV
LV	LATVIEŠU	Latvian	ZOBĀRSTNIECĪBAS SAISTVIELA
LT	LIETUVIŠKAI	Lithuanian	ODONTOLOGINIS RIŠIKLIS

Kuraray Noritake Dental Inc.

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

EMERGO EUROPE

Westervoortsedijk 60,
6827 AT Arnhem, The Netherlands

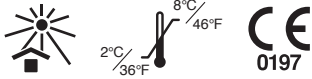
Kuraray Europe GmbH (EU Importer)

Philipp-Reis-Str. 4,
65795 Hattersheim am Main, Germany
Phone: +49 (0)69 305 35835 Fax: +49 (0)69 305 98 35835
URL: <https://www.kuraraynoritake.eu>

CLEARFIL™ SE Protect

I. INTRODUCTION

CLEARFIL SE Protect is a light-cure bonding system, consisting of an antibacterial self-etching primer and a fluoride-releasing bonding agent. The primer simultaneously treats both dentin and enamel. The general clinical benefit of this product is to restore tooth function for the following INDICATIONS FOR USE.

II. INDICATIONS FOR USE

CLEARFIL SE Protect is indicated for the following uses:

- [1] Direct restorations using light-cured composite resin or compomer.
- [2] Cavity sealing as a pretreatment for indirect restorations.
- [3] Treatment of hypersensitive and/or exposed root surfaces.
- [4] Intraoral repairs of fractured crowns/bridges made of porcelain, hybrid ceramics or composite resin using light-cured composite resin.
- [5] Surface treatment of prosthetic appliances made of porcelain, hybrid ceramics and cured composite resin.
- [6] Core build-ups using light- or dual-cured composite resin.
- [7] Cavity sealing under amalgam restorations.

III. CONTRAINDICATION

Patients with a history of hypersensitivity to methacrylate monomers

IV. POSSIBLE SIDE EFFECT

The oral mucosa may turn whitish when contacted by the PRIMER due to the coagulation of protein. This is a temporary phenomenon that will disappear in a few days.

V. INCOMPATIBILITIES

- [1] Do not use eugenol containing materials for pulp protection or temporary sealing, since the eugenol could retard the bonding system curing process.
- [2] Do not use hemostatics containing ferric compounds, since they may impair adhesion and the remaining ferric ion may cause discoloration in the layer or surrounding gingiva.

VI. PRECAUTIONS

1. Safety precautions

1. This product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylate monomers or any other components.
2. If any hypersensitivity occurs, such as dermatitis, discontinue use of the product and consult a physician.
3. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect the patient's eyes from splashing material.
4. If the product comes in contact with human body tissues, take the following actions:
 - <If the product gets in the eye>
 - Immediately wash the eye with copious amounts of water and consult a physician.
 - <If the product comes in contact with the skin or the oral mucosa>
 - Immediately wipe it off with a cotton pledget moistened with alcohol or gauze and wash with copious amounts of water.
5. Use caution to prevent the patient from accidentally swallowing the product.
6. Avoid looking directly at the dental curing light when curing the product.
7. Keep the product out of reach of children.
8. Avoid using the same disposable brush tip for different patients to prevent cross infection. Discard the tip after use and sterilize the brush tip handle.
9. This product contains a small amount of surface treated sodium fluoride (less than 1%). Using on children under the age of 6 may have a potential risk of fluorosis.
10. Dispose of this product as a medical waste to prevent infection.

2. Handling and manipulation precautions

1. PRIMER and BOND will set to a gel if left under an operating light or natural light (sunlight from windows). Use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 3 minutes from dispensing.
2. The emitting tip of the dental curing light should be held as near and vertical to the resin surface as possible. If a large resin surface is to be light cured, it is advisable to divide the area into several sections and light-cure each section separately.
3. Low intensity of light causes poor adhesion. Check the lamp for service life and the dental curing light guide tip for contamination. It is advisable to check the dental curing light intensity using an appropriate light evaluating device at appropriate intervals.
4. The use of this product is restricted to dental professionals.

3. Storage precautions

1. If the liquid may not flow easily, since it has not been used for a long term, shake the container before use.
2. Do not use after the expiration date. Note the expiration date on the outside of package.
3. CLEARFIL SE Protect must be refrigerated (2-8°C/36-46°F) when not in use; and should be brought to room temperature before using. After taking out of the refrigerator, the product must be left standing until it comes to room temperature; otherwise the liquid might drop more than necessary or ooze after use.
4. Keep away from extreme heat or direct sunlight.
5. Replace the bottle cap as soon as possible after resin has been dispensed from the bottle.

6. The product must be stored in a proper place where only dental practitioners can access it.

VII. COMPONENTS

Please see the outside of the package for contents and quantity.

1) PRIMER*1, 3 (antibacterial self-etching primer)

Principal ingredients:

- 10-Methacryloyloxydecyl dihydrogen phosphate (MDP) (10-30%)
- 12-Methacryloyloxydodecylpyridinium bromide (MDPB)*2, 3
- <antibacterial-monomer> (1-10%)
- 2-Hydroxyethyl methacrylate (HEMA) (25-45%)
- Hydrophilic dimethacrylate (5-15%)
- Water (25-45%)
- N,N'-Dimethylaminoethyl methacrylate (1-5%)

[NOTE]

- *1 This PRIMER has an antibacterial property against S. mutans.
- *2 This monomer makes a resin matrix after copolymerization with other monomers. It has the quaternary ammonium salt group, and exhibits the antibacterial properties in a monomer state.
- *3 The antibacterial properties of the PRIMER and MDPB were evaluated by in-vitro tests.

2) BOND (fluoride-bonding agent)

Principal ingredients:

- 10-Methacryloyloxydecyl dihydrogen phosphate (MDP) (1-10%)
- Bisphenol A diglycidylmethacrylate (Bis-GMA) (25-45%)
- 2-Hydroxyethyl methacrylate (HEMA) (20-40%)
- Hydrophobic dimethacrylate (10-30%)
- dl-Camphorquinone (< 1%)
- Accelerators (< 3%)
- N,N-Diethanol-p-toluidine (< 1%)
- Colloidal silica (1-10%)
- Surface treated sodium fluoride (< 1%)

Units in parentheses are mass %.

3) Accessories

- Disposable brush tips
- Brush tip handles*
- Mixing dish*
- Light blocking plate*
- Outer case*

[NOTE]

The PRIMER and BOND can easily be dispensed using the outer case container. The PRIMER and BOND shall be refrigerated when not in use.
*Consumables

VIII. CLINICAL PROCEDURES

A. Direct restorations, cavity sealing and treatment of hypersensitive teeth or exposed root surfaces using light-cured composite resin (or compomer)

A-1. Cleaning tooth structure

An adequately cleaned cavity assures maximum adhesive performance. Be sure the cavity is adequately cleaned.

A-2. Moisture control

In order to produce optimal results, avoid contamination of the treatment area from saliva or blood. A rubber dam is recommended to keep the tooth clean and dry.

A-3. Cavity preparations

Remove any infected dentin using CARIES DETECTOR as a guide and prepare the cavity in the usual manner.

[CAUTION]

The primer of this bonding system has an antibacterial property, but it is advisable to remove carious lesions thoroughly in the usual manner when preparing a cavity.

A-4. Pulp protection

Any actual or near pulp exposure could be covered with a hard setting calcium hydroxide material. There is no need for cement lining or basing. Do not use eugenol materials for pulp protection.

A-5. Acid etching uncut enamel

If there is a possibility of resin spreading over uncut enamel, apply K-ETCHANT Syringe to the enamel, let it remain for 10 seconds, wash with water, and then dry.

[CAUTION]

The use of PRIMER alone may not sufficiently condition uncut enamel. Overfilling of resin onto unetched, uncut enamel could cause marginal discoloration.

A-6. Tooth surface treatment

1. Dispense the necessary amount of PRIMER into a well of the mixing dish immediately before application.

[CAUTION]

PRIMER will set to a gel if left under an operating light or natural light (sunlight from windows). Use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 3 minutes from dispensing.

2. Apply PRIMER to the entire cavity wall with a disposable brush tip. Leave it in place for 20 seconds. Use caution not to allow saliva or exudate to contact the treated surfaces for at least 20 seconds.

3. After conditioning the tooth surface for 20 seconds, evaporate the volatile ingredients with a mild oil-free air stream.

[CAUTION]

Avoid pooling of PRIMER. Do not wash after applying PRIMER. Observe the drying method and treatment time to ensure optimum adhesion. Avoid touching the treated surface. If the treated surface is contaminated, wash it with water, dry, or clean with alcohol, and treat with PRIMER again.

A-7. Bonding

1. Dispense the necessary amount of BOND into a well of the mixing dish.

[CAUTION]

BOND will set to a gel if left under an operating light or natural light (sunlight from windows). Use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 3 minutes from dispensing.

2. Apply BOND to the entire surface of the cavity with a disposable brush tip.
3. After applying BOND, create a uniform bond film using a gentle oil-free air flow.

[CAUTION]

A strong air blast will scatter the bonding agent, resulting in poor adhesion.

4. Light-cure the BOND for 10 seconds with a dental curing light (irradiate wave length:400- 515nm).

A-8. Either A-8a or A-8b

A-8a. Direct restorations using light-cured composite resin (or compomer)

Apply composite resin (e.g. CLEARFIL MAJESTY ES-2) or compomer into the cavity, light-cure, finish and polish according to the manufacturer's instructions.

A-8b. Cavity sealing and treatment of hypersensitive teeth or exposed root surfaces

Apply a thin coat of flowable composite resin (e.g. CLEARFIL MAJESTY ES Flow) to the tooth, and light-cure the composite according to its' Instructions for Use. Remove unpolymerized resin with a cotton pledget moistened with alcohol.

B. Intraoral repairs of fractured crowns/bridges made of porcelain, hybrid ceramics or composite resin using lightcured composite resin

B-1. Preparation of fractured surfaces

- 1) Facing material surface
Using a diamond point, remove a layer of the fractured surface, and place a bevel at the marginal area.
- 2) Metal surface
Roughen the metal surface with a diamond point.

B-2. Acid etching of facing material surface

Apply K-ETCHANT Syringe to the facing material surface and leave it in place for 5 seconds before washing and drying. If the adherent surface extends to uncut enamel, apply K-ETCHANT Syringe to the enamel surface and leave it in place for 10 seconds before washing and drying.

B-3. Precious metal surface

When precious metal is used, apply ALLOY PRIMER according to the manufacturer's instructions.

B-4. Silane treatment

1. Dispense one drop each of PRIMER and CLEARFIL PORCELAIN BOND ACTIVATOR into a well of the mixing dish and mix them.

[NOTE]

PRIMER and CLEARFIL PORCELAIN BOND ACTIVATOR should be mixed immediately before application.

[CAUTION]

PRIMER or the mixture will set to a gel if left under an operating light or natural light (sunlight from windows). Use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 3 minutes from dispensing.

2. Apply mixture to the facing material and metal surfaces with a disposable brush tip.
3. Leave it in place for 5 seconds. Use caution not to allow saliva or exudate to contact the treated surfaces.
4. If the adherent surface extends to the tooth, leave mixture in place for 20 seconds.
5. After that, evaporate the volatile ingredients with a mild oil-free air stream. The treated metal surface must be sufficiently dried; if not, adhesion might be seriously impaired.

[CAUTION]

Do not wash treated surface. If the treated surface is contaminated with saliva, wash it with water, dry, or clean with alcohol, and treat with the mixture again.

B-5. Bonding

1. Dispense the necessary amount of BOND into a well of the mixing dish.

[CAUTION]

BOND will set to a gel if left under an operating light or natural light (sunlight from windows). Use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 3 minutes from dispensing.

2. Apply BOND to the entire cavity wall with a disposable brush tip.
3. After applying BOND, create a uniform bond film using a gentle oil-free air flow.

4. Light-cure the BOND for 10 seconds with a dental curing light (irradiate wave length:400-515nm).

B-6. Light-cured composite resin filling

Apply composite resin (e.g. CLEARFIL MAJESTY ES-2) into the cavity, light-cure, finish and polish according to the manufacturer's instructions.

[NOTE]

Use opaque resin such as CLEARFIL ST OPAQUER for metal surface to prevent metal shine through.

C. Treatment of prosthetic appliances

Silanating treatment of prosthetic appliances made of esthetic materials (porcelain, hybrid ceramics or composite resin) can be accomplished using PRIMER in conjunction with CLEARFIL PORCELAIN BOND ACTIVATOR.

C-1. Acid etching of adherent surface

Apply K-ETCHANT Syringe to the adherent surface and leave it in place for 5 seconds before washing and drying.

C-2. Silane treatment

Use the same procedure described in B-4.

C-3. Cementing

Cement the prosthetic appliance using PANAVIA cement according to the manufacturer's instructions.

D. Core build-ups using light- or dual-cured composite resin

D-1. Cleaning tooth structure

An adequately cleaned tooth surface assures maximum adhesive performance. Be sure the tooth surface is adequately cleaned.

D-2. Moisture control

In order to produce optimal results, avoid contamination of the treatment area from saliva or blood. A rubber dam is recommended to keep the tooth clean and dry.

D-3. Tooth preparations

Remove existing restorations and/or caries and place pin, post or matrix as needed.

D-4. Pulp protection

In case of pulp exposure or near pulp exposure, a hard setting calcium hydroxide material is recommended. Do not use eugenol materials for pulp protection.

D-5. Application of primer

1. Dispense the necessary amount of PRIMER into a well of the mixing dish immediately before application.

[CAUTION]

PRIMER will set to a gel if left under an operating light or natural light (sunlight from windows). Use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 3 minutes from dispensing.

2. Apply PRIMER to all the tooth surfaces (and the post if applicable) with a disposable brush tip. Leave it in place for 20 seconds. Use caution not to allow saliva or exudate to contact the treated surfaces for at least 20 seconds.
3. After conditioning all the tooth surfaces (and the post) for 20 seconds, evaporate the volatile ingredients with a mild oil-free air stream.

[CAUTION]

Avoid pooling of PRIMER. Do not wash after applying PRIMER. Observe the drying method and treatment time to ensure optimum adhesion. Avoid touching the treated surface. If the treated surface is contaminated, wash it with water, dry, or clean with alcohol, and treat with PRIMER again.

D-6. Bonding

1. Dispense the necessary amount of BOND into a well of the mixing dish.

[CAUTION]

BOND will set to a gel if left under an operating light or natural light (sunlight from windows). Use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 3 minutes from dispensing.

2. Apply BOND to all the tooth surfaces with a disposable brush tip.
3. After applying BOND, create a uniform bond film using a gentle oil-free air flow.

[CAUTION]

A strong air blast will scatter the bonding agent, resulting in poor adhesion.

4. Light-cure the BOND for 10 seconds with a dental curing light (irradiate wave length:400- 515nm).

D-7. Core build-ups

1. Place light- or dual-cured composite resin for core-build up (e.g. CLEARFIL PHOTO CORE), using a hand instrument.
2. Light-cure the core material according to the manufacture's instructions.
3. After curing, check the surface hardness with an explorer. Initial crown preparation may begin immediately.

E. Cavity sealing under amalgam restorations

E-1. Cleaning tooth structure

E-2. Moisture control

E-3. Cavity preparations

Prepare the cavity in the normal manner for an amalgam restoration.

E-4. Pulp protection

E-5. Acid etching uncut enamel

E-6. Tooth surface treatment

E-7. Bonding

Use the same procedures described in A-1 to A-7 for the above steps.

E-8. Amalgam filling

Mix amalgam powder and mercury, condense the mixture into the cavity, finish and polish according to the manufacture's instructions.

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

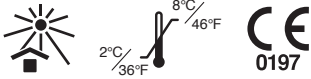
[NOTE]

If a serious incident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user/patient resides.

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"CLEARFIL", "PANAVIA" and "CLEARFIL MAJESTY" are registered trademarks or trademarks of KURARAY CO., LTD.
„CLEARFIL“, „PANAVIA“ ir „CLEARFIL MAJESTY“ yra „KURARAY CO., LTD“ registruotieji prekės ženklai arba prekės ženklai.