# K-ETCHANT GEL

<b>EN</b>	ENGLISH	English	DENTAL ETCHING AGENT
<b>FR</b>	FRANÇAIS	French	AGENT DE MORDANCAGE DENTAIRE
<b>ES</b>	ESPAÑOL	Spanish	AGENTE GRABADOR DE USO DENTAL
<b>IT</b>	ITALIANO	Italian	AGENTE MORDENZANTE DENTALE
<b>NL</b>	NEDERLANDS	Dutch	DENTAAL ETSMIDDEL
<b>DE</b>	DEUTSCH	German	DENTALES ÄTZMITTEL
<b>SV</b>	SVENSKA	Swedish	DENTALT ETSNINGSMEDEL
<b>NO</b>	NORSK	Norwegian	DENTALT ETSEMIDDEL
<b>FI</b>	SUOMI	Finnish	HAMPAAN ETSUSAINE or ETSUSAINE
<b>DA</b>	DANSK	Danish	DENTALT ÆTSNINGSMIDDEL
<b>PT</b>	PORTUGUÊS	Portuguese	AGENTE DE CONDICIONAMENTO DENTÁRIO
<b>EL</b>	ΕΛΛΗΝΙΚΑ	Greek	ΟΔΟΝΤΙΚΟΣ ΠΑΡΑΓΟΝΤΑΣ ΑΔΡΟΠΟΙΗΣΗΣ
<b>TR</b>	TÜRKÇE	Turkish	DENTAL ASITLEME AJANI
<b>PL</b>	POLSKI	Polish	WYTRAWIACZ STOMATOLOGICZNY
<b>RO</b>	ROMÂNĂ	Romanian	AGENT DENTAR GRAVANT
<b>HR</b>	HRVATSKI	Croatian	DENTALNO SREDSTVO ZA JETKANJE
<b>HU</b>	MAGYAR	Hungarian	FOGÁSZATI MARATÓANYAG
<b>SL</b>	SLOVENŠČINA	Slovenian	DENTALNO SREDSTVO ZA JEDKANJE
<b>CS</b>	ČESKY	Czech	DENTÁLNÍ LEPTACÍ PROSTŘEDEK
<b>SK</b>	SLOVENSKY	Slovak	DENTÁLNÝ LEPTACÍ PROSTRIEDOK
<b>BG</b>	БЪЛГАРСКИ	Bulgarian	СТОМАТОЛОГИЧЕН ЕЦВАЦ АГЕНТ
<b>UK</b>	УКРАЇНСЬКА	Ukrainian	СТОМАТОЛОГІЧНИЙ ПРОТРАВЛЮВАЧ
<b>ET</b>	EESTI	Estonian	DENTAALNE SÖÖVITUSAINE
<b>LV</b>	LATVIEŠU	Latvian	ZOBĀRSTNIECĪBAS KODINĀŠANAS LĪDZEKLIS
<b>LT</b>	LIETUVIŠKAI	Lithuanian	ODONTOLOGINĖ ĖSDINIMO MEDŽIAGA

**Kuraray Noritake Dental Inc.**

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

**EMERGO EUROPE**

Westervoortsedijk 60,  
6827 AT Arnhem, The Netherlands

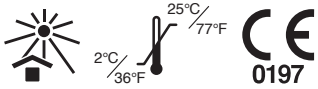
**Kuraray Europe GmbH (EU Importer)**

Philipp-Reis-Str. 4,  
65795 Hattersheim am Main, Germany  
Phone:+49 (0)69 305 35835 Fax:+49 (0)69 305 98 35835  
URL:<https://www.kuraraynoritake.eu>

1561J015R

# K-ETCHANT GEL

## I. INTRODUCTION

K-ETCHANT GEL is an etching gel and consists of 40% phosphoric acid aqueous solution and colloidal silica. The general clinical benefit of this product is to restore tooth function for the following INDICATIONS FOR USE.

## II. INDICATIONS FOR USE

Etching the enamel and dentin for adhesive restorations

## III. PRECAUTIONS

[Safety precautions]

- If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching, or numbness, discontinue the use of the product and seek medical attention.
- If the product comes in contact with human body tissues, take the following actions:
  - < If the product gets in the eye >  
Immediately rinse the eye with copious amounts of water and consult a physician.
  - < If the product comes in contact with the skin or the oral mucosa >  
The product may cause inflammation or erosion with its irritancy. Immediately wipe it off with a cotton pledget moistened with alcohol or gauze and rinse with copious amounts of water.
- Use caution to prevent the patient from accidentally swallowing the product.
- To prevent cross infection, avoid sharing the same gel or brush tip among different patients.
- Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity.
- Dispose of this product as a medical waste to prevent infection.

[Handling and manipulation precautions]

- The use of this product is restricted to dental professionals.
- Do not use the product for purposes other than listed in INDICATIONS FOR USE.
- When using this product, provide moisture control with a rubber dam if necessary.
- If the product adheres on clothes, wash off with water.
- The product may be hardened when it is not in use. In that case, the content returns to liquid as shaking the bottle with the cap.

[Storage precautions]

- The product must be used by the expiration date indicated on the package.
- The product must be stored at 2-25°C/ 36-77°F.
- The product must be stored in proper places where only dental practitioners can access it.

## IV. COMPONENTS

Please see the outside of the package for contents and quantity.

### K-ETCHANT GEL

Principal ingredients:

- Phosphoric acid
- Water
- Colloidal Silica
- Dye

## V. CLINICAL PROCEDURE

- Dispense one drop of the product into a well of mixing dish and apply it to the entire cavity surface (enamel and dentin) with a small brush.
- After 30-60 seconds, wash thoroughly and dry with an air syringe.

[Caution]

Care should be taken to prevent saliva from coming in contact with the etched tooth surface. Any contaminated tooth surface with saliva etc. must be re-etched.

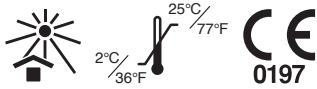
[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

[NOTE]

If a serious incident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user/patient resides.

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