PANAVIATM21 DISPENSER

CE
0197

EN	ENGLISH	English	DENTAL ADHESIVE RESIN CEMENT
FR	FRANÇAIS	French	CIMENT RÉSINE ADHÉSIF DENTAIRE
ES	ESPAÑOL	Spanish	CEMENTO DE RESINA ADHESIVO DE USO DENTAL
IT	ITALIANO	Italian	CEMENTO RESINA ADESIVO DENTALE
NL	NEDERLANDS	Dutch	DENTAAL COMPOSITET CEMENT
DE	DEUTSCH	German	DENTALER Kompositzement
SV	SVENSKA	Swedish	DENTAL VIDHÅFTANDE RESINCEMENT
NO	NORSK	Norwegian	DENTAL ADHESIV RESINSEMENT
FI	SUOMI	Finnish	HAMPAAN YHDISTELMÄMUOVISEMENTTI
DA	DANSK	Danish	DENTAL ADHÆSIV RESINCEMENT
PT	PORTUGUÊS	Portuguese	CIMENTO DE RESINA ADESIVA DENTÁRIO
EL	ΕΛΛΗΝΙΚΑ	Greek	ΟΔΟΝΤΙΚΗ ΠΗΤΙΝΩΔΗΣ ΣΥΓΚΟΛΛΗΤΙΚΗ ΚΟΝΙΑ
TR	TÜRKÇE	Turkish	DENTAL ADEZİV REZİN SİMAN
PL	POLSKI	Polish	ADHEZYJNY ŻYWICZNY CEMENT DENTYSTYCZNY
RO	ROMÂNĂ	Romanian	CIMENT DENTAR PE BAZĂ DE RĂȘINI ADEZIVE
HR	HRVATSKI	Croatian	DENTALNI ADHEZIVNI CEMENT NA BAZI SMOLE
HU	MAGYAR	Hungarian	FOGÁSZATI, MŰGYANTA ALAPÚ RAGASZTÓCEMENT
SL	SLOVENŠČINA	Slovenian	DENTALNI ADHEZIVNI KOMPOZITNI CEMENT
CS	ČESKY	Czech	DENTÁLNÍ KOMPOZITNÍ UPEVŇOVACÍ CEMENT
SK	SLOVENSKY	Slovak	DENTÁLNY KOMPOZITNÝ ŽIVICOVÝ CEMENT
BG	БЪЛГАРСКИ	Bulgarian	СТОМАТОЛОГИЧЕН АДХЕЗИВЕН КОМПОЗИТЕН ЦИМЕНТ
UK	УКРАЇНСЬКА	Ukrainian	СТОМАТОЛОГІЧНИЙ АДГЕЗИВНИЙ КОМПОЗИТНИЙ ЦЕМЕНТ
ET	EESTI	Estonian	DENTAALNE ADHESIIVNE KOMPOSIITSEMENT
LV	LATVIEŠU	Latvian	ZOBĀRSTNIECĪBAS ADHEZĪVS SVEĶU CEMENTS
LT	LIETUVIŠKAI	Lithuanian	ODONTOLOGINIS DERVINIS CEMENTAS FIKSAVIMUI

Kuraray Noritake Dental Inc.

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

EMERGO EUROPE

Westervoortsedijk 60,
6827 AT Arnhem, The Netherlands

Kuraray Europe GmbH (EU Importer)

Philipp-Reis-Str. 4,
65795 Hattersheim am Main, Germany
Phone:+49 (0)69 305 35835 Fax:+49 (0)69 305 98 35835
URL:<https://www.kuraraynoritake.eu>

1561R497R

PANAVIA™ 21 DISPENSER

CE
0197

I. INTRODUCTION

This dispenser is specially designed to extrude equal amounts of Catalyst and Universal Paste.

Refer to the Instructions for Use of PANAVIA 21 prior to use.

II. CLINICAL PROCEDURES

A. Dispensing procedure

1. Remove the syringe cap(1).
2. Slowly turn the dispensing knob(2) one complete turn to the right until it clicks.
[NOTE]
Always dispense paste in complete turn increments.
3. Dispense the extruded paste onto the mixing pad being carefully not to press the tip too hard. Discard the pastes if the amount of both pastes were not dispensed equally, and dispense it again.
4. Replace the syringe cap(1) as indicated on the insertion guide(3) when the product is stored.

B. Replacing syringe refill

1. Unlock the dispenser cover(4) using a coin or other small object, and remove.
2. Remove the empty syringe pair from the dispenser by gently and carefully pull on the joint(8); avoid damage to the dispenser.
3. Pull the clutch(5) back to the end.
4. Insert a refill pair into the dispenser chamber. Place the flange(7) into the flange holder(6).
5. Return dispenser cover(4).

[NOTE]

Dispose of this product as a medical waste to prevent infection.

[WARRANTY]




Kuraray Noritake Dental Inc. will replace any product that is proved to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

[NOTE]

If a serious incident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user/patient resides.

[NOTE]

"PANAVIA" is a registered trademark or trademark of KURARAY CO., LTD.

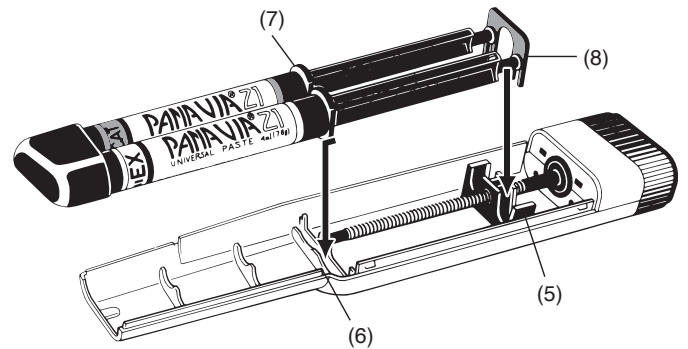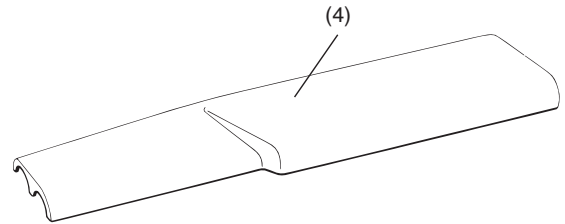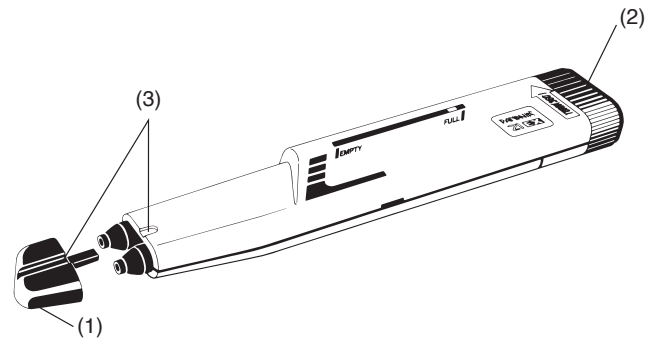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„PANAVIA” ir reģistrēta preču zīme vai uzņēmuma KURARAY CO., LTD preču zīme.
„PANAVIA“ yra „KURARAY CO., LTD“ registruotasis prekės ženklas arba prekės ženklas.