

252FP6212-02 06/2019

Dental Zirconia

 **KATANA™ Zirconia Block**



ENGLISH PRECAUTIONS

Dry milling is recommended.

If wet milling/grinding is performed by using cooling water contaminated by silica-based glass ceramics(lithium disilicate glass,etc.), the translucency of the zirconia may be reduced after baking. Before wet milling/grinding, clean the milling/grinding chamber, cooling water tank and filter insert. The cooling water must be changed in order to assure optimum results.



Kuraray Noritake Dental Inc.

300 Higashiyama, Miyoshi-cho, Miyoshi, Aichi 470-0293, Japan
TEL +81-561-32-8953 FAX +81-561-32-8976

CE 1639



Kuraray Europe GmbH

Philipp-Reis-Str. 4,
65795 Hattersheim am Main, Germany

US Distributed by

Kuraray America, Inc.

32 Old Slip, 7th Floor, New York, NY 10005