

TECHNICAL INFORMATION UPDATE



KATANA™ Zirconia Block

**18 Minutes Sintering
using the CEREC SpeedFire**

Speed Process for Chair Side

KATANA™ Zirconia Block, the innovative multi-layered zirconia block*¹, can be used with Dentsply Sirona's CEREC system*². Now the 18 minutes sintering program*³ is developed and installed into CEREC SpeedFire. New sintering technology makes it possible to fabricate full zirconia restoration in approx.35 minutes.



*¹ CL shade is not a multi layered block.

*² For using this product, CEREC software 4.5.2 or later is required, and for 18 minutes sintering, software 4.6 or later is required.

*³ In case that wall thickness is less than 3mm using dry milling.

*⁴ Dry milling is recommended. If wet milling/grinding is performed by using cooling water contaminated by silica-based glass ceramics (lithium disilicate glass, etc.), the translucency of the zirconia may be reduced after baking. Before wet milling/grinding, clean the milling/grinding chamber, cooling water tank and filter insert. The cooling water must be changed in order to assure optimum results.



Kuraray Noritake Dental Inc.

300 Higashiyama, Miyoshi-cho, Miyoshi, Aichi 470-0293, Japan

Website www.kuraraynoritake.com

- Before using this product, be sure to read the Instructions for Use supplied with the product.
- The specifications and appearance of the product are subject to change without notice.
- Printed color can be slightly different from actual color.

"KATANA" is a trademark of NORITAKE CO., LIMITED.

"CEREC" and "SpeedFire" are trademarks of Dentsply Sirona Inc.

US Distributed by

Kuraray America, Inc.

33 Maiden Lane, 6th Floor
New York, NY 10038

EC REP

Kuraray Europe, GmbH

Philipp-Reis-Str. 4
65795 Hattersheim am Main, Germany

R& Only

