

**TECHNICAL INFORMATION UPDATE**



**KATANA™ Zirconia Block**

**18 Minutes Sintering  
using the CEREC SpeedFire**

**Speed Process for Chair Side**

KATANA™ Zirconia Block, the innovative multi-layered zirconia block\*<sup>1</sup>, can be used with Dentsply Sirona's CEREC system\*<sup>2</sup>. Now the 18 minutes sintering program\*<sup>3</sup> is developed and installed into CEREC SpeedFire. New sintering technology makes it possible to fabricate full zirconia restoration in approx.35 minutes.



\*<sup>1</sup> CL shade is not a multi layered block.  
 \*<sup>2</sup> For using this product, CEREC software 4.5.2 or later is required, and for 18 minutes sintering, software 4.6 or later is required.  
 \*<sup>3</sup> In case that wall thickness is less than 3mm using dry milling.  
 \*<sup>4</sup> Dry milling is recommended. If wet milling/grinding is performed by using cooling water contaminated by silica-based glass ceramics (lithium disilicate glass, etc.), the translucency of the zirconia may be reduced after baking. Before wet milling/grinding, clean the milling/grinding chamber, cooling water tank and filter insert. The cooling water must be changed in order to assure optimum results.

**Kuraray Noritake Dental Inc.**  
 300 Higashiyama, Miyoshi-cho, Miyoshi, Aichi 470-0293, Japan  
 Website [www.kuraraynoritake.com](http://www.kuraraynoritake.com)

- Before using this product, be sure to read the Instructions for Use supplied with the product.
- The specifications and appearance of the product are subject to change without notice.
- Printed color can be slightly different from actual color.

"KATANA" is a trademark of NORITAKE CO., LIMITED.  
 "CEREC" and "SpeedFire" are trademarks of Dentsply Sirona Inc.

US Distributed by  
**Kuraray America, Inc.**  
 33 Maiden Lane, 6th Floor  
 New York, NY 10038

**Kuraray Europe, GmbH**  
 Philipp-Reis-Str. 4  
 65795 Hattersheim am Main, Germany

