

TECHNICAL INFORMATION UPDATE



KATANA™ Zirconia Block

**18 Minutes Sintering
using the CEREC SpeedFire**

Speed Process for Chair Side

KATANA™ Zirconia Block, the innovative multi-layered zirconia block*¹, can be used with Dentsply Sirona's CEREC system*². Now the 18 minutes sintering program*³ is developed and installed into CEREC SpeedFire. New sintering technology makes it possible to fabricate full zirconia restoration in approx.35 minutes.



*¹ CL shade is not a multi layered block.
 *² For using this product, CEREC software 4.5.2 or later is required, and for 18 minutes sintering, software 4.6 or later is required.
 *³ In case that wall thickness is less than 3mm using dry milling.
 *⁴ Dry milling is recommended. If wet milling/grinding is performed by using cooling water contaminated by silica-based glass ceramics (lithium disilicate glass, etc.), the translucency of the zirconia may be reduced after baking. Before wet milling/grinding, clean the milling/grinding chamber, cooling water tank and filter insert. The cooling water must be changed in order to assure optimum results.



Kuraray Noritake Dental Inc.

300 Higashiyama, Miyoshi-cho, Miyoshi, Aichi 470-0293, Japan

Website www.kuraraynoritake.com

US Distributed by

Kuraray America, Inc.

33 Maiden Lane, 6th Floor
New York, NY 10038



Kuraray Europe, GmbH

Philipp-Reis-Str. 4
65795 Hattersheim am Main, Germany

- Before using this product, be sure to read the Instructions for Use supplied with the product.
- The specifications and appearance of the product are subject to change without notice.
- Printed color can be slightly different from actual color.

"KATANA" is a trademark of NORITAKE CO., LIMITED.
 "CEREC" and "SpeedFire" are trademarks of Dentsply Sirona Inc.

R Only



CE 0120