# ALLOY PRIMER

|           |             |            |                                          |
|-----------|-------------|------------|------------------------------------------|
| <b>EN</b> | ENGLISH     | English    | Metal adhesive primer                    |
| <b>FR</b> | FRANÇAIS    | French     | AGENT DE PRETRAITEMENT DU METAL          |
| <b>ES</b> | ESPAÑOL     | Spanish    | Acondicionador adhesivo para metales     |
| <b>IT</b> | ITALIANO    | Italian    | Primer adesivo per metalli               |
| <b>NL</b> | NEDERLANDS  | Dutch      | Adhesief primer voor metalen             |
| <b>DE</b> | DEUTSCH     | German     | Metall-Primer                            |
| <b>SV</b> | SVENSKA     | Swedish    | Primer med vidhäftningsförmåga på metall |
| <b>NO</b> | NORSK       | Norwegian  | Primer som hefter til metall             |
| <b>FI</b> | SUOMI       | Finnish    | Metallin esikäsitteaine                  |
| <b>DA</b> | DANSK       | Danish     | Metal adhæsiv primer                     |
| <b>PT</b> | PORTUGUÊS   | Portuguese | Primer adesivo para metais               |
| <b>EL</b> | ΕΛΛΗΝΙΚΑ    | Greek      | συγκολλητικό primer μετάλλων             |
| <b>TR</b> | TÜRKÇE      | Turkish    | Metal adeziv primer                      |
| <b>PL</b> | POLSKI      | Polish     | Primer adhezyjny do metali               |
| <b>RO</b> | ROMÂNĂ      | Romanian   | Primer adeziv pentru materiale metalice  |
| <b>HR</b> | HRVATSKI    | Croatian   | Metalni adhezivni premaz                 |
| <b>HU</b> | MAGYAR      | Hungarian  | Fémadhezív primer                        |
| <b>SL</b> | SLOVENŠČINA | Slovenian  | Kovinski lepilni primer                  |
| <b>CS</b> | ČESKY       | Czech      | Primer na kovy                           |
| <b>SK</b> | SLOVENSKY   | Slovak     | Kovový primér                            |
| <b>BG</b> | БЪЛГАРСКИ   | Bulgarian  | Праймер за адхезия към метал             |
| <b>UK</b> | УКРАЇНСЬКА  | Ukrainian  | Адгезивний праймер для металу            |
| <b>ET</b> | EESTI       | Estonian   | Metallpraimer                            |
| <b>LV</b> | LATVIEŠU    | Latvian    | Metāla adhezīva praimeris                |
| <b>LT</b> | LIETUVIŠKAI | Lithuanian | Adhezyvinis metalo gruntas               |

**Kuraray Noritake Dental Inc.**

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

**EMERGO EUROPE**

Westervoortsedijk 60,  
6827 AT Arnhem, The Netherlands

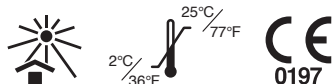
**Kuraray Europe GmbH (EU Importer)**

Philipp-Reis-Str. 4,  
65795 Hattersheim am Main, Germany  
Phone: +49 (0)69 305 35835 Fax: +49 (0)69 305 98 35835  
URL: <https://www.kuraraynoritake.eu>

1561R517R

# ALLOY PRIMER

## I. INTRODUCTION

ALLOY PRIMER is a metal adhesive primer.

The product is used to enhance the adhesion between dental metals and resin-based materials. The general clinical benefit of this product is to restore tooth function for the following INDICATIONS FOR USE.

## II. INDICATIONS FOR USE

- Adhesion of metal inlays, onlays, crowns, bridges and metal cast posts
- Repair of fractured resin-based facing crowns and porcelain-fused metal crowns and bridges
- Fabrication of removable dentures with a metal base, clasp or attachment
- Repair of dentures

## III. CONTRAINDICATION

Patients with a history of hypersensitivity to methacrylate monomers or acetone

## IV. PRECAUTION

### 1. Safety precautions

1. Avoid using of the product for patients with a history of hypersensitivity to methacrylate monomers or acetone.
2. If any hypersensitivity occurs, such as dermatitis, discontinue use of the product and consult a physician.
3. Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomers or acetone.
4. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect the patient's eyes from splashing material.
5. If the product comes in contact with human body tissues, take the following actions:
  - <If the product gets in the eye>  
Immediately rinse the eye with copious amounts of water and consult a physician.
  - <If the product comes in contact with the skin or the oral mucosa>  
Immediately wipe it off with a cotton pledget or gauze moistened with ethanol and rinse with copious amounts of water.
6. Use caution to prevent the patient from accidentally swallowing the product.
7. Do not use the same disposable brush tip for different patients to prevent cross contamination. Discard the tip after use and sterilize the brush tip handle after each patient.
8. If the patient begins to feel sick after inhaling the acetone contained in the product, encourage the patient to breathe copious amounts of fresh air.
9. Dispose of this product as a medical waste to prevent infection.

### 2. Handling and manipulation precautions

1. If the metal adherent surface is contaminated with saliva before or after the application of the product, clean the surface with a cotton pledget moistened with ethanol and apply the primer again.
2. In cementing precious metal cores with a resin cement (e.g. PANAVIA F2.0), use a tooth surface conditioner (e.g. PANAVIA F2.0 ED PRIMER II) after applying ALLOY PRIMER.
3. If the metal frame cannot be properly released from the working model after the preparation of a wax denture, investment or soldering, it is advisable to sandblast the frame before the preparation of the wax denture.
4. Provide moisture control with rubber dam, if necessary.
5. The use of this product is restricted to dental professionals.
6. Do not use the product for any other purposes other than those specified in INDICATIONS FOR USE.

### 3. Storage precautions

1. Note that ALLOY PRIMER is flammable. Do not use or store it near open flames.
2. Do not use after expiration date. Note expiration date on outside of package or container.
3. The product must be stored at room temperature (2-25°C/36-77°F). Avoid exposure to direct sunlight.
4. The product is volatile. The cap should be replaced as soon as possible after use. If the container is left uncapped, the volatile acetone will evaporate.

## V. COMPONENTS

### 1. Components

ALLOY PRIMER : 5 ml / 1 bottle

### 2. Ingredients

Principal ingredients

- Acetone (> 90%)
- 10-Methacryloyloxydecyl dihydrogen phosphate (MDP) (< 0.5%)
- 6-(4-Vinylbenzyl-N-propyl) amino-1,3,5-triazine-2,4-dithione (VBATDT) (< 1%)

Units in parentheses are mass %.

## VI. CLINICAL PROCEDURES

### A. Adhesion of metal inlays, onlays, crowns, bridges and metal cast posts

*When cementing a crown with a resin cement*

1. Treating the crown
  - 1) Sandblast the inner surface of the crown. Then, wash the crown with water and dry.
  - 2) Apply the product to the inner surface of the crown with a disposable brush tip and leave it for drying.
2. Treating metal core (when the metal core is built up with precious metal alloy)
  - 1) Roughen the metal core with a diamond point. Then, wash the surface with water and dry.
  - 2) Apply the product to the metal core with a disposable brush tip and leave it for drying.
- 3) Apply a tooth surface conditioner (e.g. PANAVIA F2.0 ED PRIMER II) to the metal abutment and tooth structure

#### [CAUTION]

Metal cores built up with precious metal alloy should be coated with a tooth surface conditioner (e.g. PANAVIA F2.0 ED PRIMER II) after the application of the product.

If the metal core is contaminated with saliva before or after the application of the product, it should be cleaned with a cotton pledget moistened with ethanol and coated with the primer again.

3. Cementing (see the Instructions for Use of a resin cement used)  
Cement with a resin cement (e.g. PANAVIA F2.0).

### B. Repair of fractured resin-based facing crowns and porcelain-fused metal crowns and bridges

*When repairing with a bonding agent and a light-cure composite resin*

1. Treating a fractured porcelain surface
  - 1) Remove a layer of porcelain with a diamond point and make a bevel at the marginal area.
  - 2) Treat the surface with a phosphoric acid etching agent (e.g. K-ETCHANT GEL) for the specified time, wash with water and dry.
2. Treating exposed metal
  - 1) Roughen with a diamond point. Then, wash with water and dry.
  - 2) Apply the product to the metal adherent surface with a disposable brush tip and leave it for drying.

#### [CAUTION]

If the metal adherent surface is contaminated with saliva before or after the application of the primer, clean the surface with a cotton pledget moistened with ethanol and apply the primer again.

3. Applying a bonding agent (see the Instructions for Use of a bonding agent used)  
Apply a bonding agent (e.g. CLEARFIL SE BOND, CLERFIL TRI-S BOND) to the porcelain and metal surfaces.
4. Filling a composite resin (see the Instructions for Use of a light-cure composite resin used)
  - 1) To mask out metal color, apply an opaque resin (e.g. CLEARFIL ST OPAQUER) to the metal surface and light-cure it for the specified time.
  - 2) Apply a composite resin (e.g. CLEARFIL AP-X) and light-cure it for the specified time.

### C. Fabrication of removable dentures with a metal base, clasp or attachment

*When using denture base acrylic resins (heat-cured acrylic resin, self-cured acrylic resin)*

1. Treating the metal frame  
Sandblast the metal frame. Then, wash the surface with water and dry.
2. Applying the product  
Apply the product to the metal adherent surface with a disposable brush tip and leave it for drying.
3. Curing a denture base acrylic resins (see the Instructions for Use of a denture base acrylic resin used)  
Fabricate a denture base acrylic resin and cure it.

### D. Repair of dentures

*When using self-cured acrylic resin*

1. Treating the metal adherent surface
  - 1) Grind the metal adherent surface with a carborundum point to expose a fresh surface.
  - 2) Sandblast the surface, wash and dry.
2. Applying the product  
Apply the product to the metal adherent surface with a disposable brush tip and leave it for drying.

3. Applying a self-cured acrylic resin (see the Instructions for Use of a self-cured acrylic resin used)  
Apply a self-cured acrylic resin on the metal adherent surface.

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proved to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

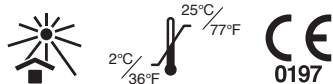
[NOTE]

If a serious incident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user/patient resides.

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"CLEARFIL", "CLEARFIL TRI-S BOND", "CLEARFIL AP-X" and "PANAVIA" are registered trademarks or trademarks of KURARAY CO., LTD.
„Kuraray Noritake Dental Inc.“ pakeičia visus produktus su įrodomais trūkumais.