# CLEARFIL™ NEW BOND

EN	ENGLISH	English	DENTAL BONDING AGENT
FR	FRANÇAIS	French	AGENT DE LIAISON DENTAIRE
ES	ESPAÑOL	Spanish	AGENTE DE ADHESIÓN DE USO DENTAL
IT	ITALIANO	Italian	AGENTE DI ADESIONE DENTALE
NL	NEDERLANDS	Dutch	DENTALE BONDING
DE	DEUTSCH	German	Dentaler Haftvermittler
SV	SVENSKA	Swedish	DENTALT BONDINGSMATERIAL
NO	NORSK	Norwegian	DENTALT BINDINGSMIDDEL
FI	SUOMI	Finnish	HAMPAAN SIDOSAINE
DA	DANSK	Danish	DENTALT BONDINGMATERIALE
PT	PORTUGUÊS	Portuguese	AGENTE DE ADESÃO DENTÁRIO
EL	ΕΛΛΗΝΙΚΑ	Greek	ΟΔΟΝΤΙΚΟΣ ΣΥΓΚΟΛΛΗΤΙΚΟΣ ΠΑΡΑΓΟΝΤΑΣ
TR	TÜRKÇE	Turkish	DENTAL BONDING AJAN
PL	POLSKI	Polish	STOMATOLOGICZNY SYSTEM WIAŻĄCY
RO	ROMÂNĂ	Romanian	AGENT DENTAR ADEZIV
HR	HRVATSKI	Croatian	DENTALNI ADHEZIV
HU	MAGYAR	Hungarian	FOGÁSZATI KÖTŐANYAG
SL	SLOVENŠČINA	Slovenian	DENTALNO VEZIVNO SREDSTVO
CS	ČESKY	Czech	DENTÁLNÍ POJIVO
SK	SLOVENSKY	Slovak	DENTÁLNE TMELIACE ČINIDLO
BG	БЪЛГАРСКИ	Bulgarian	СТОМАТОЛОГИЧЕН БОНДИНГ АГЕНТ
UK	УКРАЇНСЬКА	Ukrainian	СТОМАТОЛОГІЧНИЙ АДГЕЗИВ
ET	EESTI	Estonian	DENTAALNE ADHESIIV
LV	LATVIEŠU	Latvian	ZOBĀRSTNIECĪBAS SAISTVIELA
LT	LIETUVIŠKAI	Lithuanian	ODONTOLOGINIS RIŠIKLIS

## Kuraray Noritake Dental Inc.

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

## EMERGO EUROPE

Westervoortsedijk 60,  
6827 AT Arnhem, The Netherlands

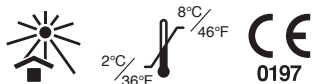
## Kuraray Europe GmbH (EU Importer)

Philipp-Reis-Str. 4,  
65795 Hattersheim am Main, Germany  
Phone: +49 (0)69 305 35835 Fax: +49 (0)69 305 98 35835  
URL: <https://www.kuraraynoritake.eu>

# CLEARFIL™ NEW BOND

## I. INTRODUCTION

CLEARFIL NEW BOND is a chemically-curing dentin and enamel bonding agent developed for use with composite restorative materials, especially chemically-curing composite. The general clinical benefit of this product is to restore tooth function for the following INDICATIONS FOR USE.

## II. INDICATIONS FOR USE

CLEARFIL NEW BOND is indicated for the following uses:  
Enamel and dentin bonding agent for direct composite restoration

## III. CONTRAINDICATION

The product should not be used to treat patients with a history of hypersensitivity to methacrylate monomer. Patients with previous allergic history to the product may develop rash or dermatitis.

## IV. INCOMPATIBILITY

Do not use a materials containing eugenol; eugenol has been shown to retard the polymerization and curing process.

## V. PRECAUTIONS

### 1. Safety precautions

- Avoid using the product on patients with a known history of hypersensitivity to methacrylate monomers.
- If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching, or numbness, discontinue the use of the product and seek medical attention.
- Use caution when using the product and prevent contact with the soft oral tissue or skin. If the product come in contact with the soft oral tissue or skin, wipe away with a cotton pledget moistened with alcohol, then immediately rinse with copious amounts of water. Prior to use the product, cover the patient's eyes with a towel to protect them from splashing materials. If the product gets in the eye, immediately rinse the eye with copious amounts of water and consult an ophthalmologist.
- Use caution to prevent swallowing any of the product.
- Do not use the same product for different patients to prevent cross contamination.
- Avoid direct contact with the skin and soft tissue to prevent hypersensitivity. Wear protective gloves when using the product.
- Dispose of this product as a medical waste to prevent infection.

### 2. Handling and manipulation precautions

- Provide moisture control with rubber dam, if necessary.
- Any exposed pulp or areas near the pulp should be covered with suitable material.
- Do not use a material containing eugenol for pulp protection or temporary sealing.
- Do not mix the product with other bonding agent. The specified performance might not be delivered.
- If the product has been stored in a refrigerator, it should be returned to room temperature before being dispensed. This is particularly important because, if the product is used before the liquid returns to room temperature, the liquid might be dispensed excessively, or it might ooze out the cap after use, due to expansion.
- After the liquid has been dispensed, the bottle should be capped securely as soon as possible to prevent the ethanol contained in the liquid from evaporating.
- The use of this product is restricted to dental professionals.
- Do not use the product for any other purposes other than those specified in INDICATIONS FOR USE.
- Before using the related products cited in Instructions for Use, read the manufacturer's instructions.

### 3. Storage precautions

- The product must be stored in a refrigerator (2 - 8°C / 36 - 46°F) when not being used, and should be brought to room temperature before using. Keep away from extreme heat, direct sunlight or flame.
- The product must be carefully stored and used by a licensed dental professional.
- The product must be used before the expiration date indicated on the package.

## VI. COMPONENTS

Components are available individually and the sets comprise a part of the components.

### 1. Components

CLEARFIL NEW BOND  
Catalyst Liquid: 6ml  
Universal Liquid: 6ml

### 2. Ingredients

#### 1) Catalyst Liquid

Principal ingredients

- Bisphenol A diglycidylmethacrylate (Bis-GMA) (20-40%)
- Hydroxyethyl methacrylate (HEMA) (20-40%)
- 10-Methacryloyloxydecyl dihydrogen phosphate (MDP) (5-25%)
- Hydrophobic aliphatic dimethacrylate (15-35%)
- Benzoyl peroxide (< 3%)

#### 2) Universal Liquid

Principal ingredients

- N,N'-Diethanol p-toluidine (< 3%)
- Sodium benzen sulfinate (< 5%)

- Ethanol (> 90%)

Units in parentheses are mass %.

## VII. CLINICAL PROCEDURES

Enamel and dentin bonding agent for direct composite restoration.

1. Moisture control  
In order to produce optimal results, avoid contamination of the treatment area from saliva or exudates. A rubber dam is recommended to keep the tooth clean and dry.
2. Cavity preparations  
Remove any infected dentin using a caries detection material(e.g. CARIES DETECTOR) according to the manufacture's instructions, and prepare the cavity in the usual manner.
3. Pulp protection  
Any actual or near pulp exposure could be covered with a hard setting calcium hydroxide material. There is no need for cement lining or basing. Do not use eugenol materials for pulp protection.
4. Etching  
Dispense one drop of an etching agent(e.g.K-ETCHANT GEL) into a well of mixing dish and apply it to the entire cavity surface with a small brush. After 30 to 60 seconds, wash thoroughly and dry with an air syringe. Care should be taken to prevent saliva from coming in contact with the etched tooth surface. Any contaminated tooth surface must be re-etched.
5. Application of CLEARFIL NEW BOND  
Dispense one drop of the Catalyst Liquid into a mixing dish. To this Catalyst drop add one drop of the Universal Liquid. Stir the two liquids quickly with a disposable brush tip. The mixed CLEARFIL NEW BOND must then be applied to the etched cavity surface within one minute. Carefully spread the bonding agent over the surface of the cavity with a gentle air flow for 2 or 3 seconds. In this way the volatile substances evaporate and a thin film of the bonding agent is formed which does not harden as long as it is in contact with oxygen.
6. Loading the filling material  
Load a composite resin prepared in advance (or mixed amalgam) into the cavity, self-cure according to the manufacturer's instructions.
7. Finishing  
Contour the restoration and adjust the occlusion using a fine diamond point. Polish with silicon rubber points or polishing discs.

### [NOTE]

- The loading of the composite resin should be accomplished within one minute after evaporating the ethanol.
- If you want to load the filling material into multiple teeth, it is good practice to use a composite resin syringe. After loading, shape the restoration properly.

### [WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proved to be defective. Kuraray Noritake Dental Inc. dose not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

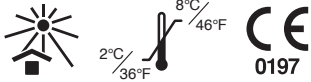
### [NOTE]

If a serious incident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user/patient resides.

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"CLEARFIL" is a registered trademark or trademark of KURARAY CO., LTD.
„CLEARFIL“ yra „KURARAY CO., LTD“ registruotasis prekės ženklas arba prekės ženklas.