**ENGLISH INSTRUCTIONS FOR USE****I. INTRODUCTION**

CLEARFIL PHOTO BRIGHT is a light-curing composite resin for anterior use. It has precise colour matching, excellent surface gloss and mechanical properties, making it ideal for esthetic applications. Its right viscosity means it adapts easily to the surface of the tooth and other layers of composite. The product is available in several translucent enamel colours and opaque dentin colours. It is classified as a Type 2 and Class 2 (Group 1) material by ISO 4049. The general clinical benefit of this product is to restore tooth function for the following INDICATIONS FOR USE.

**II. INDICATIONS FOR USE**

CLEARFIL PHOTO BRIGHT is recommended for the following restorative applications:

- Class III, IV and V restorations.
- Direct veneers

**III. CONTRAINDICATION**

Patients with a history of hypersensitivity to methacrylate monomers

**IV. INCOMPATIBILITY**

Do not use eugenol containing materials for pulp protection or temporary sealing, since the eugenol can retard the bonding system curing process.

**V. PRECAUTIONS****1. Safety precautions**

1. This product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylate monomers or any other components.
2. If any hypersensitivity such as dermatitis occurs, discontinue the use of the product and consult a physician.
3. Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomer.
4. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect the patient's eyes from splashing material.
5. If the product comes in contact with human body tissues, take the following actions:
  - < If the product gets in the eye>  
Immediately rinse the eye with copious amounts of water and consult a physician.
  - < If the product comes in contact with the skin or the oral mucosa>  
Immediately wipe it off with a cotton pledge moistened with alcohol or gauze and rinse with copious amounts of water.
6. Use caution to prevent the patient from accidentally swallowing the product.
7. Avoid looking directly at the dental curing light when curing the product.
8. To prevent cross infection, avoid sharing the same paste among different patients.
9. Dispense the paste on a mixing paper before use.
10. Do not try to apply the paste directly from the syringe onto the patient's tooth.
11. Dispose of this product as a medical waste to prevent infection.

**2. Handling and manipulation precautions**

1. The use of this product is restricted to dental professionals.
2. The product should be returned to room temperature before dispensing, if it has been taken out of a refrigerator. Failure to do this might cause breakage of the syringe.
3. The paste should be used as soon as practicable after being dispensed from the syringe. If the paste is to be left for a while before use, it should be covered with a light blocking plate.
4. Do not use the product in conjunction with the other composite resin. Mixing materials may cause a change in physical properties, possibly a decrease, from the properties expected.
5. If the adhesive surface extends to uncut enamel, apply a phosphoric acid etching agent (e.g., K-ETCHANT GEL) and leave it in place for 10 seconds before washing and drying.
6. When curing the product, note the light curing depth in this Instructions for Use.
7. The emitting tip of the dental curing light should be held as near and vertical to the resin surface as possible. If a large resin surface is to be light cured, it is advisable to divide the area into several sections and light-cure each section separately.
8. Low light intensity causes poor adhesion. Check the lamp for service life and the dental curing light guide tip for contamination. It is advisable to check the dental curing light intensity using an appropriate light-evaluating device at appropriate intervals.

**3. Storage precautions**

1. Do not use the product after the expiration date. Note the expiration date on the outside of package.
2. The product must be stored at 2 - 25°C / 36 - 77°F when not in use.
3. Keep away from extreme heat and direct sunlight.
4. After the paste has been dispensed, the syringe should be capped securely as soon as possible to prevent the paste inside the syringe from being hardened by ambient light, and to keep foreign matter from entering the syringe.
5. The product must be stored in proper places where only dental practitioners can access it.

**VI. SHADES SYSTEM AND COMPONENTS****1. Shades**

CLEARFIL PHOTO BRIGHT is available in 18 shades:  
UL(A1), US(A2), DY(A3), G(A3.5), XL(B0.5), L(B1), U(B2), UY(B2.5), Y(B3), YB(B4), DGO(OC2)\*, DG(C3), LO(OA1)\*, UO(OA2)\*, DYO(OA3)\*, YO(OB3)\*, YBO(OB4)\*, B(Cv)\*, "DGO(OC2)", LO(OA1), UO(OA2), DYO(OA3), YO(OB3), and YBO(OB4) are opaque shades.

**2. Components**

Please see the outside of the package for contents and quantity.

CLEARFIL PHOTO BRIGHT  
CLEARFIL PHOTO BRIGHT Shade guide

**3. Ingredients**

Pre-polymerized organic filler (40-60%/  
Silanated colloidal silica (10-20%)/  
Silanated silica filler (< 15%)/Silanated silica glass filler (< 15%)/  
Triethyleneglycol dimethacrylate (TEGDMA) (< 10%)/  
Bisphenol A diglycidylmethacrylate (Bis-GMA) (< 8%)/  
Urethane tetramethacrylate (< 5%)/  
Hydrophilic aliphatic dimethacrylate (< 3%)/  
Benzoyl peroxide (< 1%)/di-Camphorquinone (< 1%)

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**II. CLINICAL PROCEDURES****1. Shade taking**

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**5. Treating tooth surface and bonding**

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**6. Place and Applying and curing CLEARFIL PHOTO BRIGHT**

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Table 1: Relationship between curing time and curing depth for each dental curing unit.

Type	Curing time (Sec.)	Curing depth (mm)
Conventional halogen*	20	
Fast halogen*	10	1.5
Plasma arc*	5	
LED*	20	

**Dental curing unit**

Type	Light source	Range of wavelength and light intensity
Conventional halogen	Halogen lamp	Light intensity <sup>1)</sup> of 300-550 mW/cm <sup>2</sup> with wavelength range of 400-515 nm
Fast halogen	Halogen lamp	Light intensity <sup>1)</sup> over 550mW/cm <sup>2</sup> wavelength range from 400-515 nm
Plasma arc	Xenon lamp	Light intensity <sup>1)</sup> over 2000mW/cm <sup>2</sup> in wavelength range of 400-515 nm, and light intensity over 450 mW/cm <sup>2</sup> in wavelength range of 400-430 nm
LED	Blue LED <sup>2)</sup>	Light intensity <sup>1)</sup> over 300mW/cm <sup>2</sup> in wavelength range from 400-515 nm

1) Emission spectrum peak: 450-480nm

2) Evaluated according to ISO 10650-1.

3) Wavelength distribution and light intensity values measured with a spectroradiometer, calibrated using an IEC or NIST (National Institute of Standards and Technology) standard lamp

4) Evaluated according to ISO 10650-2.

**[NOTE]**

If you have selected two or more shades, a layer of the first colour must be applied and cured before a layer of the next colour is applied.

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**[WARNING]**

Kuray Noritake Dental Inc. will replace any product that is proved to be defective.

Kuray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

**[NOTE]**

If a serious incident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user/patient resides.

**[NOTE]**  
"CLEARFIL MAJESTY", "CLEARFIL TRI-S BOND", "CLEARFIL SE BOND", "CLEARFIL PHOTO BOND", "CLEARFIL ST" and "CLEARFIL" are registered trademarks or trademarks of KURARAY CO., LTD.

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Bisphénol A diglycidylmethacrylate (Bis-GMA) (< 8%)/

Urethane tetramethacrylate (< 5%)/

Hydrophilic aliphatic dimethacrylate (< 3%)/

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