EN	ENGLISH	English	DENTAL ADHESIVE RESIN CEMENT
FR	FRANÇAIS	French	CIMENT RÉSINE ADHÉSIF DENTAIRE
ES	ESPAÑOL	Spanish	CEMENTO DE RESINA ADHESIVO DE USO DENTAL
IT	ITALIANO	Italian	CEMENTO RESINA ADESIVO DENTALE
NL	NEDERLANDS	Dutch	DENTAAL COMPOSITET CEMENT
DE	DEUTSCH	German	DENTALER Kompositzement
SV	SVENSKA	Swedish	DENTAL VIDHÅFTANDE RESINCEMENT
NO	NORSK	Norwegian	DENTAL ADHESIV RESINSEMENT
FI	SUOMI	Finnish	HAMPAAN YHDISTELMÄMUOVISEMENTTI
DA	DANSK	Danish	DENTAL ADHÆSIV RESINCEMENT
PT	PORTUGUÊS	Portuguese	CIMENTO DE RESINA ADESIVA DENTÁRIO
EL	ΕΛΛΗΝΙΚΑ	Greek	ΟΔΟΝΤΙΚΗ ΠΗΤΙΝΩΔΗΣ ΣΥΓΚΟΛΛΗΤΙΚΗ ΚΟΝΙΑ
TR	TÜRKÇE	Turkish	DENTAL ADEZİV REZİN SİMAN
PL	POLSKI	Polish	ADHEZYJNY ŻYWICZNY CEMENT DENTYSTYCZNY
RO	ROMÂNĂ	Romanian	CIMENT DENTAR PE BAZĂ DE RĂȘINI ADEZIVE
HR	HRVATSKI	Croatian	DENTALNI ADHEZIVNI CEMENT NA BAZI SMOLE
HU	MAGYAR	Hungarian	FOGÁSZATI, MŰGYANTA ALAPÚ RAGASZTÓCEMENT
SL	SLOVENŠČINA	Slovenian	DENTALNI ADHEZIVNI KOMPOZITNI CEMENT
CS	ČESKY	Czech	DENTÁLNÍ KOMPOZITNÍ UPEVŇOVACÍ CEMENT
SK	SLOVENSKY	Slovak	DENTÁLNY KOMPOZITNÝ ŽIVICOVÝ CEMENT
BG	БЪЛГАРСКИ	Bulgarian	СТОМАТОЛОГИЧЕН АДХЕЗИВЕН КОМПОЗИТЕН ЦИМЕНТ
UK	УКРАЇНСЬКА	Ukrainian	СТОМАТОЛОГІЧНИЙ АДГЕЗИВНИЙ КОМПОЗИТНИЙ ЦЕМЕНТ
ET	EESTI	Estonian	DENTAALNE ADHESIIVNE KOMPOSIITSEMENT
LV	LATVIEŠU	Latvian	ZOBĀRSTNIECĪBAS ADHEZĪVS SVEĶU CEMENTS
LT	LIETUVIŠKAI	Lithuanian	ODONTOLOGINIS DERVINIS CEMENTAS FIKSAVIMUI

Kuraray Noritake Dental Inc.

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

EMERGO EUROPE

Westervoortsedijk 60,
6827 AT Arnhem, The Netherlands


Kuraray Europe GmbH (EU Importer)

Philipp-Reis-Str. 4,
65795 Hattersheim am Main, Germany
Phone: +49 (0)69 305 35835 Fax: +49 (0)69 305 98 35835
URL: <https://www.kuraraynoritake.eu>

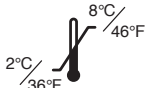PANAVIATM 21

I. INTRODUCTION

PANAVIA 21 is an adhesive resin cement with improved handling properties and increased bond strength to dentin. The properties of PANAVIA 21 are due to the adopted paste-to-paste mixing system and the use of a primer.

PANAVIA 21 consists of PANAVIA 21 Paste, ED PRIMER, OXYGUARD II, PANAVIA ETCHING AGENT V and accessories.

The general clinical benefit of PANAVIA 21 is to restore tooth function for the following INDICATIONS FOR USE.

II. INDICATIONS FOR USE

PANAVIA 21 is indicated in the following clinical situations:

- [1] Cementation of "adhesion" bridges or splints
- [2] Cementation of metal inlays, onlays, crowns or bridges
- [3] Cementation of porcelain or composite inlays, onlays or crowns
- [4] Cementation of root canal posts or cast metal cores
- [5] Bonded amalgam restorations

III. CONTRAINDICATIONS

Patients with a history of hypersensitivity to methacrylate monomers. Patients known to be allergic to any of the ingredients contained in this product.

IV. POSSIBLE SIDE EFFECTS

The oral mucosal membrane may turn whitish when contacted by ED PRIMER due to the coagulation of protein. This is usually a temporary phenomenon that will disappear in a few days.

Instruct patients to avoid irritating the affected area while brushing. In individual cases ulceration has been reported.

V. INCOMPATIBILITIES

- [1] Do not use eugenol-containing materials for pulp protection or temporary sealing since the eugenol can retard the curing process.
- [2] Do not use hemostatics containing ferric compounds since these materials may impair adhesion and may cause discoloration of the tooth margin or surrounding gingiva due to remaining ferric ions.

VI. PRECAUTIONS

1. Safety precautions

1. This product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylate monomers or any other components.
2. If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching or numbness, discontinue the use of the product and seek medical attention.
3. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect them in the event of splashing material.
4. If the product comes in contact with human body tissues, take the following actions:
 - <If the product gets in the eye>
Immediately wash the eye with copious amounts of water and consult a physician.
 - <If the product comes in contact with the skin or the oral mucosa>
Immediately wipe the area with a cotton pellet or a gauze moistened with alcohol; rinse with copious amounts of water.
5. Use caution to prevent the patient from accidentally swallowing the product.
6. Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomers or any other components.
7. Do not use the same disposable brush tip for different patients to prevent cross contamination. Discard the tip after use and sterilize the brush tip handle after each patient.
8. Dispose of this product as a medical waste to prevent infection.

2. Handling and manipulation precautions

[Common precautions]

1. The product must not be used for purposes other than specified in [II.INDICATIONS FOR USE].
2. The use of this product is restricted to dental professionals.
3. To prevent poor performance and handling, observe the specified light curing times and other handling requirements.
4. Use a pulp capping agent in a cavity close to the pulp or in the event of accidental pulp exposure.
5. Clean the cavity sufficiently to prevent poor bonding. If the adherend surface is contaminated with saliva or blood, wash it thoroughly and dry before cementation.
6. Do not mix the product with any other dental materials.
7. Use a rubber dam to prevent contamination and to control moisture.
8. If the instruments of this product are damaged, take care of yourself from any damages and immediately discontinue use them.

[PANAVIA 21 Paste]

1. Do not use a lentulo spiral to insert the paste into the root canal; this can accelerate the polymerization of the paste beyond the desirable limits.
2. If you want to place dental posts into multiple root canals of a posterior tooth, complete the post placement of one root canal before proceeding with another, and prevent the excess paste from entering the other root canals.

3. Any paste that has hardened on the gingiva will come off; however, try to remove the excess paste before it sets whenever possible.

[ED PRIMER]

1. ED PRIMER should be brought to room temperature before using. After taking out of the refrigerator, the product must be left standing for 15 minutes, until it comes to room temperature; otherwise, bubbles will form in the liquid when dispensing or it will ooze after use.
2. Dispense ED PRIMER Liquid A and B from each container by pointing the container downward as vertically as possible; careful dispensing is necessary to assure equal amounts of both liquids otherwise, the physical property of the product may be affected.
3. ED PRIMER Liquid A and B should be mixed whenever used. Do not use them separately or individually as a single agent.
4. If the treated surface is contaminated with saliva or tissue exudates, wash it with water, dry, or clean with ethanol, and treat with ED PRIMER again; otherwise, optimal adhesion will be impaired.

[PANAVIA ETCHING AGENT V]

If the treated adherent surface is contaminated with saliva or blood, rinse with water, dry and re-treat again with PANAVIA ETCHING AGENT V.

3. Storage precautions

1. The product must be used by the expiration date indicated on the package.
2. The product must be refrigerated (2 - 8°C/ 36 - 46°F) when not in use, and should be brought to room temperature for 15 minutes before using.
3. All the components except for OXYGUARD II and PANAVIA ETCHING AGENT V must be stored in a refrigerator (2 - 8°C/ 36 - 46°F) when not in use.
4. OXYGUARD II and PANAVIA ETCHING AGENT V must be stored at 2 - 25°C/ 36 - 77°F when not in use.
5. The product must be kept away from extreme heat or direct sunlight.
6. The product must be stored in a proper place where only dental practitioners can access it.
7. After the paste has been dispensed, the syringe should be capped securely as soon as possible to prevent the paste inside the syringe from being hardened by ambient light, and to prevent foreign matters from entering the syringe.
8. Replace the bottle cap as soon as possible after the liquid has been dispensed from the bottle.

VII. COMPONENTS

1. Components

Please see the outside of the package for contents and quantity.

- 1) PANAVIA21 Paste (Catalyst Paste and Universal Paste)
- 2) ED PRIMER (Liquid A and Liquid B)
- 3) OXYGUARD II
- 4) PANAVIA ETCHING AGENT V
- 5) Accessories
 - PANAVIA 21 DISPENSER
 - Mixing pad (S)
 - Mixing spatula
 - Mixing dish (PN)
 - Sponge pledget
 - Brush tip handle (white)
 - Brush tip handle (black)
 - Disposable brush tips (black)

2. Ingredients

- 1) PANAVIA 21 Paste (Catalyst Paste and Universal Paste)

Principal ingredients

- (1) Catalyst Paste
 - 10-Methacryloyloxydecyl dihydrogen phosphate (< 10%)
 - Hydrophobic aromatic dimethacrylate (< 10%)
 - Hydrophobic aliphatic dimethacrylate (< 5%)
 - Silanated silica filler (> 80%)
 - Colloidal silica (< 3%)
 - Benzoyl peroxide (< 1%)

- (2) Universal Paste

- Hydrophobic aromatic dimethacrylate (< 20%)
- Hydrophobic aliphatic dimethacrylate (< 15%)
- Hydrophilic aliphatic dimethacrylate (< 10%)
- Silanated titanium oxide (< 10%)
- Silanated barium glass filler (> 50%)
- Catalysts (< 1%)
- Accelerators (< 1%)
- Pigments (< 0.1%)

- 2) ED PRIMER (Liquid A and Liquid B)

Principal ingredients

- (1) Liquid A
 - 2-Hydroxyethyl methacrylate (HEMA) (30-50%)
 - 10-Methacryloyloxydecyl dihydrogen phosphate (20-40%)
 - N-Methacryloyl-5-aminosalicylic acid (< 1%)
 - Water (5-45%)
 - Accelerators (1-10%)
- (2) Liquid B
 - N-Methacryloyl-5-aminosalicylic acid (< 1.5%)
 - Water (> 80%)
 - Catalysts (1-10%)
 - Accelerators (1-10%)

- 3) OXYGUARD II

Principal ingredients

- Glycerol
- Polyethylene glycol
- Catalysts
- Accelerators

- Dyes

4) PANAVIA ETCHING AGENT V

Principal ingredients

- Phosphoric acid
- Polyvinylpyrrolidone
- Colloidal silica
- Water
- Dyes

Units in parentheses are mass %.

VIII. CLINICAL PROCEDURES

VIII-1 Cementation (INDICATIONS FOR USE [1] to [4])

- [1] Cementation of "adhesion" bridges or splints
- [2] Cementation of metal inlays, onlays, crowns or bridges
- [3] Cementation of porcelain or composite inlays, onlays or crowns
- [4] Cementation of root canal posts or cast metal cores

1. Enamel etching

Only uncut enamel requires etching when PANAVIA 21 is used as a luting agent.

Applications times are:

	Buccal	Lingual
PANAVIA ETCHING AGENT V	30 seconds + 30 seconds	30 seconds + 60 seconds

[CAUTION]

The etching agent must not be applied to a tin-plated surface since the etchant will damage the plated surface.

2. Pulp protection

Any actual or near pulp exposure could be covered with a hard setting calcium hydroxide material. There is no need for cement lining or basing. Do not use eugenol materials for pulp protection.

3. Pretreatment of metal

- 1) Sandblast the adherend surface until the metal luster disappears; a dull mat sheen is ideal.

Abrasive : 30 - 50 µm alumina particles
Air pressure : 0.4 - 0.7 MPa (4 - 7 kgf/cm²).

- 2) Rinse the restoration completely and clean in an ultrasonic cleaner with water for 2 - 3 minutes.

- 3) If precious or semi-precious metal restorations are used, the adherend surface should also be tin-plated.

[CAUTION]

Do not touch the adherend surface to avoid contamination. If the adherend metal surface is contaminated, for example with saliva, etc., then clean the restoration in an ultrasonic cleaner with water and a neutral detergent for about 2 - 3 minutes. Then wash with a water spray for 1 minute, and dry with air.

4. Pretreatment of porcelain or cured composite resin

Margin adaptation is an important factor especially on occlusal surfaces. Since cement wear is higher than restoration wear, it is recommended that a margin gap of over 50 µm is not accepted for occlusal surface restorations.

[NOTE]

TC shade is recommended for inlays and onlays.

- 1) Sandblast the porcelain adherend surface using 30 - 50 µm alumina particles with an air pressure of 0.4 - 0.7 MPa (4 - 7 kgf/cm²).

Rinse the restoration completely and clean in an ultrasonic cleaner with water for 2 - 3 minutes.

- 2) Then apply phosphoric acid etching agent to the adherend surface for 5 seconds, wash and dry completely.

- 3) Apply CLEARFIL PORCELAIN BOND ACTIVATOR in conjunction with CLEARFIL NEW BOND or CLEARFIL PHOTO BOND according to the instructions for use for the material.

5. Application of ED PRIMER

- 1) Dispense 1 drop each of Liquid A and B, stir for 3 - 5 seconds and apply to enamel and dentin with a sponge pledget. ED PRIMER should remain for 60 seconds.

[NOTE]

ED PRIMER should not be applied to (semi-) precious metal, porcelain or composite.

ED PRIMER should not be rinsed; if rinsed, reapply.

- 2) Evaporate the volatile ingredients using a gentle air flow until the surface appears glossy. A strong air flow should not be used.

[CAUTION]

Do not use a strong air blast, only a gentle air flow.

- 3) Excess ED PRIMER should be removed from the root canal using a small paper point.

6. Preparation of PANAVIA 21 Paste

1) Dispensing

Dispense equal amounts of PANAVIA 21 Catalyst and Universal Pastes.

Slowly turn the dispenser knob 1 complete turn to the right until it clicks. If additional material is required, turn the knob another complete turn to the right.

[CAUTION]

Partial turns should not be used since this is an inaccurate dispensing method. Do not dispense material in partial turns as inaccurate ratios may result.

For detailed descriptions, refer to the Instructions for Use of PANAVIA 21 DISPENSER.

2) Mixing

The pastes should be mixed for 20 - 30 seconds until a smooth, uniform paste results.

[NOTE]

1. PANAVIA 21 Paste has anaerobic setting properties so working time can be increased by spreading the mix thinly over the mixing pad.
2. Working and Setting times are shown below.
Mixing (at 25°C): 20 - 30 seconds
Working time (at 25°C): 4 minutes
Setting time from time of seating (at 35°C): 1 minutes
Complete setting (at 35°C): 3 minutes

7. Application of PANAVIA 21 Paste

Apply a thin layer of mixed PANAVIA 21 Paste to the adherend surface of the restoration being careful to avoid trapped air bubbles.

[NOTE]

Do not apply PANAVIA 21 Paste to primed enamel or dentin since the setting time will be overly fast.

PANAVIA 21 Paste should only be applied to the adherend surface, not to the primed enamel or dentin. ED PRIMER used for the primed enamel and dentin contains a polymerization accelerator which speeds up the setting process of PANAVIA 21 Paste when the primer is contacted.

8. Seating the restoration

Seat the restoration or place the inlay in the cavity and hold with gentle pressure. Seating should be completed in less than 1 minute.

9. Removal of excess paste

Excess PANAVIA 21 Paste should be removed with a small brush, starting with the interproximal surfaces.

10. Application of OXYGUARD II

- 1) Apply OXYGUARD II to all restoration margins with a small brush and leave for 3 minutes.

- 2) OXYGUARD II can be easily removed using a water spray.

11. Finishing

Excess cement can be removed using an explorer or small scaler followed by polishing.

Occlusal adjustments and finishing is then performed as usual.

VIII-2 Bonded amalgam restorations (INDICATIONS FOR USE [5])

[5] Bonded amalgam restorations

1. Cavity preparation

After determining the presence and extent of caries, remove all infected dentin and prepare the cavity as usual.

2. Pulpal protection

Generally there is no need for cement lining or basing. However, any actual or near-pulpal exposure area should be covered with a hard setting calcium hydroxide material.

3. Follow steps 5 and 6 of instructions for use for cementation (VIII-1).

4. Application of PANAVIA 21 Paste

Apply a thin layer of mixed PANAVIA 21 Paste to the entire primed cavity surface being careful to avoid trapping air bubbles. The procedure should be performed in less than 1 minute.

5. Condensation of amalgam

After proper trituration, the mixed amalgam should be placed into the cavity while the PANAVIA 21 Paste is still "wet" and be condensed. Carving the amalgam can be accomplished as usual.

6. Removal of excess paste

Excess PANAVIA 21 Paste should be removed with a small brush, starting with the interproximal surfaces.

7. Follow steps 10 and 11 of instructions for use for cementation (VIII-1).

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proved to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.


[NOTE]

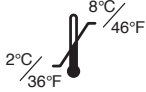If a serious incident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user/patient resides.

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"CLEARFIL", "PANAVIA" and "OXYGUARD" are registered trademarks or trademarks of KURARAY CO., LTD.
„Kuraray Noritake Dental Inc.“ pakeičia visus produktus su įrodymais trūkumais.
„CLEARFIL“, „PANAVIA“ ir „OXYGUARD“ yra „KURARAY CO., LTD“ registruotieji prekės ženklai arba prekės ženklai.