EN	ENGLISH	English	Dental Ceramic
FR	FRANÇAIS	French	Céramique Dentaire
ES	ESPAÑOL	Spanish	Cerámica Dental
IT	ITALIANO	Italian	Ceramica Dentale
NL	NEDERLANDS	Dutch	DENTAAL PORSELEIN
DE	DEUTSCH	German	Dentalkeramik
SV	SVENSKA	Swedish	Dentalkeramik
NO	NORSK	Norwegian	Dentalkeramikk
FI	SUOMI	Finnish	Hammaskeramiikka
DA	DANSK	Danish	Dental Keramik
PT	PORTUGUÊS	Portuguese	Cerâmica Dentária
EL	ΕΛΛΗΝΙΚΑ	Greek	Οδοντικό Κεραμικό Υλικό
TR	TÜRKÇE	Turkish	Dental Seramik
PL	POLSKI	Polish	Dentystyczny materiał ceramiczny
RO	ROMÂNĂ	Romanian	Ceramică dentară
HR	HRVATSKI	Croatian	Dentalna keramika
HU	MAGYAR	Hungarian	Fogászati kerámia
SL	SLOVENŠČINA	Slovenian	Dentalna keramika
CS	ČESKY	Czech	Dentální keramika
SK	SLOVENSKY	Slovak	Dentálna keramika
BG	БЪЛГАРСКИ	Bulgarian	Стоматологична керамика
UK	УКРАЇНСЬКА	Ukrainian	Стоматологічна кераміка
ET	EESTI	Estonian	Dentaalkeraamika
LV	LATVIEŠU	Latvian	Zobārstniecības keramika
LT	LIETUVIŠKAI	Lithuanian	Odontologinė keramika

UK REP	EN	UK Responsible Person	PL	Osoba odpowiedzialna w Wielkiej Brytanii
	FR	Personne responsable au Royaume-Uni	RO	Persoana responsabilă din Marea Britanie
	ES	Responsable en el Reino Unido	HR	Osoba odgovorna za Ujedinjeno Kraljevstvo (UK)
	IT	Responsabile per il Regno Unito	HU	Felelős személy az Egyesült Királyságbán
	NL	Verantwoordelijke in het VK	SL	Odgovorna oseba v Združenem kraljestvu
	DE	Für UK verantwortliche Person	CS	Odpovědná osoba ve Spojeném království
	SV	Ansvarig person i Storbritannien	SK	Zodpovedná osoba v Spojenom kráľovstve
	NO	Ansvarlig person i Storbritannia	BG	Отговорно лице за Обединеното кралство
	FI	Vastuushenkilö Yhdistyneessä kuningaskunnassa	UK	Відповідальна особа у Великій Британії
	DA	Ansvarlig person i Det Forenede Kongerige	ET	Suurbritannia vastutav isik
	PT	Responsável no Reino Unido	LV	Apvienotās Karalistes atbildīgā persona
	EL	Υπεύθυνο Πρόσωπο για το Ηνωμένο Βασίλειο	LT	Atsakingas asmuo Jungtinėje Karalystėje
	TR	Birleşik Krallık Sorumlusu		

Kuraray Noritake Dental Inc.

300 Higashiyama, Miyoshi-cho, Miyoshi, Aichi 470-0293, Japan
TEL +81-561-32-8953 FAX +81-561-32-8976

EC REP

EMERGO EUROPE

Westervoortsedijk 60, 6827 AT Arnhem, The Netherlands

Kuraray Europe GmbH (EU Importer)

Philipp-Reis-Str. 4,
65795 Hattersheim am Main, Germany
Phone: +49 (0)69 305 35835 Fax: +49 (0)69 305 98 35835
URL: <https://www.kuraraynoritake.eu>

CERABIEN™ ZR

I. Introduction

This product is a dental porcelain, which has a potassium-aluminosilicate glass formulation as its primary chemical component. This product is suitable for a wide range of clinical applications, with variable shades. The general clinical benefit of this product is to restore tooth function for the following INDICATIONS FOR USE.

II. Indications for Use

For use in prosthetic dentistry to create an all-ceramic prosthesis.

-Ceramic for coverage of a zirconia substructure

III. Components

For the components (including shades and accessories) for this product, see "CERABIEN ZR" Technical Manual and FC Paste Stain Technical Instructions.

IV. Composition

Powder type (Shade base, Margin, Body, Enamel, Internal Stain, External Stain, etc.): Potassium-aluminosilicate glass (>70 %), inorganic pigments (0-30 %)
 FC Paste Stain: Potassium-aluminosilicate glass (45-65 %), pigments (2-15 %), glycerol (5-15 %), butane-1,3-diol (25-35 %)
 "CERABIEN" ZR FORMING LIQUID: Water (70-90 %), polyethylene glycol (5-30 %), additives (<5 %)
 MEISTER LIQUID: Water (50-70 %), propane-1,2-diol (30-50 %), additives (<5 %)
 IS LIQUID: 2-Phenoxyethanol (>99 %)
 ES LIQUID: Propane-1,2-diol (>99 %)

Units in parentheses are mass %.

For detailed composition, see "CERABIEN ZR" Technical Instructions.

V. Type and Class (ISO6872:2015)

Type: I / Class: 1

VI. Physical Properties

See Table 2.

VII. Directions for Use

For "Baking Schedule" for this product, see Table 1. For detailed information about how to use this product (LIQUIDS to be mixed, the directions for use of FC Paste Stain and "Baking Schedule" for FC Paste Stain, etc.), see "CERABIEN ZR" Technical Instructions and FC Paste Stain Technical Instructions.

Directions for Use A

Use Directions for Use A for the following:

[1] Restorations using a zirconia framework

A-1. Conditioning the zirconia framework surface

If you are using "KATANA Zirconia", condition the zirconia framework surface by following steps (1) and (2) given below. If you are using any dental milling ceramic materials other than "KATANA Zirconia", condition the framework surface according to the manufacturer's instructions for use.

(1) Morphological correction

Check the fit at the inner surfaces and margins between the framework and the abutment on the plaster model. In addition, check the space for building up with porcelain materials, the thickness of the zirconia and any cracks on the framework surface. After checking, grind the entire framework using a diamond bur or a silicone point containing diamond particles. When grinding the framework, work on under running water or while it is wet. Perform detailed anatomical form (crown contours, surface textures, mamelon structures, areas where Margin Porcelain will be applied) if necessary, and polish the areas which come in contact with the opposing teeth.

(2) Sandblasting and Cleaning

Alumina sandblast (at 0.2 MPa or less) the framework surface, excluding the areas which come in contact with the opposing teeth, followed by 10 minutes of ultrasonic cleaning.

A-2. Preparation of the plaster model

Apply a plaster surface hardener to the plaster model in the areas which will come in contact with porcelain (the pontic, abutment margins, opposing model, etc.), dry and then apply Magic Separator over the model for better release.

A-3. Building-up of Shade Base, Opacious Body, and Baking

Use Margin (Shoulder porcelain) if necessary. When building these up, take into account the "triangular structure". Bake the framework. Select either Shade Base or Opacious Body in the shade you need. The Shade Base and Opacious Body can also be mixed. To strengthen the bond of the product to the zirconia, apply a thin coat of Shade Base, Opacious Body or a mixture of both to the entire surface of the zirconia framework (including the margins) to make the first layer, followed by baking. For the second layer, apply Shade Base or a mixture of Shade Base and Opacious Body approximately 0.2 mm thick, or Opacious Body approximately 0.3 mm thick, over the entire surface of the framework, followed by baking.

A-4. Building-up of Cervical and Body

Build-up Cervical or a mixture of Body and Cervical on the cervical area and Body on the contour area. Adjust the shade using Modifier or Mamelon, if necessary. Cut back the proximal and labial surfaces, and then create mamelon structures at the incisal edge. After cutting back, ensure that the Body maintains at least 0.8 mm thickness.

A-5. Building-up of Enamel

Build-up Enamel one-third of the way from the incisal edge, so that the original contour can be restored. Do not apply too much.

A-6. Baking the layered porcelain

Bake the layered porcelain materials (Cervical, Body, Enamel, etc.). The surface of a correctly baked restoration should have a slight gloss. If necessary, create internal details (such as mamelon structures) inside the crown contour at the cutting edges using diamond burs, etc.

A-7. Application of Internal Stain and Baking

Apply Internal Stain if necessary, and then bake the restoration.

A-8. Building-up of Translucent or Luster and Baking

Build-up Translucent or Luster and Clear Cervical (if needed), across the entire surface of the crown contour and approximately 10 percent beyond the target contour of the crown, to allow for shrinkage due to baking. Bake the restoration. The surface of a correctly baked restoration should have a slight gloss.

A-9. Morphological correction

Perform morphological corrections of the crown and adjust the surface contours using a diamond bur (such as a Pro-Tech Point-Ex) or a paper-abrasive material (such as Noritake Meister Cones), followed by cleaning with water. When it is necessary to reproduce a gingival color, apply Tissue (Tissue porcelain), followed by baking.

A-10. Application of Glaze and Stain, Baking and Finishing

Glaze the restoration using either of the following two methods:

- 1) Perform self-glaze baking and polish, if needed.
- 2) Apply a thin coat of External Stain (Glaze), FC Paste Stain (Glaze or Clear Glaze) or *CZR PRESS Glaze, followed by baking. If it is necessary to adjust the color, apply External Stain or FC Paste Stain and bake the restoration. After baking, finish the restoration by polishing the restoration surface and making morphological corrections using a silicone point.
 *CZR PRESS Glaze is a CZR PRESS LF item.

A-11. Additional layering

For additional layering, Margin MRP (Margin repair porcelain) can be applied to the marginal area and Add-on to the crown, followed by baking.

Directions for Use B

Use Directions for Use B for the following:

[2] Anatomical zirconia coping

B-1. Conditioning the anatomical zirconia coping surface and preparation of the plaster model

If you are using "KATANA Zirconia", condition the anatomical zirconia coping surface by following steps (1) and (2) given in "A-1. Conditioning the zirconia framework surface" in Directions for Use A. For preparation of the plaster model, follow steps according to the procedure given in "A-2. Preparation of the plaster model". If you are using dental milling ceramic materials other than "KATANA Zirconia", condition the anatomical coping according to the manufacturer's Instructions for Use.

B-2. Wash firing

To strengthen the bond of the product to the zirconia, apply a thin coat of Internal Stain, Translucent, Luster, etc. to the entire surface of the zirconia coping to make the first layer, followed by baking.

B-3. Application of Internal Stain and Baking

As needed for staining, apply Internal Stain and bake.

B-4. Building-up of Translucent or Luster and Baking

Build-up Translucent or Luster and Clear Cervical (if needed), to cover the entire surface of the crown, and approximately 10 percent beyond the target contour of the crown, to allow for shrinkage due to baking. Bake the coping. The surface of a correctly baked restoration should have a slight gloss.

B-5. Morphological correction

Perform morphological corrections according to the procedure given in "A-9 of Directions for Use A"

B-6. Application of Glaze and Stain, Baking and Finishing

Apply Glaze and Stain, bake and finish according to the procedure given in "A-10 of Directions for Use A".

B-7. Additional application

Make additional application by following the procedure given in "A-11 of Directions for Use A".

Directions for Use C

Use Directions for Use C for the following:

[3] Full-zirconia restorations

C-1. Conditioning the full-zirconia restoration surface

If you are using "KATANA Zirconia", condition the full-zirconia restoration surface by following steps (1) and (2) given in "A-1. Conditioning the zirconia framework surface" in Directions for Use A. If you are using dental milling ceramic materials other than "KATANA Zirconia", condition the restoration according to the manufacturer's Instructions for Use.

C-2. Application of Glaze and Stain, Baking and Finishing

Apply External Stain (FL Glaze or VC Glaze), FC Paste Stain (Glaze or Clear Glaze) or *CZR PRESS Glaze over the restoration, until it is 30µm thick or thicker. Then bake the restoration. If there is insufficient gloss after baking, apply additional layer, and bake the restoration. If it is necessary to adjust the color, apply External Stain or FC Paste Stain and bake the restoration. After baking, finish the restoration by polishing the restoration surface and making morphological corrections using a silicone point.

*CZR PRESS Glaze is a CZR PRESS LF item.

Directions for Use D

Use Directions for Use D for the following:

[4] Restorations (using a zirconia framework) made using CZR PRESS LF (porcelain) and CZR PRESS (pressed ingot)

[5] Restorations (anatomical zirconia coping) made using CZR PRESS LF (porcelain)

D-1. Building-up, Baking or Pressing

[When using a zirconia framework]

Build-up and bake or press according to the Instructions for Use supplied with CZR PRESS LF and CZR PRESS.

[When making the Body portion with zirconia]

Build-up and bake according to the Instructions for Use supplied with CZR PRESS LF.

D-2. Morphological corrections

Perform morphological corrections according to the procedure given in "A-9 of Directions for Use A".

D-3. Application of Glaze and Stain, Baking and Finishing

When using FC Paste Stain (Glaze or Clear Glaze), apply the material thinly over the restoration and bake the restoration. For restorations made using CZR PRESS; however, apply the material until it is 30µm thick or thicker. Then bake the restoration. If there is insufficient gloss after baking, apply additional application, and bake the restoration. If it is necessary to adjust the color, apply FC Paste Stain and bake the restoration. When using LF External Stain (Glaze) of CZR PRSS LF, apply the material and bake according to the product's Instructions for Use. If it is necessary to adjust the color, apply FC Paste Stain and bake the restoration. After baking, finish the restoration by polishing the restoration surface and making morphological corrections using a silicone point.

VIII. Remarks on Handling

Contraindications:

If the patient is hypersensitive to potassium-aluminosilicate glass or any other components, this product must not be used.

Caution and Warning:

1. If the patient or the dental professional demonstrates a hypersensitivity reaction, such as rash, dermatitis, etc., discontinue use of the product and seek medical attention immediately.
2. When mixing materials, or grinding a restoration being fabricated, use an approved dust mask and vacuum with an air filter to protect your lungs from inhaling either dust or liquid.
3. When mixing materials, or grinding a restoration being fabricated, use safety glasses to prevent dust or liquid from getting into your eyes. If dust or liquid gets into your eyes, immediately rinse with copious amounts of water and seek medical attention.
4. Do not use for any purposes except for fabricating dental restorations. This product is for dental application only.
5. The restoration is very hot immediately after baking. Do not touch it with your bare hands for a considerable amount of time after removing it from the furnace.
6. FC Paste Stain, IS LIQUID and ES LIQUID are flammable. Do not use them near fire or high-temperature objects.
7. IS LIQUID is a stimulant liquid. When using it, have good ventilation and if necessary, wear a protective mask, safety glasses and gloves.
8. Stains and plaque can accumulate on a restoration, once it has been put in position in the patient's oral cavity, depending on the patient's eating habits or food choices. Instruct patients about the importance of cleaning their restorations.

Precautions in use:

1. When using a porcelain furnace for the first time, perform a baking test run in advance, since the working temperatures of porcelain furnaces may vary from one device to another.
2. Use only the product with the LIQUIDS named in the Technical Instructions; otherwise, the restoration might change color when it is baked.
3. Do not mix the product with any dental porcelain other than the product or use it in unauthorized combinations. CZR PRESS LF and CZR PRESS may be used with the product if they are used according to the procedures stated in their respective Instructions for Use.
4. Do not use Internal Stain as an alternative to External Stain. Do not use External Stain or FC Paste Stain as an alternative to Internal Stain.
5. Do not apply External Stain over FC Paste Stain.
6. Do not mix External Stain with FC Paste Stain. You may mix Internal Stain shades with each other, or External Stain shades with each other, or FC Paste Stain shades with each other, when making color adjustments.
7. FC Paste Stain may be used for restorations fabricated using "CERABIEN ZR" or CZR PRESS. For detailed information, see the FC Paste Stain Technical Instructions.
8. If the FC Paste Stain in the container has any precipitate at the bottom, stir before use until the contents are uniformly mixed, taking care not to entrap any air bubbles.
9. When adjusting the viscosity of FC Paste Stain, dispense the required amount of FC Paste Stain from the container and mix ES LIQUID with it until you obtain the desired consistency. Do not put the ES LIQUID directly into the FC Paste Stain container.
10. Do not use Margin MRP or Add-on for additional build-up if you are following the C or D directions for use in the Instructions for Use; otherwise, too much luster will be developed.
11. Dispose of this product as a medical waste to prevent infection.

Storage:

1. The product must be stored in a cool and dry place. Keep away from direct sunlight.
2. The product should be stored at 1-30°C (33.8-86°F).
3. After using FC Paste Stain, wipe any remaining paste off the rim of opening of the container and tighten the cap securely to prevent the contents from drying out.
4. The product must be stored in an appropriate place where only dental personnel have access.
5. The product must be used by the expiration date indicated on the package.
6. Do not store FC Paste Stain, IS LIQUID and ES LIQUID near fire or high-temperature objects.

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

[NOTE]

If a serious incident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user/patient resides.
Refill does not include instructions for use (IFU). We provide the IFU in electronic format on the dedicated eIFU website.
"CERABIEN" and "KATANA" are registered trademarks or trademarks of NORITAKE CO., LIMITED.

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IV. Composition

Type de poudre (Shade base, Margin, Body, Enamel, Internal Stain, External Stain, etc.):

Verre aluminosilicate de potassium (>70 %), pigments inorganiques (0-30 %)
FC Paste Stain: Verre d'aluminosilicate de potassium (45-65 %), pigments (2-15 %), glycerol (5-15 %), butane-1,3-diol (25-35 %)
«CERABIEN» ZR FORMING LIQUID: Eau (70-90 %), glycol polyéthylénique (5-30 %), additifs (<5 %)
MEISTER LIQUID: Eau (50-70 %), propane-1,2-diol (30-50 %), additifs (<5 %)
IS LIQUID: 2-phénoxyéthanol (>99 %)
ES LIQUID: Propane-1,2-diol (>99 %)

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*1 96 kPa = 72 cmHg (29 inchesHg)

*2 The baking conditions are established on the assumption that a single crown is baked. Adjust the dry-out time and baking temperature when baking bridges.

*3 This is baking temperature for cases when Opacious Body, Cervical, Body, Enamel, Translucent, Luster, Clear Cervical, Mamelon, Modifier and Tissue are applied additionally.

*4 CZR PRESS Glaze is a CZR PRESS LF item.

*5 On the occasion of use to CZR Press or CZR Press LF surface, please refer to "CERABIEN ZR" FC Paste Stain IFU (Instruction for Use).

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„Kuraray Noritake Dental Inc.“ pakeis bet kokį produktą, kuris tikrai yra sugedęs.