

## YOUR CONTACT

Kuraray Europe GmbH  
BU Medical Products  
Philipp-Reis-Strasse 4  
65795 Hattersheim am Main

Phone +49 (0) 69-305 35 833  
Fax +49 (0) 69-305 98 35 833  
E-Mail [dental@kuraray.eu](mailto:dental@kuraray.eu)  
Website [www.kuraray-dental.eu](http://www.kuraray-dental.eu)



**Kuraray Noritake Dental Inc.**  
1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

## HAPPY WITH THEIR TEETH AGAIN

As you can see, rinsing TEETHMATE™ DESENSITIZER with water from the tooth surface and gingiva shows great results. Your patients will love the neutral taste and the result. And most of all - your patients will be happy with their teeth again.



Source: Dr. Ken Kobayashi, Japan

"TEETHMATE" is a trademark of Kuraray Co., Ltd.

**kuraray**

*Noritake*

**CLINICAL  
INSPIRATION**

**TEETHMATE™ DESENSITIZER**

## CREATING HYDROXYAPATITE

Dr. Kobayashi and Dr. Honda show you how to treat sensitive teeth effectively. And for the first time naturally. TEETHMATE™ DESENSITIZER creates the human body's hardest mineral, hydroxyapatite, to make such treatments effective.

They create hydroxyapatite exactly where it is needed. They use it to close exposed dentin tubules and enamel cracks. And they treat their patients' teeth after bleaching.

Let's see how Dr. Kobayashi and Dr. Honda treat dentinal hypersensitivity after bleaching, after gingival recessions and after scaling and root planing.



Source: Dr. Ken Kobayashi, Japan



Source: Dr. Yuji Honda, Japan

## CASE 1: TEETH TREATMENT AFTER BLEACHING

- ① ② The pictures show the application of TEETHMATE™ DESENSITIZER on the upper front teeth after bleaching.
- ③ After the treatment you can see on the picture that TEETHMATE™ DESENSITIZER doesn't influence the esthetics.

## CASE 2: DENTIN TREATMENT AFTER GINGIVAL RECESSION

- ④ The patient has complained about sensitivity of tooth 23.
- ⑤ First, Dr. Honda cleans the tooth surface and the class V composite with a fine abrasive.
- ⑥ Then he rubs TEETHMATE™ DESENSITIZER on the sensitive area for 30 seconds and washes it away with water.

## CASE 3: DENTIN TREATMENT AFTER GINGIVAL RECESSION

- ⑦ Dr. Honda uses a commercially available interdental brush for a better application to the approximal areas.
- ⑧ It is good practice to use a rubber cup when applying TEETHMATE™ DESENSITIZER over a wide area.

## CASE 4: DENTIN TREATMENT AFTER SCALING AND ROOT PLANING

- ⑨ The patient has complained about sensitivity of tooth 22. First Dr. Honda removes plaque and calculus using an air scaler. He uses a fine abrasive to clean the tooth surface.
- ⑩ He applies TEETHMATE™ DESENSITIZER to the tooth, rubs it in for 30 seconds and washes it away with water.
- ⑪ The TEETHMATE™ DESENSITIZER treatment shows an invisible result.