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MADE IN JAPAN

NET. WT. 3kg

Read the instructions carefully before use.
Please keep the manual for future reference.

△Remarks on Safety

1. Avoid inhaling gypsum dust. Use a dust collector and an approved dust mask.
2. Avoid exposure to eyes. In case of contact with eyes, flush eyes with copious amounts of water and consult a physician.
3. Wear protection goggles for cutting and trimming hardened stones.
4. Avoid ingesting. Keep out of the reach of children.
5. Some people are sensitive to skin contact. Wear rubber gloves to protect your skin.
6. This material is for dental application. Do not use for any purpose not specified in the instruction manual.

△Remarks on Storage and Disposal

1. This material readily absorbs moisture. Keep it sealed and away from high temperature and moisture.
2. Do not mix this powder with any other dental stone.
3. Washing hardened materials or slurry down the drain may clog it. Use a plaster trap.

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— DENTAL SUPER-HARD STONE —

Noritake's finest plaster, SUPER ROCK Ex is a superhard plaster.

FEATURES

1. It has excellent fluidity under the influence of a vibrator, yet can be stacked high.
2. It is slightly flexible, hence resistant to chipping during trimming.
3. Its wet compressive strength is relatively low, facilitating easier model trimming, yet when completely dry, it is super hard.
4. It yields a glossy surface texture with silicon, rubber, alginate or hydrocolloid impression materials.

APPLICATIONS

For models for full or partial dentures, crowns or bridge, etc.

INSTRUCTIONS

1. Add 20ml water per 100g powder.
2. Mix with a vacuum mixer for 60 seconds.
3. Use a vibrator during the pour to prevent bubbles.
4. It can be removed from the impression after 30-40 minutes.

PROPERTY

Mixing Water (ml water/100g powder)	20
Setting Time (minute)	10
Setting Expansion (%)	0.08
Compressive Strength...1 hour later (MPa)	50.0
Compressive Strength...dry (MPa)	100.0

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