

Esthetic White & New Shades *CZR* *ERABIEN* *CZR* *ERABIEN* *PRESS*

Whiter than our conventional bleach shades for beautiful smile!

Esthetic White Shades



EW Shades compared to NW0

	Opacity degree	Color
EW00	4 times higher than NW0	White
EW0	2 times higher than NW0	
EW	The same as NW0	
EWY	The same as NW0	Very lightly yellow-tinted



■ Esthetic White Shade Guide



■ Noritake Shade Guide

Color Combination Table

	EW00	EW0	EW	EWY
KATANA Zirconia	KT10			KT15
Shade Base /Shade Base Stain	SB White / SS White (if necessary) ※			SBB1 / SSB1
Body /Press Ingot	EW00B / EW00	EW0B / EW0	EWB / EW	EWYB / EWY
Enamel	Silky E1		Silky E2	
Luster Translucent	ELT2		ELT1	
External Stain	—			B+
Glaze Powder	CZR PRESS Glaze Powder			

※ Use "SS White" or "SB White" in case you need to mask the abutment color or to emphasize the whiteness.

EW Line-ups CZR

Item	Shade	Application
Shade Base	SB White	For the case when you need to emphasize the whiteness.
Body	EW00B	EW00 has the highest opacity, followed by EW0, EW and EWY.
	EW0B	
	EWB	
	EWYB	
Enamel	Silky E1	Recommendable for EW00 and EW0 shades To express the calcification of natural enamel To produce very bright bleached enamel
	Silky E2	Recommendable for EW and EWY shades
Luster Porcelain	ELT1	Recommendable for EW and EWY shades
	ELT2	Recommendable for EW00 and EW0 shades
	ELT3	For the case when you need to emphasize the whiteness

EW Line-ups CZR PRESS

Item	Shade	Application
Shade Base Stain	SS White	For the case when you need to emphasize the whiteness.
Ingot EW	EW00	EW00 has the highest opacity, followed by EW0, EW and EWY.
	EW0	
	EW	
	EWY	

CZR Clear Cervical & IS Gray

		Application
Clear Cervical	CCV-1	For brighter shades like A1 to A3 shades
	CCV-2	For darker shades like A3.5 to A4 shades
	CCV-3	For creating the color of the exposed tooth root
	CCV-4	More reddish than CCV-3 For creating anterior lingual fossa
Internal Stain	Gray	For adjusting value



■ CZR Esthetic White Kit



■ CZR Press Esthetic White Kit



■ CZR Professional Kit

Noritake

Kuraray Noritake Dental Inc.
300 Higashiyama, Miyoshi-cho, Miyoshi, Aichi 470-0293, Japan
Phone +81-561-32-8953 Fax +81-561-32-8976
<http://www.kuraraynoritake.com>